



BigHorn™ Ramp “Shim-Johnny” Kit

Installation Instructions

Materials Needed: Measuring Tape and Carpet/Utility Knife

- 1). Measure threshold height to determine how much of the Shim material you will need to use (Fig. 1).
- 2). Carefully cut Shim strips at the thick end to desired height/thickness using scissors or a carpet/utility knife (Fig. 2).
- 3). Peel the tape backing off of the Shim strips (Fig. 3). Place Shim strips equal distances apart on your substrate, covering the length of the ramp (Fig. 4).
- 4). Remove the tape backing from the underside of your BigHorn™ Ramp (Fig. 5).
- 5). Place the BigHorn™ Ramp on top of the Shim strips to secure the bond between the ramp, the Shim strips and the substrate.
Be sure the back end of the BigHorn™ (the thickest end) is flush with the door threshold (Fig. 6).
- 6). We recommend you place a heavy object atop the ramp for 15-20 minutes to ensure the ramp properly adheres.
- 7). Your BigHorn™ Ramp is now ready to use!



Fig. 1

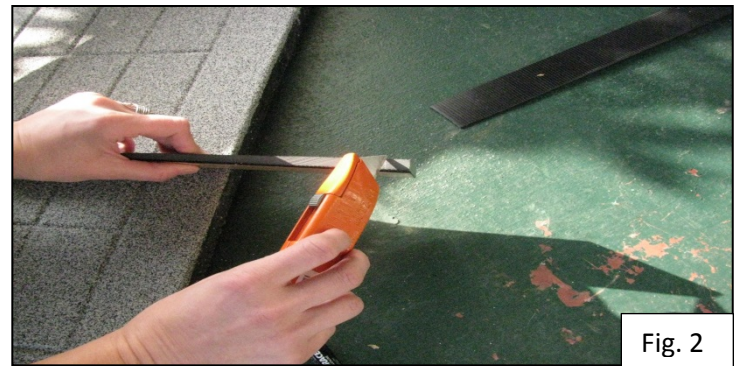


Fig. 2



Fig. 3

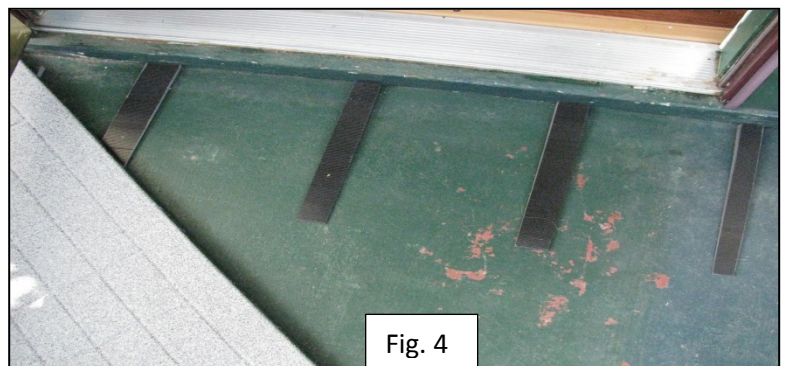


Fig. 4



Fig. 5



Fig. 6