DELABIE









5104...

NT 5104

Indice C

510420 / 510429 / 510434

510429 / 510430 / 510436

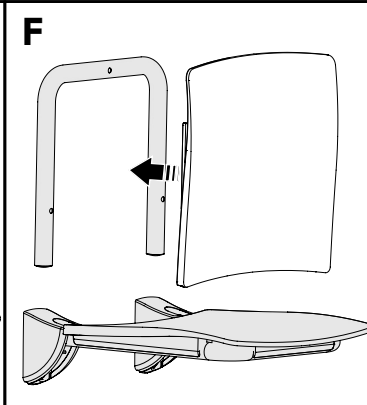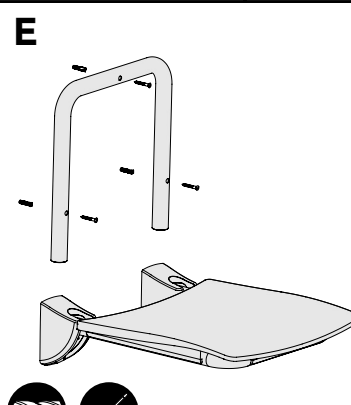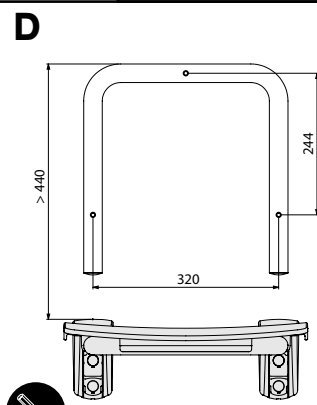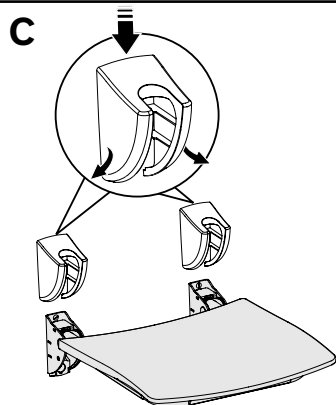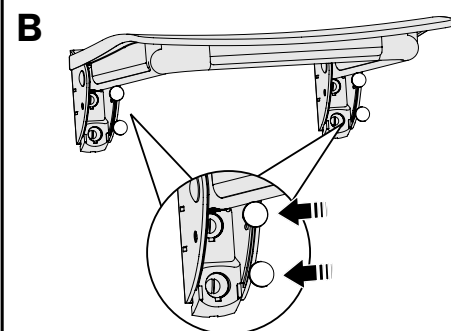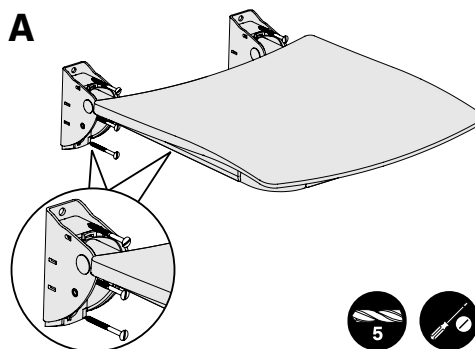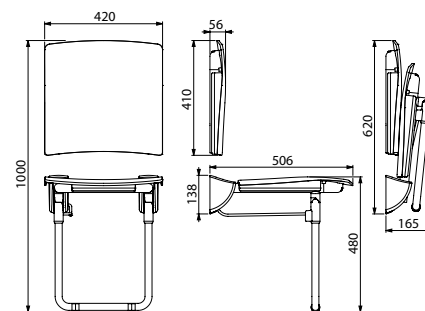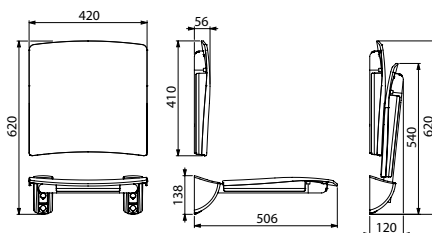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FR

The shower seat should be installed by professional installers according to current regulations and recommendations in your country. The screws provided are intended for concrete walls. The choice of screws and rawl plugs used should be suitable for wall mounting and should be reinforced if necessary.

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SHOWER SEAT

Recommended mounting height for models without leg: position the seat 460-500mm from the floor.
Maximum user weight: 135kg for models without legs, and 170kg for models with legs.
To avoid any risk of slipping, we recommend installing a grab bar on either side of the seat.

Fig. A : Fix the seat to the wall and lower the seat into the horizontal position.

Fig. B : Insert the screw cap covers.

Fig. C : Place the Nylon covers over the fixing points by pulling the sides slightly apart and pushing them downwards and against the wall. Clip the covers onto the fixings.

SHOWER SEAT BACK REST

• Fix the frame to the wall.

Important : If installed with a Delabie shower seat (**Fig. D, E**), the installation height must be strictly observed to avoid obstructing the seat when it is being raised.

• Clip the seat back rest onto the frame (**Fig. F**).

MAINTENANCE AND CLEANING

Clean using a damp, soft cloth or with cleaning products and disinfectants.
Do not use acid-based products or scouring powder

Product conforms to European standard EN 12182 and the Directive 93/42/CEE.

DELABIE

After Sales Care Support:

For the UK market only: Tel. 01491 821 821 - e-mail: technical@delabie.co.uk

The installation guide is available on: www.delabie.co.uk

For all other markets: Tel. +33 (0)3 22 60 22 74 - e-mail: sav@delabie.fr

The installation guide is available on: www.delabie.com

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EN

Maximum user weight: 135kg.

- **Fig. A :** Assemble the parts, using the screws supplied. Ensure that the screws are inserted correctly into the tubes.
- **Fig. B :** Clip the 2 slats ① and ② with the cut away sections onto the horizontal tubes, then clip the 3rd slat with no cutaways between the two outer slats ③.

Clean with a soft, damp cloth or with current cleaning products and disinfectants. Do not used acid based products or scouring powder.

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