B.Scan Evo

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Česky

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English

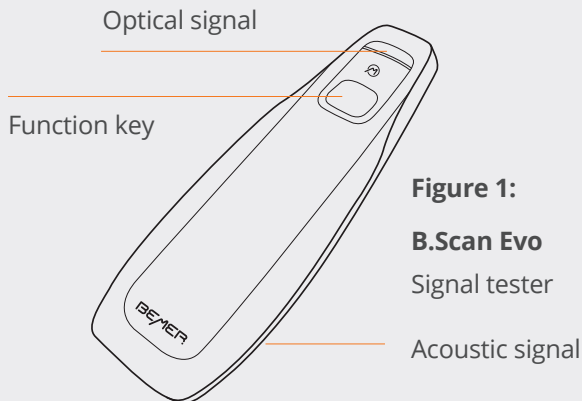
Figure 1:
B.Scan Evo
Signal tester

The B.Scan Evo is a signal tester for magnetic fields that lets you check the function of your BEMER application modules (PEMF) as well as their safety features. The B.Scan Evo makes the multidimensional BEMER signal structure audible and is therefore also an effective demonstration tool for prospective customers. The B.Scan Evo can be used with Human-Line and Horse-Line products.

How to use the B.Scan Evo:

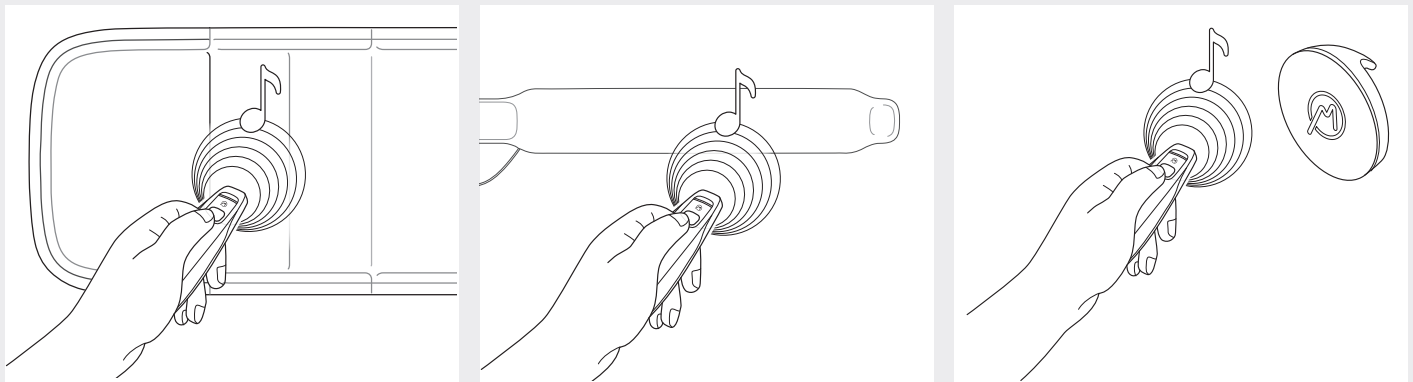
Figure 2: Examples showing how to use the signal tester to check that your application modules are functioning correctly – or for effective demonstrations.

Pressing the button once switches the device on, pressing it for longer (several seconds) switches it off again. If a signal structure is not detected, the B.Scan Evo switches itself off automatically after 10 seconds.

Replacing the battery in the B.Scan Evo:

- 1** Loosen the two screws (see Fig. 3A).
- 2** Now carefully remove the back (see Fig. 3B).
- 3** Slide the old battery sideways (in the direction of the arrow) out of its holder (see Fig. 3C).

Then slide the new battery (type CR 2032) into the holder. Note the “+/-pole” sign (see Fig. 3C).

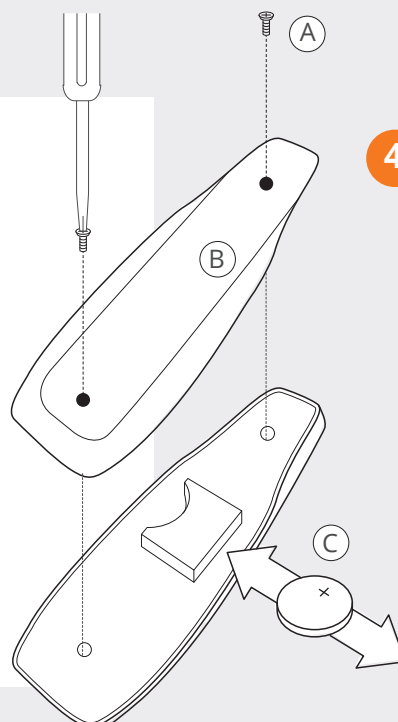
Figure 3:

- 4** Reattach the back of the device and screw it back together.

Subject to change without notice
GC-3468 2180-2307
© BEMER Int. AG
LI-9495 Triesen

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