DEUTSCH

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Please read these instructions for use carefully

and keep them for later use, make them accessi ble to other users, and observe the information

1. GETTING TO KNOW YOUR DEVICE

Many women experience abdominal cramps and severe menstrual pain before and during their period. On top of that, a lot of women also suffer

rom illnesses which cause pain in the lower abdomen, such as endometriosis. This pain is often so severe that women seek immediate pain re-lief. The innovative Menstrual Relax is a self-adhesive TENS device with

a heat function to provide pain relief for menstrual discomfort and pain

caused by endometriosis. The TENS technology used in the Menstrual

To alleviate pain even more pleasantly, a soothing heat function can be

activated. Made from medical-quality silicone, the elegantly designed Menstrual Relax is extremely slim and flexible, and adapts perfectly to

the area to be treated. The battery ensures energy-efficient treatments of 20 minutes each (up to 6 treatments are possible per battery charge). The Menstrual Relax is a safe and easy way of relieving pain, and offers

users a completely new experience in dealing with lower abdominal pair

TENS, or transcutaneous electrical nerve stimulation, refers to the electrical stimulation of nerves through the skin. TENS is an effective non-phar-

macological method for treating different types of pain from a variety of causes. It has no side effects if administered correctly. The method

has been clinically tested and approved and can be used for simple

self-treatmintary tested and approved and can be used or simple self-treatment. The pain-relief or pain-suppressing effect is achieved by inhibiting the transfer of pain within nerve fibres and by increasing the

secretion of endorphins within the body, as their effect on the central

nervous system reduces the sensation of pain. The method has been scientifically substantiated and approved as a form of medical treatment.

Any symptoms that could be relieved using TENS must be checked

by your GP. Your doctor will also give you instructions on how to carry out a TENS self-treatment regime. TENS has been clinically tested and

approved for menstrual discomfort and for the relief of pain caused by

TENS devices work by passing electrical currents over the skin. The gel

pads are used as a transfer medium and are subject to natural wear and

tear. The gel pads must be replaced when they stop providing sufficient contact and the Menstrual Relax no longer sticks to the skin completely.

Failure to replace the pads may lead to skin irritation as a result of height

Check that the exterior of the cardboard delivery packaging is intact and

make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging

naterial has been removed. If you have any doubts, do not use the de

The following symbols are used on the device, in these instructions for

lectronic Equipment EC Directive - WEEE.

Do not dispose of batteries containing harmful sub-

Warning indicating a risk of injury or damage to your

Safety note indicating possible damage to the device

The device can emit effective output values above

10 mA, averaged over every 5-second inte

use, on the packaging and on the type plate for the device

stances with household waste

Note on important information

Manufacturer

Date of manufacture

ice and contact your retailer or the customer service address pr

2. INCLUDED IN DELIVERY

3. SIGNS AND SYMBOLS

Ix Menstrual Relax

Pb Cd Hg

 \mathbf{A}

(j)

4x self-adhesive gel pads

1x these instructions for use

What is TENS and what can it do?

elax can treat pain in a targeted manner. The Menstrual Relax has a

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Ivanically isolated applied part (F stands for "float-

Do not use if you have implanted electrical devices (e.g.

This product satisfies the requirements of the applicable

Device protected against foreign objects ≥12.5 mm and

Observe the instructions
Read the instructions before starting work and/or

Permissible operating temperature and humidity

Permissible transport temperature and humidity

Marking to identify the packaging material.

Serial number

Medical device

Article number

Type number

ous electrical nerve stimulation).

4. INTENDED USE FOR TENS

The device is intended to treat pain using TENS technology (transcutane

This device is intended for self-treatment in the home, not in professional healthcare facilities. The application is suitable for all adults.

on the front of the neck (particularly not the carotid sinus), or

the two large pectorals. This can increase the risk of ventricular

Pain relief for menstrual discomfort and pain due to endometriosis

MD

REF

Target group

Indications

Clinical benefits:

Period pain in womer

Contraindications:

Pain due to endometriosis

Stimulation should not be carried out

on or through the head.

directly on the eyes,

over the mouth.

across the heart.

Do NOT use the device

In the genital area.

es (e.g. a pacemaker).

In the neck/carotid artery area

If you have implanted electrical devic-

If you have metal implants (e.g. IUDs).

A = material abbreviation, B = material number 1–7 = plastics, 20–22 = paper and cardboard

Separate the product and packaging elements and

dispose of them in accordance with local regulations

Separate the packaging elements and dispose of them

Permissible storage temperature and humidity

ing"); meets the requirements for leakage currents for

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wash the affected area with water and seek medical assistance.

Choking hazard! Small children could swallow and choke on batteres. Therefore, keep batteries out of the reach of small children! If a battery has leaked, put on protective gloves to handle the device.
 Protect batteries from excessive heat. Risk of explosion! Never throw batteries into a fire

Do not disassemble, split or crush the batteries.

Only use chargers specified in the instructions for use Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.

Fully charge the battery prior to initial use (see section 6). In order to achieve as long a battery service life as possible, fully charge the battery at least every 6 months.

∠! CAUTION Protect the device from heavy impacts


 Do not use the device if it shows signs of damage or does not function properly. In these cases, contact Customer Services.
 Do not expose the device to direct sunlight or high temperatures. · Protect the device from dust, dirt and moisture. Never immerse the device in water or other liquids.

Do not use this device by or in the vicinity of other heating devices, heating elements such as dryers or ovens, and do not use it in the vi-cinity (-1 m) of shortwave or microwave devices (e.g. mobile phones), as doing so can result in unpleasant current peaks.

If the device is damaged or you have any doubts, do not use it and contact your retailer or Customer Services at the address provided. Switch the device off immediately if it is faulty or not working properly. Never attempt to open and/or repair the device vourself. may only be carried out by Customer Services or authorised retailers. Failure to comply will invalidate the warranty.

2. LED heat/battery level indicator 3. TENS LED 4. ON/OFF button 5. Decrease intensity 6. Left electrode 7. Heating surface 8. Right electrode USB-C connection

6. DEVICE DESCRIPTION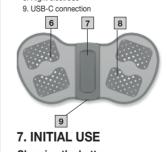

Charging the battery (i) Note Before using the Menstrual Relax for the first time, let it charge for a minimum of 2 hours. To do so, proceed as follows:

1. Open the cover of the USB-C connection on the Menstrual Relax

provided to a USB mains adapter (output: max. 5V/2A, not included in delivery) and the Menstrual Relax. 3. Then insert the mains adapter into a 4. Alternatively, you can charge the Menstrual Relax by using your computer/laptop. To do so, connect the Menstrual Relax to a USB port on your computer/laptop, using the USB

2. Connect the USB-C charging cable

5. Whilst charging, the battery level indicator LED flashes green. As soon as the Menstrual Relax has been completely charged, the LED battery level indicator will be continuously with the electrode surfaces placed on the chest, upper back or illuminated in green The device must never be used near to the heart. The stimulation electrodes must not be placed on any part of the front ribcage (where the ribs and breastbone are located), and especially not on You cannot use the Menstrual Relax while it is being charged. Make sure that there is no skin contact with the electrodes during th

charging cable

ibrillation and induce cardiac arrest (see Chapter 5 General On the skeletal skull structure, or around the mouth, throat or Attaching the gel pads

If you have a known or acute cardiac arrhythmia or disorders of 1. Remove one of the protective films On acutely or chronically diseased (injured or irritated) skin (e.g.

If you use an insulin pump.

If you are pregnant.

If you have cancer

the healing process

If you have a high temperature (e.g. > 39°C).

the heart's impulse and conduction system.

If you suffer from a seizure disorder (e.g. epilepsy

Warning! Undesirable side effects

5. GENERAL WARNINGS

General warnings

ecurrent malignant growths.

If you have any skin diseases or open wounds

If you are receiving medical treatment

Only use the Menstrual Relax:

how to use the device. Children must not play with the device.

bathing or showering.

Do not use after consuming alcohol.

ence pain, stop using it immediately.

revent unintentional stimulation

do this could result in spot burns.

trical impulses to your body.

General precautions

use. Any form of improper use can be dangerous.

On people.

If you have unexplained chronic pain in any part of the body.

If you suffer from diabetes.

If you have any sensory impairment that reduces the feeling of pain

In the event of complaints linked to the stimulation treatment. If you suffer from persistently irritated skin due to long-term stimulation at the same electrode site.

For the intended purpose and as specified in these instructions for

For external use only.

With the original accessories supplied, which can be re-ordered. Failure to do so invalidates the warranty.

This device is NOT intended for use by people with restricted phys-

Do not use in humid environments (e.g. in the bathroom) or when

bile phones), as doing so can result in unpleasant current peaks.

Do not expose the device to direct sunlight or high temperatures.

For hygiene reasons, the electrodes may only be used on one person.

If the device does not work properly, or if you feel unwell or experi-

Switch off the device first before removing or relocating electrodes to

Do not modify the Menstrual Relax (e.g. by trimming). This increases

the current density, which is potentially hazardous (max. recommend ed output value for the electrodes is 9 mA/cm², an effective current density beyond 2 mA/cm² requires increased awareness).

Make sure that the electrodes are in full contact with the skin.

· If the electrodes wear out, skin irritation can occur as an even distri-

bution of current over the entire surface is no longer guaranteed. For

Do not use whilst asleep, driving a vehicle or operating machinery.

Do not use whilst undertaking any activity where an unexpected re-

action (e.g. strong muscle contractions even at low intensity) could

be dangerous.

Ensure that no metallic objects (e.g. belt buckles or necklaces) come

into contact with the electrodes during stimulation. If you are wear

ing jewellery or have piercings in the area to be treated (e.g. a navel

piercing), these must be removed before using the device; failure to

Keep the device away from children to prevent potential hazards.

Do not use this device whilst using other devices that transmit elec-

Do not use in the vicinity of highly flammable substances, gases or

During the initial few minutes, use the device while sitting or lying

down to minimise the risk of accidental injuries as a consequence of isolated cases of vagal responses (feeling of faintness). If you feel

faint, switch off the device immediately, lie down and support the legs

in an elevated position (approx. 5-10 minutes).

Treatment of the skin with moisturising lotions or ointments before-

hand is not recommended as this considerably increases the elec-

Store the device in a dry place (use indoors only). To avoid the risk

The actual temperature of the Menstrual Relax may vary depending

on the condition of your skin, your age, the location of the pain, etc. If the heat function feels too hot, stop treatment immediately. You can

Wait at least one hour after eating before starting treatment with the

can vary greatly depending on the skin type and structure).

- Use the gel pads on clean, dry skin and avoid lotions or greasy

The gel pads can be used for a maximum of 20 applications (this

· If your skin or eyes come into contact with fluid from a battery cel

wash the affected area with water and seek medical assistance

If the device is damaged, do not use it and contact your retailer or the specified Customer Services address.

To ensure that the device functions effectively, do not drop it or dis-

Check the device for signs of wear and tear or damage. If there are

such signs of wear and tear or damage or if the device was used

Switch the device off immediately if it is faulty or not working properly

Never attempt to open and/or repair the device yourself. Repairs may only be carried out by Customer Services or authorised retailers. Fail-

The manufacturer is not liable for damage resulting from improper

If your skin or eyes come into contact with fluid from a battery cell,

ure to comply with this instruction will void the warranty.

properly, it must be returned to the manufacturer or retailer before

continue the TENS treatment without the heat function.

Consult your doctor if you have sensitive skin

Do not use on open wounds.

Damage

WARNING

Notes on handling batteries

of fire and/or electric shock, protect the device from high humidity

trode wear and may cause unpleasant current peaks.

Keep packaging material away from children (risk of suffocation!).

Protect the device from impact, dust, dirt and moisture.

Never immerse the device in water or other liquids

this reason, the electrodes should be replaced.


inflamed skin - whether painful or not reddened skin rashes

After an operation, if strong muscle contractions could affect

If you have an acute or chronic disease of the gastrointestinal

If you are connected to a high-frequency surgical device.

(e.g. allergies), burns, bruises, swellings, both open and healing wounds, and post-operative scars that are healing).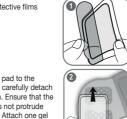

2. Carefully attach the gel pad to the Menstrual Relax and then carefully detach the second protective film. Ensure that the edge of the gel pads does not protrude over the Menstrual Relax. Attach one gel

WARNING

Remove the protective film slowly and carefully. Make sure that the self-adhesive gel pads are not damaged, as damage or unevenness on

8. USAGE

The device is not a substitute for medical consultation and treat-ment. Always consult your doctor first in the event of pain or illness. Starting use Before using the device, consult your doctor if any of the following (i) Note

apply to you:

If you suffer from an acute medical condition, in particular if you In total, the Menstrual Relax has one heat setting with heat generation up in local, the winestruct neterior has one need setting with near generation by to 43 °C maximum, and 15 intensity levels for TENS application. We recommend you start off using a low intensity level until you have got used suspect or have been diagnosed with high blood pressure, a blood agulation disorder, propensity to thrombo-embolic conditions or to the device. The Menstrual Relax has an automatic switch-off function After 20 minutes of use, it automatically switches itself off.

1. Stick the Menstrual Relax onto your

lower abdomen. You can attach the pad flexibly depending on where the pain is.

Just make sure that the electrodes do not come into contact with the hip bones. 2. Briefly hold down the ON/OFF button 🖰 to switch on the pad. The heat LED flash

to heat up. If the heat LED flashes red,

please charge the Menstrual Relax as described in the "Charging the battery" section. 3. To switch on electrical nerve stimulation (TENS), briefly press the ON OFF button () again. The TENS LED flashes. To start stimulation press the Λ button once. The lowest intensity is now activated. If you want to use the TENS function without heat function, press the

ical, sensory or mental skills or a lack of experience and/or a lack of ON/OFF button (b) again. Now only the TENS LED is flashing and you knowledge, unless they are supervised by a person who has respon can start stimulation by pressing the ^ button. sibility for their safety or they receive instructions from this person on Select your desired intensity level using the A/V buttons. 6. To switch off the Menstrual Relax, press and hold the ON/OFF button \circlearrowleft for 3 seconds. The heat LED and the TENS LED go out. The device is only intended for domestic/private use, not for com-

Switch off the Menstrual Relax before removing it. After using the Menstrual Relax, take care when removing it from your lower abdo men. Reapply the protective films to the gel pads to protect them

Notes on use

Wait at least 30 minutes between applications.
Only use the Menstrual Relax as often as you find comfortable. If Always pull gently on electrodes to remove them from the skin to prevent injuries in the unusual case of highly sensitive skin.
Hold the device away from sources of heat and do not use it in close treatment with the Menstrual Relax starts to feel unpleasant, stop proximity (approx. 1 m) to shortwave or microwave devices (e.g. mo-

The Menstrual Relax has a function which detects whether the device is in contact with the body. If there is not sufficient contact with the body, the device beeps and switches itself off. If the device is attached to your body but still beeps, clean the gel pads and try starting the device again. If the continuous beeping persists, replace the gel pads.

If the adhesive capability of the gel pads decreases, replace them

immediately. Do not use the Menstrual Relax again until the gel pads have been replaced. Otherwise the uneven adhesion of the gel pads could lead to skin lesions. In the event of unpleasant current peaks, switch off the Menstrua

Relax and move it slightly.

If you find that using the Menstrual Relax causes you to have a much heavier flow, stop using it. **WARNING!**

Do not apply the electrodes to the front wall of the chest | Impulse width | 100 μs | 250 μs | 75 μs | Impulse frequency | 100 Hz | 8 Hz | 85 Hz |

Cycle duration 120 s 120 s 160 s Impulse frequency

Period _ The frequency indicates the number of ndividual impulses per second and is displayed in Hz (Hertz). It can be calculated by determining the cyclic value for the time

Impulse width This indicates the duration of an individual impulse in microseconds. The impulse width therefore determines, among other

things, the penetration depth of the electricity, where usually larger muscle mass

9. CLEANING AND MAINTENANCE WARNING

Clean the device after use with a soft, slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution. Ensure that no water enters the device. If this happens, only use the device again once it has fully dried out.

To ensure that the adhesive electrodes retain their adhesion for as long as possible, clean them carefully with a damp, lint-free cloth. Do not use any chemical or abrasive cleaning agents for cleaning

10. ACCESSORIES AND REPLACEMENT **PARTS** For accessories and replacement parts, please visit www.beurer.com or

contact the corresponding service address in your country (according to the service address list). Accessories and replacement parts are also available from retailers. 1 x self-adhesive gel pad set 110.008

1 x USB-C cable 11. WHAT IF THERE ARE PROBLEMS?

Problem	Possible cause	Solution
The device does not switch on when the ON/OFF button is pressed.	The battery may be flat.	Fully charge the battery. If it is still not possible to switch the device on, please contact Customer Services.
The device does not stick to the body.	The gel pads may be dirty or worn. The treatment area of the skin may not be clean enough.	Clean the adhesive surface of the gel pads using a damp, lint-free cloth. The gel pads must be replaced if they still do not stick securely. Clean the skin prior to any application; do no use skincare lotions or oils prior to treatment. Shaving may increase the life of gel pads.
There is no noticeable stimulation.	The gel pads may be dirty or worn. The battery may need charging.	Ensure that the gel pads are in firm con- tact with the treatment area. The battery is almost flat. Charge it fully.
An unpleasant sen- sation is experienced where the gel pads are applied.	The gel pads are worn. This may cause skin irritation, as even distribution of the current across the entire area is no longer guaranteed.	Replace the gel pads.
Skin in the treatment area turns red.	Redness under the gel pads that disappears quickly is not danger- ous, and is caused by the locally stimulated, increased blood flow caused by TENS and heat.	Immediately stop treatment and wait until your skin has returned to its normal condition. However, consult your doctor before you continue treatment if the skin irritation persists and if it is accompanied by an itchy sensation or inflammation. This may be caused by an allergic reaction to the adhesive surface.
The skin under the heat surface turns red.	Reddening under the heat surface of the Menstrual Relax can be explained by the heavily stimulated blood flow in the heated tissue.	The skin should return to its normal condition a few minutes after treatment. However, consult your doctor before continuing to use the heat function if the redness persists and if you experience any itching or inflammation.
The Menstrual Relax becomes too hot.		Press the ON/OFF button $\textcircled{1}$ to stop the heat function and just continue using the TENS function.

The empty, completely flat batteries must be disposed of in specially designated collection boxes, at recycling points or at electronics retai s. You are legally required to dispose of the batteries. The codes below are printed on batteries containing

12. DISPOSAL

harmful substances: Pb = battery contains lead Cd = battery contains cadmium Hg = battery contains mercury For environmental reasons, do not dispose of the device in household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive Waste Electrical and Electronic Equipment (WEEE). If

you have any questions, please contact the local authorities 13. TECHNICAL SPECIFICATIONS

Pb Cd Hg

Name and model EM 55

	Туре	EM 50
	Output waveform	Biphasic rectangular impulses
	Impulse duration	75-250 μs
	Impulse frequency	8-100 Hz
	Output voltage	Max. 80 Vpp (at 1000 Ω)
	Heat settings	1 (up to 43°C)
	Output current	Max. 80 mApp (at 1000 Ω)
	Voltage supply	Lithium polymer battery 1000 mAh
	Treatment time	20 minutes; the device then switches off automatically
	Intensity	Adjustable from 0 to 15
ne	Operating conditions	5°C – 40°C (41°F – 104°F) at a relative air humidity of 15–90%
	Storage conditions	0°C - 40°C (32°F - 104°F) at a relative air humidity of 0-90%
	Dimensions	Approx. 116 x 198 x 19.7 mm
	Weight	86.1 g

Subject to technical changes e serial number is located on the device.

It is also not necessary to carry out safety checks in accordance with 14. NOTES ON ELECTROMAGNETIC

found at the end of the instructions for use.

COMPATIBILITY

WARNING

The device is suitable for use in all environments listed in these in tructions for use, including domestic environments.

The device may not be fully usable in the presence of electromagnetic disturbances. This could result in issues such as error messages o

This device complies with the European standard EN60601-1-2 (Group 1

Class B, in compliance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3

IEC 61000-4-8) and is subject to special precautionary measures with regard to the electromagnetic compatibility. Please note that portable and

mobile HF communication systems may interfere with this device. More details can be requested from the stated Customer Service address or

For this device, a functional test and instruction in accordance with Sec

tion 5 of the Medical Devices Operator Ordinance (MPBetreibV) is no

usurbalines. This count result in issues such as error messages or the failure of the display/device.

• Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are

working properly.

The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity this can result in faulty operation.

Keep portable RF communication devices (including peripheral

equipment, such as antenna cables or external antennas) at least 30 cm away from all device parts, including all cables included in delivery. Failure to comply with the above can impair the performance

Failure to comply with the above can impair the performance of the

15. WARRANTY/SERVICE

Further information on the warranty and warranty conditions can be found in the warranty leaflet supplied.

Subject to errors and changes