



GL 48 / LD 04


Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany • www.beurer.com
www.beurer-gesundheitsratgeber.com • www.beurer-healthguide.com



Lancet needles / Lanzetten / lancettes / lancetas / lancette:
SteriLance Medical (Suzhou) Inc., No.168 PuTuoShan Road
New District, 215153 Suzhou, Jiangsu
PEOPLE'S REPUBLIC OF CHINA



Emergo Europe B.V., Westervoortsedijk 60
6827 AT Arnhem, The Netherlands

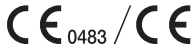

Warnings and safety notes

Before first use



Always read the full instructions for use before taking your first blood glucose measurement. Check the measuring unit in accordance with the instructions for use.


Risk of infection



The monitor and accessories may come into contact with human blood and are therefore a possible source of infections.

- Never use a monitor, lancing device or accessories together with other people.
- When using the measurement system on different people, please observe the general hygiene regulations.
- Use a new, sterile lancet needle for taking each blood sample.
- Prior to taking the blood sample, wash your hands with warm water and soap, and then dry them.

Preparing the monitor and lancing device

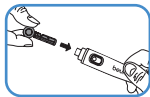

Inserting the lancet needle


Turn off cap from the lancing device. Insert lancet needle in the lancing device. Hold the shaft of the lancet needle and remove protective disc by turning. Retain the protective disc. Reapply the cap to the lancing device. Turn the moving upper part of the cap until the desired penetration depth is displayed. Pull the tensioning device back until it engages.



Inserting the test strip

Insert the test strip into the monitor with the electrode first. The start screen is shown for a short moment. Then wait briefly until the flashing blood droplet symbol can be seen on the display.

Taking a blood sample

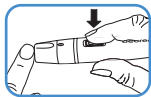

Pricking your finger

Firmly position the lancing device slightly to the side of the centre of the fingertip. Press the trigger.

The blood sample taken must:


- be in droplet form and must not be smeared
- be at least 1.2 mm (original size ●).

Evaluating critical measured values

Lo		Very low blood glucose level < 20 mg/dL (< 1.1 mmol/L)	Seek medical attention immediately.
65 mg/dL	3.6 mmol/L	Low blood glucose level < 70 mg/dL (< 3.9 mmol/L)	Have a snack. Follow your doctor's instructions.
150 mg/dL	8.3 mmol/L	High blood glucose level On an empty stomach > 100 mg/dL (> 5.6 mmol/L) 2 hours after a meal > 140 mg/dL (> 7.8 mmol/L)	Seek medical attention.

300 mg/dL	16.7 mmol/L	High blood glucose level, possibly ketones >240 mg/dL (> 13.3 mmol/L)	Ketone test recommended. Consult your doctor.
HI		Very high blood glucose level >630 mg/dL (> 35.0 mmol/L)	Take another measurement using a new test strip. If the result is identical, seek medical attention immediately.

Information on the display

LP	Batteries empty.	Replace all batteries.
Ht / Lt	Temperature is outside the permitted range.	Repeat the test with a new strip at room temperature +20 °C to +26 °C.
Err 	Used or contaminated test strip.	Insert a new test strip that has not expired and repeat the measurement.
Err001	System error.	Remove batteries. Insert batteries again. Should the problem persist, contact Customer Services.
Err002	Insufficient blood on the test strip.	Repeat the measurement using a new test strip.
Err004	Measurement process malfunction.	Check the necessary steps before the blood glucose measurement and repeat the measurement using a new test strip.