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ENGLISH

PRODUCT DESCRIPTION

This plug in power supply converts the AC mains voltage to a DC low voltage. This power supply can be used with electronic medical devices.

OPERATION

This power supply unit is operational as soon as it is plugged into a mains socket. Using this power supply outside of the nominal rating may result in failure.

TECHNICAL DATA

Input: 100-240V 50-60Hz

Output: 6V DC 600mA only in connection with Beurer blood pressure monitor BM 23, BM 28, BM 40, BM 44, BM 45, BM 49, BM 55, BM 58, BM 60, BM 65, BM 75, BM 77

Protection: This device is double insulated and protected against short circuit and overload by a primary fuse.

SAFETY INSTRUCTIONS

- Keep the charger in a dry place (indoor use only). In order to avoid the risk of fire and/or electric shock, the charger must be protected against high humidity and water.
- Do not connect/disconnect or touch the mains part with wet hands.
- Only connect the power supply to a mains supply.
- **Do not attempt to open the charger.** In case of a defect please return to an authorised servicecentre.
- Do not use the power supply in a potentially explosive atmosphere.
- Do not cover the power supply risk of overheating.
- Prevent contact with oil, grease etc.
- Only clean with a dry cloth.

Working conditions:

Working temperature: 0°C to 40°C degrees Celsius only in connection with Beurer blood pressure monitor

Relative atmospheric humidity: 20%-90%

Storage conditions:

Storage temperature: -25°C to +70°C degrees Celsius

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Shenzhen Longxc Power Supply Co.,LTD