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# Soft  
REF 608.52

## EN Sterile single-use lancets

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are included in the delivery. Before use, ensure that there is no visible damage to the lancets and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

	<b>Important</b> Safety note indicating possible damage to the device/accessory
	<b>Product information</b> Note on important information
	Observe the instructions for use
	Medical device
	Type number
	Sufficient for <n> tests
	Unique Device Identifier (UDI)
	Batch designation
	Use by
	Article number
	Do not use if packaging is damaged
	Single sterile barrier system with protective packaging on the outside
	Not suitable for reuse
	Do not resterilise
	Sterilised using radiation
	Biohazard
	Temperature limit
	Humidity limitation
	Manufacturer
	Date of manufacture
	Importer
	Authorized representative in the European Community
	<b>CE labelling</b> This product satisfies the requirements of the applicable European and national directives.
	Marking to identify the packaging material. A = material abbreviation, B = material number: 1-7 = plastics, 20-22 = paper and cardboard
	Separate the product and packaging elements and dispose of them in accordance with local regulations.
	Swiss authorised representative

### Included in delivery

- 100 lancets, 33G
- These instructions for use

### Purpose

The lancet together with the lancing device is designed to take a blood sample from the fingertip to measure the glucose content in human capillary blood.

## Target group

The lancet is intended for use by specialists and for personal use. The intended patient group is adults and children over 2 years of age.

## Indications

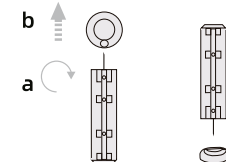
Can be used for diabetes or blood glucose measurements and capillary blood sampling.

## ⚠️ RISKS TO THE USER

- Do not use the lancet after the use-by date.
- Do not use the lancet if the protective cap has previously been removed from the lancet.
- Do not store the lancet in the lancing device.
- If you drop the lancing device with a lancet inserted, carefully pick it up and dispose of the lancet.
- The lancet is intended for single use only and must not be reused. Lancets become blunt and painful when reused.
- Make sure the puncture area is hygienically clean.
- Change the puncture area each time you take a measurement, e.g. use a different finger or the other hand. Repeatedly using the same area may cause inflammation, numbness or scarring.
- Keep the product away from children.

**i** Only use the lancets with a lancing device from Beurer. If other lancing devices are used, proper functioning of the lancets cannot be guaranteed. In addition, if the lancet and lancing device are not compatible, the function of the lancing device may be permanently impaired.

1. Insert the lancet into the lancing device and remove the protective disc with a twisting movement.
2. Read the instructions for use of your lancing device. If you are using the lancing device from your Beurer starter set, you will find these in the instructions for use of the measuring system.
3. After use, insert the needle of the lancet into the retained protective disc (c).

### ⚠️ Risk of infection

The lancet comes into contact with human blood and is therefore a possible source of infection.

It is therefore essential to comply with the generally applicable safety precautions for handling blood.

### Disposal

After use, please dispose of the lancets in a puncture-proof container. Carefully dispose of all blood samples and materials with which you have come into contact in order to prevent injury and infection of others.

### Customer service address

Please contact our customer service if you have any questions. Please find the service address in the attached address insert.

### Where are the lancets available?

Available without need for a prescription from your pharmacist or outlets selling Beurer blood glucose monitors.

Please contact our customer service if you have any further questions concerning the lancets and the compatible lancing device.

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Emergo Europe B.V., Westervoortsedijk 60,
Beurer Schweiz AG, Hauptstrasse 77,