
# beurer

## BiteX Go BR 75

CE 0483
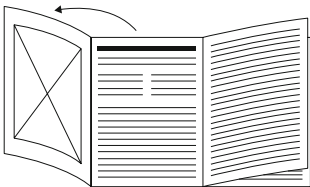
<b>DE</b>	Insektenstichheiler Gebrauchsanweisung.....	4
<b>EN</b>	Insect bite healer Instructions for use.....	32
<b>FR</b>	Appareil soulageant les piqûres d'insectes Mode d'emploi.....	55
<b>ES</b>	Aparato contra picaduras de insectos Instrucciones de uso.....	80
<b>IT</b>	Dopopuntura Istruzioni per l'uso.....	105
<b>TR</b>	Böcek sokması iyileştirme cihazı Kullanım kılavuzu.....	130
<b>RU</b>	Прибор для заживления кожи от укусов насекомых Инструкция по применению.....	154

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# ENGLISH

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

## TABLE OF CONTENTS

1. Included in delivery .....	33	7. Cleaning and maintenance .....	49
2. Signs and symbols .....	33	8. What if there are problems? .....	51
3. Proper use .....	37	9. Disposal.....	52
4. Warnings and safety notes.....	38	10. Technical specifications .....	53
5. Device description.....	45	11. Warranty/service .....	54
6. Usage .....	46		

## **1. INCLUDED IN DELIVERY**

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device and that all packaging material has been removed. In case of any doubt, do not use the device, and contact your retailer or the specified Customer Service address.

1 x BiteX Go BR 75

1x quick guide

1 x instructions for use

## **2. SIGNS AND SYMBOLS**

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

## **⚠ WARNING**

Indicates a potentially impending danger. If it is not avoided, there is a risk of death or serious injury.

## **⚠ CAUTION**

Indicates a potentially impending danger. If it is not avoided, slight or minor injuries may result.

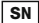
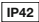
## **NOTICE**

Indicates a potentially harmful situation. If it is not avoided, the system or something in its vicinity may be damaged.

### **Product information**

Note on important information

	<p><b>Observe the instructions</b> Read the instructions before starting work and/or operating devices or machines</p>
	<p>Type BF applied part</p>
	<p>The electronic device must not be disposed of with household waste</p>
	<p>Manufacturer</p>
	<p>Serial number</p>
	<p>Device protected against solid foreign objects <math>\geq 1</math> mm in diameter and larger, and against drops of water when the housing is angled up to <math>15^\circ</math></p>
	<p>Temperature limit</p>

	Humidity limitation
	<b>CE labelling</b> This product satisfies the requirements of the applicable European and national directives.
	Medical device
	Separate the product and packaging elements and dispose of them in accordance with local regulations.
	Marking to identify the packaging material. A = Material code, B = Material number: 1-7 = Plastics, 20-22 = Paper and cardboard
	Importer symbol
	Unique Device Identifier (UDI) for unique product identification

<b>REF</b>	Item number
<b>#</b>	Type number
	Date of manufacture
<b>LOT</b>	Batch code
<b>CH REP</b>	Swiss authorised representative

### 3. PROPER USE

#### Intended use

The insect bite healer is intended for relieving itching and swelling caused by insect bites or stings by applying heat (local hyperthermia).

#### Target group

The insect bite healer is intended for use in the home healthcare environment, not in professional healthcare facilities. The insect bite healer can be used on anyone

over 3 years of age under supervision; can be used for self-treatment by anyone over 12 years of age.

## **Clinical benefits**

Treatment and relief of itching and swelling caused by insect bites or stings.

## **Indication**

Treatment with the insect bite healer is recommended for:

- itching
- swelling

caused by insect bites or stings.

# **4. WARNINGS AND SAFETY NOTES**

## **Contraindications**

### **⚠ WARNING**

- Any person using it on themselves must be over 12 years old. The product should only be used on children under 12 years old if supervised or assisted by an adult.

- Do not use the device if you suffer from diabetes. Sensitivity to pain could, in certain circumstances, be reduced and diabetics may burn themselves.
- Do not use the device on people with disabilities, children under 3 years old or people with reduced sensitivity to heat. This includes people with skin alterations due to illness or scarred tissue in the application area, after taking pain relief medication, drinking alcohol or taking drugs.
- Do not use the device on acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin, whether painful or not, reddened skin, rashes, (e.g. allergies), burns, frostbite, bruises, swellings (without signs of an insect bite/sting), open/bleeding and healing wounds, and surgical scars where the healing process could be affected).
- Do not use the device if you are suffering from a fever.
- Do not use the device on sensitive areas of skin.
- Do not use the device if you have any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- Do not use the device if you are using lotions, creams and gels locally at the same time.

- Do not use the device if you suffer from persistently irritated skin due to long-term heat application on the same area of skin.
- Disconnect the device from the power supply if you do not want to use it immediately.

## General warnings

### **WARNING**

- Seek medical advice immediately if the symptoms persist despite using the insect bite healer and stop using it.
- The device is not a substitute for medical consultation and treatment. Always consult your doctor if you are experiencing any pain or are suffering from an illness.
- If the device does not work properly, or if you feel unwell or experience pain, stop using it immediately.
- Over 3 years of age under supervision, can be used for self-treatment by anyone over 12 years of age.

- Before using the device, consult your doctor if you have any skin conditions, serious illness (in particular if you have a propensity to thromboembolic conditions or recurrent malignant growths), unexplained chronic pain in any part of the body.
- The device is not a substitute for medical consultation and treatment. If the symptoms persist despite using the Insect bite healer and stop using it. Seek medical advice immediately if the symptoms worsen whilst or after using the Insect bite healer and stop using it.
- Seek medical advice immediately if it is a tick bite. The bite may result in transmission of pathogens such as tick-borne encephalitis (TBE) or borreliosis.
- Seek medical advice immediately if the first signs of increased itching, skin redness, swelling, dizziness, nausea, shortness of breath, formation of wheals or a drop or increase in blood pressure occur.
- Do not use near to the eyes, on the eyelids, mouth or on mucous membranes. Furthermore, the device must not be used on moist skin.
- Do not allow children to play with the device.

- **Choking hazard!** Keep the device, the packaging and the accessories away from children, as there is a risk of suffocation, injury and choking. Seek medical attention immediately if swallowed.
- Please note the general hygiene measures when using the insect bite healer.
- If the device does not work properly, or if you feel unwell or experience pain, stop using it immediately.

## Usage Warning

### **NOTICE**

- This device only works in conjunction with a suitable smartphone with CE marking.
- To use it on Android phones: need to be used on Android 10.0 or higher.
- To use it on Apple phones: need to be used on iPhone 15 or newer.
- Smartphones must support the OTG standard. For some phones, OTG must be activated via settings before they can be used.
- Mobile phones or power supply devices with other power interfaces can be used only, if they strictly meet the power supply requirements. Failure to use

them in accordance with the power supply requirements may cause damage to the power supply device or this device.

## General precautions

### **⚠ CAUTION**

- Do not touch the device with wet hands. Do not allow water to spray onto the device. Only operate the device, if it is completely dry.
- Do not use any detergents or solvents to clean and maintain the device. If liquid penetrates the device this could cause damage to the electronics or other device parts and lead to a malfunction.
- The device is not recommended be used by one person. If it is used by multiple people, it should be cleaned and disinfected before use.
- Protect the device against direct sunlight and store in a dry place.
- Protect the device from impacts, moisture, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).
- Ensure that no water enters the device. If this should happen, remove the device and only use the device again when it is completely dry.

- If you store the device, no heavy objects should be placed on top of it.
- Before use, ensure that there is no visible damage to the device or accessories. In case of any doubt, do not use the device, and contact your retailer or the specified Customer Service address.
- Store the device in a location protected against climatic influences. The device must be stored in the environmental conditions specified.
- When ambient temperature is less than  $-10^{\circ}\text{C}$  or higher than  $50^{\circ}\text{C}$ , the time from minimum storage temperature to use is at least 4 hour; The time from maximum storage temperature is at least 4 hour.

## Note on electromagnetic compatibility

### **WARNING**

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the device.

- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Keep portable RF communication devices (including peripheral equipment, such as antenna cables or external antennas) at least 30 cm away from all device parts, including all cables included in delivery.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated above, this device as well as the other devices must be monitored to ensure they are working properly.
- Failure to comply with the above can impair the performance of the device.

## 5. DEVICE DESCRIPTION

The associated drawings are shown on page 3.

**1** Key ring eyelet

**2** Cover with screw fastener

**3** Device status LED

**4** Hot plate

**5** BiteSpotLight (light)

**6** Touch pads

**7** USB-C connector

**8** Main unit

## 6. USAGE

### **⚠ WARNING**

Before use, make sure that there is no insect sting remaining in the area that was stung. Remove the sting carefully before use. To achieve the best results, use the device immediately after the insect bite or sting. If you wait too long to treat the area that has been stung or bitten and itching and swelling have already started, the insect bite healer can only have a limited effect. Nevertheless, treat the area with the insect bite healer. Usually, itching can be relieved and healing can be sped up.

## **⚠ CAUTION**

Depending on the sensitivity of the person using the device and of the area of skin, the treatment temperature may feel unpleasant and cause slight skin redness and, in rare cases, irritated skin. If the heat function feels too hot, stop treatment immediately!

1. Remove the cover **2** from the main unit **8** by unscrewing it anti-clockwise.
2. Insert the main unit **8** into the USB-C port on your smartphone using the connector **7**. A slight click indicates that the device has been fully inserted.  
→ The device status LED **3** starts to light up green and the device is now ready for operation.
3. Place the hot plate **4** of the device on the area that has been stung or bitten. The integrated BiteSpotLight (light) **5** helps you locate the area that has been stung or bitten. Make sure that the entire touch pad **6** is in contact with the skin.
4. You can choose between 2 different treatment times (3 or 6 seconds) depending on how sensitive your skin is.

The 3-second treatment is recommended for first use or use on sensitive skin.

- For the 3-second treatment, place the hot plate **4** of the device on the area that has been stung or bitten as described in step 3. The touch pad **6** detects the contact with the skin.
    - The device status LED **3** lights up orange and the treatment starts.
  - For a 6-second treatment, **briefly** place the device with the hot plate **4** on the puncture or bite site 2x.
    - The device status LED **3** lights up pink and the treatment starts.
5. The device status LED **3** lights up green again and the hot plate **4** stops providing heat.
  6. Remove the main unit **8** from your smartphone.

If you do not actively switch off the device yourself, the automatic switch-off function will switch it off automatically after 2 minutes. To use the device again after the automatic switch-off, reactivate it by touching the touch pad **6** against your skin.

## 7. CLEANING AND MAINTENANCE

Observe the following hygiene instructions to avoid health risks:

- The device is intended for multiple uses.
- It is recommended that the device is cleaned before first use and each additional use.
- Before cleaning, the device must be disconnected from the power source (e.g. smartphone) and cooled down.
- Each time before cleaning the device, the cover with screw plug **2** must be fully screwed on.
- Use only soft, dry cloths for cleaning the housing.
- Never use any substances for cleaning that could potentially be toxic if they came into contact with the skin or mucous membranes, or if they were swallowed.

wed or inhaled. Do not use any abrasive cleaning products and never submerge the device in water.

- Do not clean the device in the dishwasher!
- Avoid mechanical cleaning with brushes or similar, as this could result in irreparable damage.
- When using the device on multiple areas that have been stung and bitten, it is recommended that you clean the heating plate each time. This reduces the possibility of germs and bacteria being transferred from one area of skin to the next. To disinfect the heating plate, use a cotton bud moistened with disinfectant.
- To ensure proper operation, clean the touchpad regularly according to the cleaning instructions.

## 8. WHAT IF THERE ARE PROBLEMS?

Problem	Possible cause	Solution
The device does not switch on.	The USB-C connector <b>7</b> is not inserted correctly	Disconnect the device from your smartphone. Plug the device with the USB-C connector <b>7</b> into the USB-C port of your smartphone again. A slight click and the device status LED lit up in green <b>3</b> signal that the device is correctly plugged in and ready for use.
	The device may be faulty.	Please contact Customer Services.
The device status LED <b>5</b> flashes green for 5 seconds.	The heating of the hot plate <b>4</b> exceeds the maximum temperature.	Disconnect the power supply, wait 3 minutes for the temperature to return to normal, and then resume the treatment.

Problem	Possible cause	Solution
The device does not react to contact with the skin and does not initiate treatment	Because there is dirt (e.g. dust) on the touch pad <b>6</b> , contact has not been detected.	Please clean the touch pad <b>6</b> as described in the “Cleaning and maintenance” section.
	Because the contact surface between the touch pad <b>6</b> and the skin is not large enough or the pressure applied is too low, contact has not been detected.	Make sure that the entire touch pad <b>6</b> is in contact with the skin. To do this, press the device harder onto the affected area of skin.

## 9. DISPOSAL

For environmental reasons, do not dispose of the device in household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with

EC Directive Waste Electrical and Electronic Equipment (WEEE). If you have any questions, please contact the local authorities responsible for waste disposal.

## 10. TECHNICAL SPECIFICATIONS

Type	BR 70
Model	BR 75
Dimensions (L x W x H)	Ø 14,2 mm x 43 mm
Weight	approx. 5 g
Power supply	5V± 0.25V, Max. 500mA
Max. temperature	50 °C ±2 °C (122 °F ±3.6 °F)
Functions	2 time settings (3s/6s)
Permissible storage and transport temperature and humidity	-10 °C to 50 °C, relative humidity 30% to 85%
Permissible operating temperature and humidity	10 °C to 40 °C, relative humidity 30% to 85% Air pressure: 700 hPA to 1060 hPA

---

Automatic device switch-off

2 min after last application

---

The serial number is located on the device. Subject to technical changes.

The device complies the respective national regulations and the European standard EN 60601-1-2 (Group 1, Class B, in accordance with CISPR 11, CISPR 14-1, CISPR-32, IEC 61000-3-2, IEC 61000-3-3, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-4, IEC 61000-4-5, IEC 61000-4-6, IEC 61000-4-8, IEC 61000-4-11) and is subject to particular precautions with regard to electromagnetic compatibility.

## 11. WARRANTY/SERVICE

Further information on the warranty and warranty conditions can be found in the warranty leaflet supplied.

### **Notification of incidents**

For users/patients in the European Union and identical regulation systems, the following applies: If during or through use of the product a major incident occurs, notify the manufacturer and/or their representative of this as well as the respective national authority of the member state in which the user/patient is located.

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UK Importer: Beurer UK Ltd., Suite 16, Stonecross Place, Stonecross Lane North, WA3 2SH Lowton, United Kingdom
Beurer GmbH • Söflinger Str. 218