
<b>DE</b> Notrufarmband Gebrauchsanweisung .....	4	<b>TR</b> Acil durum bilekliği Kullanım kılavuzu.....	116
<b>EN</b> Emergency wristband Instructions for use.....	27	<b>PL</b> Opaska dla seniora Instrukcja obsługi .....	138
<b>FR</b> Bracelet SOS Mode d'emploi.....	49	<b>NL</b> SOS-armband Gebruiksaanwijzing .....	160
<b>ES</b> Pulsera de emergencia Instrucciones de uso .....	71	<b>SV</b> Nödlarmsarmband Bruksanvisning.....	182
<b>IT</b> Bracciale salvavita Istruzioni per l'uso .....	94		

- DE** Klappen Sie vor dem Lesen der Gebrauchsanweisung die Seite 3 aus.
- EN** Unfold page 3 before reading the instructions for use.
- FR** Dépliez la page 3 avant de lire le mode d'emploi.
- ES** Despliegue la página 3 antes de leer las instrucciones de uso.
- IT** Prima di leggere le istruzioni per l'uso aprire la pagina 3.
- TR** Kullanım kılavuzunu okumadan önce 3. sayfayı açın.
- PL** Przed przeczytaniem instrukcji obsługi otworzyć stronę 3.
- NL** Vouw pagina 3 uit om de gebruiksaanwijzing te kunnen lezen.
- SV** Vik ut sid. 3 innan du läser bruksanvisningen.

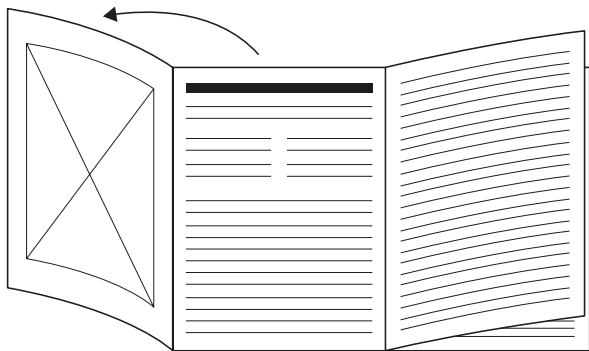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Read these instructions for use carefully. Observe the warnings and safety notes. Keep these instructions for use for future reference. Make the instructions for use accessible to other users. If the device is passed on, provide the instructions for use to the next user as well.

## Contents

1. Getting to know your device .....	27
2. Included in delivery .....	28
3. Signs and symbols .....	28
4. Intended use.....	30
5. Warnings and safety notes .....	30
6. Device description.....	31
7. Usage .....	32
7.1 Initial use .....	32
7.2 Charging the battery .....	33
7.3 Opening/closing the wristband .....	34
7.4 Adjusting the wristband size .....	35
7.5 Switching the emergency wristband on and off .....	36
7.6 How the emergency process works .....	37
7.7 Fall detection/manual emergency notifications .....	38
7.8 Cancelling (false alarm) / ending the emergency mode .....	39
7.9 Battery level indicator .....	40
7.10 Checking the network strength.....	41
7.11 Updates/changes to settings.....	41
7.12 Restarting the emergency wristband/troubleshooting.....	42
8. Cleaning and maintenance.....	42
9. Accessories and replacement parts.....	43
10. What if there are problems? .....	43
11. Disposal.....	45
12. Technical specifications .....	45
13. Warranty .....	47

## 1. GETTING TO KNOW YOUR DEVICE

In order to use the emergency wristband, a fee-based subscription is required in the “My Gardia” app. The emergency contacts also require an active Internet connection in order to make full use of the app, in particular to receive an emergency notification.

The Emergency Wristband is a mobile communication device that allows you to trigger an emergency notification in an emergency. If it detects a fall, the emergency wristband automatically sends an emergency notification for you via the fall detection function, or you can trigger an emergency notification yourself at the push of a button.

If you find yourself in an emergency, your emergency contacts (e.g. relatives, nursing staff) will be informed immediately via the “My Gardia” app. Your emergency contacts will be able to establish a voice connection (“emergency call”) and find your location. The location can only be tracked during an emergency notification.

There is also the option of booking a professional emergency call service, which will provide you with help as quickly as possible in an emergency and can be reached 24 hours a day, 7 days a week, all year round.

The emergency wristband is a mobile communication device with an integrated Spanish roaming SIM card that always connects to an available network in the respective country. So when the emergency wristband is connected to an emergency contact, a foreign mobile phone number will be displayed. In addition to the monthly subscription fees, additional costs may be incurred for an active Internet connection (so that the app can be used to its full extent), or for establishing a voice connection between the emergency contact and the emergency wristband.

For further information on the emergency wristband, the “My Gardia” app, the availability in different countries and the system requirements and compatible devices, scan the adjacent QR code or visit [gardia.net](http://gardia.net)

## 2. INCLUDED IN DELIVERY

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- 1 emergency wristband
- 1 charger
- 1 charging cable
- 1 mains plug
- 1 copy of these instructions for use

## 3. SIGNS AND SYMBOLS

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

### **WARNING**

Indicates a potentially impending danger. If it is not avoided, there is a risk of death or serious injury.

## ⚠ CAUTION

Indicates a potentially impending danger. If it is not avoided, slight or minor injuries may result.

## NOTICE

Indicates a potentially harmful situation. If it is not avoided, the system or something in its vicinity may be damaged.

 <b>Product information</b> Note on important information	 Read the instructions
 The electronic device must not be disposed of with household waste	 Marking to identify the packaging material. A = material abbreviation, B = material number: 1–7 = plastics, 20–22 = paper and cardboard
 Do not dispose of batteries containing harmful substances with household waste	ta=45°C Rated ambient temperature
 Protection class II device	 For indoor use only
 Conformity assessed marking for the United Kingdom	 <b>CE labelling</b> This product satisfies the requirements of the applicable European and national directives.
 Manufacturer	 <b>IP67</b> Dust-proof and protected against the effects of temporary immersion in water
 Do not use the mains adapter if the plug is damaged.	 Direct current
 Temperature limit	 Humidity limitation
 Polarity of DC power connection	 Energy efficiency level 6
 Switch mode power supply unit	 Safety isolating transformer, short-circuit proof
 <b>IP20</b> Protected against solid foreign objects 12.5 mm in diameter and larger	

## 4. INTENDED USE

The emergency wristband is intended to trigger an emergency notification in emergency situations. Only use the emergency wristband in emergency situations. Only use the emergency wristband on people and not on animals.

The emergency wristband is equipped with an automatic fall detection function, which triggers an emergency notification when a fall is detected.

In an emergency, the emergency wristband can be located via the “My Gardia” app. The wearer can communicate with an emergency contact via the microphone and loudspeaker. The emergency wristband is not a medical device.

The emergency wristband must only be used for the purpose for which it was designed and in the manner specified in these instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

## 5. WARNINGS AND SAFETY NOTES

### WARNING

- Do NOT use the emergency wristband if you have a pacemaker or other active implants (such as infusion pumps or brain pacemakers).
- Do not use the emergency wristband in areas where the use of mobile phones is prohibited. These include aircraft, hospitals or in the vicinity of sensitive medical devices, chemicals and fuels, as well as in blasting areas. Switch off the emergency wristband in good time before entering such areas.
- Keep packaging material away from children. Keep the device and the package contents, especially any small parts included, out of the reach of children.
- In motor vehicles, radio signals may interfere with functions if electronic systems in motor vehicles and other vehicles have not been properly installed or have not been adequately shielded. If you notice such faults, contact the manufacturer of your vehicle, a specialist workshop or the manufacturer of the components susceptible to faults and find out about their susceptibility to interference.

### CAUTION

- Never open the housing of the emergency wristband or charger by yourself. The housings should only be opened by authorised specialists when inspecting for faults and defects. If your device has a fault or defect, please contact Customer Services.

### NOTICE

- The emergency wristband uses 2G mobile phone technology. This means that its function cannot be guaranteed in remote areas and in cellars, garages, etc. (depends on the reception).
- Only use the charger and mains plug supplied to charge the battery. The use of external accessories may result in the battery not being charged correctly or in faults.

- The emergency wristband is protected against temporary immersion in water, but we advise you not to wear the emergency wristband when swimming or bathing and we advise against keeping it under water.
- Keep the emergency wristband away from chemicals such as soaps, shampoos, sun cream, conditioners, lotions and perfumes.

## Notes on handling batteries

### **⚠ WARNING**

- If your skin or eyes come into contact with fluid from a battery cell, wash the affected area with water and seek medical assistance.
- Protect batteries from excessive heat.
- Risk of explosion! Never throw batteries into a fire.
- Do not disassemble, split or crush the batteries.

### **⚠ CAUTION**

- Protect batteries from excessive heat.
- Batteries must not be short-circuited.
- Only use chargers specified in the instructions for use.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.
- Fully charge the battery at least every 3 months.
- Fully charge the battery prior to initial use.


## 6. DEVICE DESCRIPTION

The associated drawings are shown on the fold-out page 3.

- |   |   |
|---|---|
| <b>1</b> Loudspeaker                              | <b>8</b> Pin for closing                  |
| <b>2</b> LEDs                                     | <b>9</b> Pin to adjust the wristband size |
| <b>3</b> ON/OFF/emergency button                  | <b>10</b> Charger with USB-C port         |
| <b>4</b> Microphone                               | <b>11</b> Charging contacts               |
| <b>5</b> Barometer                                | <b>12</b> Stand                           |
| <b>6</b> Stainless steel folding clasp            | <b>13</b> Charging cable                  |
| <b>7</b> Contact points for opening the wristband | <b>14</b> USB mains plug                  |
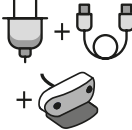
## 7. USAGE

### 7.1 Initial use

<p>1. Download the “My Gardia” app from the Apple App Store (iOS) or Google Play (Android™).</p>	
<p>2. Regardless of whether you are wearing the device or an emergency contact, register in the “My Gardia” app and follow the instructions. To set up the device, tap “Add device”. You will then be guided step-by-step in the app.</p>	
<p>3. Connect the charging cable <b>13</b> to the charger <b>10</b> and the supplied mains plug <b>14</b>. Insert the mains plug into a suitable socket.</p>	
<p>4. Use the app to scan the code on the back of the emergency wristband or enter the serial number on the back of the emergency wristband (combination of numbers and letters) into the app.</p>	
<p>5. Place the emergency wristband in the charger <b>10</b> when prompted in the app. The emergency wristband turns on automatically.</p>	
<p>6. During the flashing sequence, press the ON/OFF/emergency button <b>3</b> when prompted in the app, without removing the emergency wristband from the charger.</p>	
<p>7. The LEDs <b>2</b> light up green as confirmation. Do not remove the emergency wristband from the charger <b>10</b> until it has been fully set up.</p>	

8. Continue the set-up process in the app and follow the instructions.

9. Select a subscription and make the payment within the app.

10. Add your desired emergency contacts. Up to 6 invitations can be sent to the emergency contacts for this purpose.







When the app is opened for the first time, it will inform you of any “critical alerts” (iOS) or “missing permissions” (Android™). These must be activated/allowed so that the emergency alarm can sound in the app even if the mute function is active.

The QR code and the serial number for integrating the device are also provided on the outside of the packaging. We recommend keeping them safe or writing down the serial number.

## 7.2 Charging the battery

1. Connect the charging cable **13** to the charger **10**.
2. Connect the other end of the charging cable **13** to the supplied mains plug **14**.
3. Insert the mains plug **14** into a suitable socket.
4. Connect the emergency wristband to the charger **10**. It doesn't matter which way round you place the emergency wristband in the charger.
5. After connecting the mains plug **14** to the mains power supply, a white flashing pattern should confirm the charging process and indicate the charging status. A full charge of the battery takes up to 2 hours.
6. Once the device has been charged to 100%, you will see 4 solid white LEDs **2**. Once the wristband has been fully charged, remove the charger **10** and accessories and store them carefully.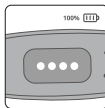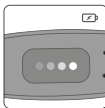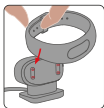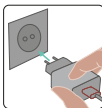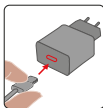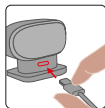


To check the battery level manually, briefly press the ON/OFF/emergency button **3** once, as explained in more detail in chapter “7.9 Battery level indicator”. The battery will last up to 21 days if there is no activity and no events occur where a fall needs to be evaluated. As soon as the battery drops below 30%, the emergency wristband will issue an initial automatic battery warning. The emergency wristband also vibrates multiple times and the LEDs **3** indicate the battery level. The lower the battery level, the more often you will receive battery warnings.


When charging, the device automatically searches for and installs available updates. This ensures that you always have the latest software version on your emergency wristband.


### 7.3 Opening/closing the wristband

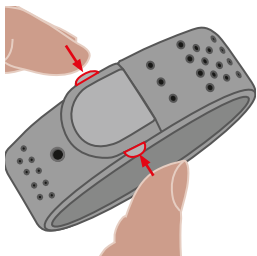
1. Open the wristband by pressing the two silver-coloured contact points **7** on the metal clasp at the top and bottom at the same time.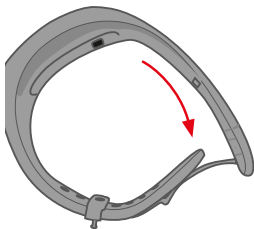

2. You can now slip your hand through the wristband opening.

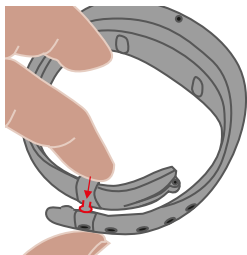
3. Close the wristband by closing the two sides of the wristband and pressing the top and bottom together until the pin **8** engages.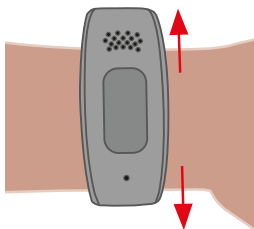


4. Place the wristband on the wrist so that it sits comfortably but does not slip. We recommend that the microphone **4** should face the inside of your body while the loudspeaker **1** should face away from you.


## 7.4 Adjusting the wristband size

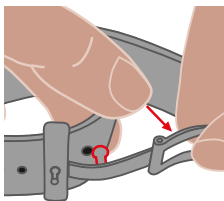1. First, open the wristband. Now hold the wristband with one hand and gently pull the metal band with the other until the pin **9** on the underside of the clasp releases from the hole in the strap.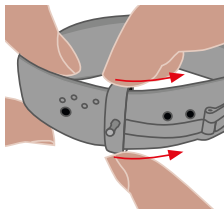

2. Now hold the strap with one hand while moving the metal band with the other.

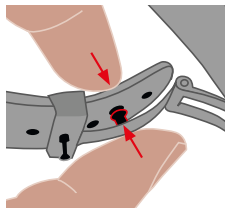3. Push the pin **9** into another wristband hole.

## 7.5 Switching the emergency wristband on and off

1. To switch on the emergency wristband, press the ON/OFF/emergency button **3** 4 times in quick succession. The LEDs **2** start to flash white. The emergency wristband also vibrates briefly. The emergency wristband is ready for use as soon as the flashing pattern and vibration have ended. When used for the first time or when the battery is completely empty, the emergency wristband will only switch on when placed in the charger **10** provided.

Press 4 times

=

Flashing white

+

Vibration

2. To switch off the emergency wristband again, press the ON/OFF/emergency button **3** again 4 times in quick succession. The LEDs **2** start to flash white again. The emergency wristband also vibrates briefly. The emergency wristband is now switched off.

Press 4 times

=

Flashing white

+

Vibration



To check if the emergency wristband is on, press the ON/OFF/emergency button **3** briefly.

If the battery level is shown, the emergency wristband is on. If not, you must switch on the emergency wristband or charge it if the battery is flat.

## 7.6 How the emergency process works

An emergency notification can be initiated both manually by pressing a button, and automatically by the fall detection function.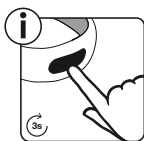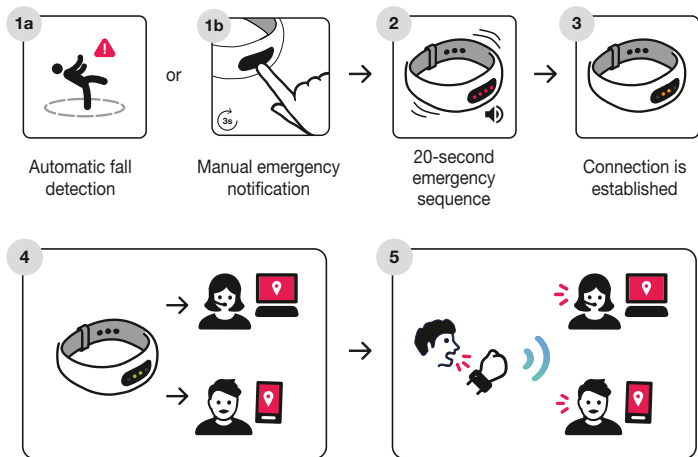

Cancel or end emergency mode

## 7.7 Fall detection/manual emergency notifications

1. a) An emergency notification is **automatically** triggered when the smart **fall detection function** senses that the wristband wearer has fallen.  
b) To **manually** send an emergency notification, press and hold the ON/OFF/emergency button **3** for more than 3 seconds.

2. The LEDs **2** then start to flash red and change to a yellow flashing pattern one after the other. An emergency alarm also sounds via the built-in loudspeakers and the device vibrates. This emergency sequence lasts 20 seconds and can be cancelled at any time until the notification is sent. If the emergency sequence is cancelled, a call cannot be made.

Flashing red

Flashing yellow

3. The emergency wristband will now try to connect to the server and send an emergency notification. This is indicated by a yellow running light. The duration of this process depends on the network connection at the location in question and can take several minutes.

Yellow running light

4. After the emergency notification has been successfully sent, the LEDs **2** show a green running light. The emergency contacts are now informed of the emergency via the app.

Green running light

5. As soon as the emergency contact makes a voice call and is successfully transferred, the wearer does not need to take any further action to accept the call. The LEDs light up steady green as soon as an active voice connection is established. The wearer can now communicate with the emergency contact via microphone **4** and loudspeaker **1**.

Green light

### NOTICE

Despite careful development and testing, certain events may not trigger an emergency notification. This may be because the impact was too weak and was not detected as a fall. There

may be falls where the wristband's sensors cannot detect the necessary parameters. Examples include gently sliding down to the floor against a wall or from a chair without a hard impact. Although the wearer may perceive this as a fall, it cannot be detected by the device. To minimise unnecessary false alarms, the fall detection function has been optimised so that dropping the device does not normally trigger an emergency notification. The fall detection function is being continuously improved and is automatically updated the next time the device is charged and an update is available and pending.

If you have also booked the professional emergency call service, the emergency notification will also be forwarded to a 24-hour emergency call centre. This establishes a voice connection with the wearer and takes the necessary measures to provide assistance as quickly as possible. In the app, you can select whether the emergency call service should accept immediately or only when all emergency contacts have declined.

## 7.8 Cancelling (false alarm) / ending the emergency mode

If there is a false alarm or if the emergency call has been ended successfully, you can cancel/end the emergency mode (regardless of whether the emergency notification was triggered manually or automatically).

- To cancel the emergency notification within the 20-second emergency sequence, press and hold the ON/OFF/emergency button **3** for 3 seconds. The LEDs **2** will then stop flashing and the emergency alarm and vibration will stop. No emergency notification will be sent to the emergency contacts.
- To cancel the emergency notification after the 20-second emergency sequence or after a successful emergency call, proceed as follows:

Press and hold the ON/OFF/emergency button **3** for 3 seconds; the 4 LEDs will then go out.

You have successfully cancelled the emergency call and the emergency mode has been ended.

The wristband is now back in its original state. We recommend charging the wristband after an emergency notification has been triggered.

False alarm triggered  
(within the first 20 s)

or

After 20 s emergency sequence  
or successful call

Cancel emergency notification (emer-  
gency contacts are not notified)

Emergency mode ended (calls not  
possible)

## 7.9 Battery level indicator

To manually view the current battery level, briefly press the ON/OFF/emergency button **3**. The LEDs **2** indicate the current battery level in white.

In addition, warnings are issued via the app and on the emergency wristband itself if the battery level falls below 30%. The emergency wristband should then be charged.

Battery level	LEDs	
100%	All LEDs light up	
87.5%	3 LEDs light up, 1 LED flashes	
75%	3 LEDs light up	
62.5%	2 LEDs light up, 1 LED flashes	

Battery level	LEDs	
50%	2 LEDs light up	
37.5%	1 LED lights up, 1 LED flashes	
25%	1 LED lights up	
12.5%	1 LED flashes	

## 7.10 Checking the network strength

To check the network strength, briefly press the ON/OFF/emergency button **3** twice. The LEDs **2** first show a purple running light and then shortly thereafter the current network strength, also in purple.

As the network strength decreases, the sound quality reduces, as does the likelihood that an emergency notification can be transferred. If the check cannot be run (check aborts), no connection to the network can be established.

Network strength	LEDs	
100%	All LEDs light up	
75%	3 LEDs light up	
50%	2 LEDs light up	
25%	1 LED lights up	

## 7.11 Updates/changes to settings

The emergency wristband is constantly being developed further. Free and automatic updates for the emergency wristband and the “My Gardia” app are provided on a regular basis. Updates can include new features, improved usability, improved security protocols, troubleshooting and other improvements.

To ensure that the emergency wristband is always up to date, it automatically searches for new updates when placed in the charger **10** and installs them independently if necessary. Once an update has been successfully installed on the emergency wristband, the LEDs **2** light up blue. Make sure to update the “My Gardia” app as soon as a new version is available in the Apple App Store (iOS) or Google Play (Android™).

The volume of the emergency wristband can be adjusted in the “My Gardia” app under the device settings. You can find more information on this in the “My Gardia” app.

## 7.12 Restarting the emergency wristband/troubleshooting

You can reset and restart the emergency wristband if you unexpectedly experience problems with the device and can no longer use it.

- To do this, connect the emergency wristband to the charger whilst holding down the ON/OFF/emergency button **3**. Hold down the button until all 4 LEDs light up white. Once you remove the emergency wristband from the charger, it should work normally again. If the problems persist, please contact Customer Services.

## 8. CLEANING AND MAINTENANCE

To ensure full functionality, clean the emergency wristband, charger and other accessories at regular intervals.

### Cleaning the emergency wristband

1. Remove the wristband from the charger **10**.
2. Clean the wristband with a soft, dry or slightly damp cloth. Do not use chemical cleaning products.
3. Allow the wristband to dry completely.
4. Check that there are no more liquids or dirt on the microphone **4** and loudspeaker **1**.

### Cleaning the charger

1. Remove the mains plug **14** and the charging cable **13** from the charger **10**.
2. Clean the charger **10** with a soft, dry or slightly damp cloth. Do not use chemical cleaning products. When cleaning, ensure that the charging contacts **11** and the USB-C socket do not get wet.
3. Then allow the charger **10** to dry completely.

### Technical maintenance

- Make sure that you replace the emergency wristband in the case of a defect or a severe reduction in battery life. Please contact our Customer Services.
- Clean the emergency wristband, the charger and all accessories at regular intervals.
- Check the emergency wristband, the charger and all accessories regularly for damage (e.g. to the housing or charging contacts) and replace damaged or defective accessories if necessary. Please contact our Customer Services.

### Functional test

- Regularly check all functions of the emergency wristband (manual emergency notification as described in chapter 7.7, battery level indicator as described in chapter 7.9 and check of network strength as described in chapter 7.10).
- Make sure that the ON/OFF/emergency button **3** works properly.
- Check whether all installed LEDs are functional.
- Test the microphone **4**, loudspeaker **1** and LEDs **2** by making a test emergency call. Also regularly check the network strength and battery level so that you can take swift action in the event of a fault. This is the only way to ensure that the device will work properly in an emergency.

## 9. ACCESSORIES AND REPLACEMENT PARTS

To purchase accessories and replacement parts, please visit [gardia.net](http://gardia.net) or contact the corresponding service address (according to the service address list) in your country. Accessories and replacement parts are also available from retailers.

Designation	Item number and/or order number
1 stainless steel folding clasp	4262492885688
Set comprising: 1 charger 1 mains cable 1 mains plug	4262492885671


## 10. WHAT IF THERE ARE PROBLEMS?

If you have any questions about the app, please check the FAQ section in the “My Gardia” app.

Problem	Possible cause	Solution
Device is off (battery level not displayed when button is pressed)	Battery is empty	Charge the battery as described in chapter “7.2 Charging the battery”
	Device is switched off	Switch on the device as described in chapter “7.5 Switching the emergency wristband on and off”
	Battery faulty or other problems	Contact Customer Services
Device does not charge	Mains plug <b>14</b> not plugged into socket	Insert the mains plug <b>14</b> into a suitable socket
	USB-A plug not plugged into mains plug <b>14</b>	Insert the USB-A plug into the mains plug <b>14</b>
	USB-C plug not plugged into charger <b>10</b>	Insert the USB-C plug into the charger <b>10</b>
	Device not correctly connected to charger <b>10</b>	Connect the device to the charger <b>10</b> . Make sure that the charging pins are in contact with the device’s charging contacts <b>11</b> and that there is no contamination. Clean the charging contacts if necessary.

<b>Problem</b>	<b>Possible cause</b>	<b>Solution</b>
Fastener cannot be opened	Incorrect handling	Open the fastener as described in chapter "7.3 Opening/closing the wristband".
Wristband size cannot be adjusted	Incorrect handling	Adjust the wristband size as described in chapter "7.4 Adjusting the wristband size".
ON/OFF/emergency button <b>3</b> does not work	Battery is empty	Charge the battery as described in chapter "7.2 Charging the battery".
	Device is switched off	Switch on the device as described in chapter "7.5 Switching the emergency wristband on and off"
	ON/OFF/emergency button <b>3</b> is faulty	Contact Customer Services
LEDs <b>2</b> are not working	Battery is empty	Charge the battery as described in chapter "7.2 Charging the battery".
	Device is switched off	Switch on the device as described in chapter "7.5 Switching the emergency wristband on and off"
	The LEDs <b>2</b> are faulty	Contact Customer Services
Emergency notification is not successfully transferred (no green running light)	No network connection	Check the network strength at this location as described in chapter "7.10 Checking the network strength".
Automatic emergency notification is not triggered following a fall	Battery is empty	Charge the battery as described in chapter "7.2 Charging the battery".
	Device is switched off	Switch on the device as described in chapter "7.5 Switching the emergency wristband on and off"
	No fall was detected	See chapter 7.7 for further information about the fall detection function. The impact may have been too weak.

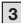Problem	Possible cause	Solution
Manual emergency notification is not triggered	Battery is empty	Charge the battery as described in chapter "7.2 Charging the battery".
	Device is switched off	Switch on the device as described in chapter "7.5 Switching the emergency wristband on and off"
	ON/OFF/emergency button  is faulty	Contact Customer Services

## 11. DISPOSAL

For environmental reasons, do not dispose of the device in household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Observe the local regulations for material disposal. Dispose of the device in accordance with EC Directive Waste Electrical and Electronic Equipment (WEEE). If you have any questions, please contact the local authorities responsible for waste disposal. You can obtain the location of collection points for old devices from the local or municipal authorities, local waste disposal companies or your retailer, for example.

Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

The codes below are printed on batteries containing harmful substances:

Pb = battery contains lead

Cd = battery contains cadmium

Hg = battery contains mercury

## 12. TECHNICAL SPECIFICATIONS

Emergency wristband

Dimensions	55 x 29 x 16 mm Wristband length: adjustable from 12.5 to 19.5 cm
Weight	Approx. 60 g
Mobile communication module	GSM/GPRS/EDGE DUAL band 900/1800 MHz
Tracking	GNSS (GPS, GLONASS, GALILEO, AGPS) / GSM triangulation / WLAN positioning (WLAN access points)

<b>Battery:</b>	
Capacity	200 mAh
Nominal voltage	3.7 V
Type	Lithium-ion
Battery life	Up to 21 days
Charging time	From 0% to 100% in approx. 2 hours
Protection class	Tested according to IP 67 – waterproof and dust-proof
Permissible operating temperature	-10°C to +50°C (the device must not be charged below 0°C)

## Transmission power

Technology	WLAN	GSM 1800	GSM 900
Frequency range	2412-2472 MHz	1710-1785 MHz	880-915 MHz
Maximum emitted transmission power	20 dBm	30 dBm	33 dBm

## Charger

Weight	Approx. 25 g
Compatible with	beurer EC 70 Emergency Wristband
Connection	USB-C
Permissible operating temperature	0°C to +50°C
Permissible storage temperature	-30°C to +50°C

## Charging cable

Length	110 cm
Connection	USB-C to USB-A

## Mains plug

Type	GQ12-050200-ZG
Input	AC 100V-240V ~50&60Hz 0.4A
Output	DC 5.0V; 2.0A (10W)
Connection	USB-A
Energy efficiency level	6
Protection class	II
Permissible operating temperature	0°C to +40°C
Permissible storage temperature	-10°C to +70°C

Subject to technical changes

We hereby confirm that this product complies with the European RED Directive 2014/53/EU. The CE Declaration of Conformity for this product can be found under: <https://gardia.net/downloads>

## 13. WARRANTY

microsynetics GmbH, Werkstraße 22, 59494 Soest, Germany (hereinafter referred to as "microsynetics") voluntarily provides a guarantee for this product, subject to the requirements below and to the extent described as follows.

**Claiming the guarantee shall not affect the Purchaser's statutory guarantee rights arising from the purchase agreement with the Seller in the event of defects. The Purchaser may exercise these statutory guarantee rights free of charge. The guarantee shall apply without prejudice to any mandatory statutory provisions on liability.**

microsynetics warrants that the product will be free from defects of material and workmanship during the guarantee period if used properly.

The guarantee applies to purchase and use of the product within the European Union (EU) and the United Kingdom (UK). The guarantee period is 2 years commencing from the purchase of the new, unused product by the buyer. The guarantee only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use. German law shall apply.

If this product is found to be defective in accordance with the following provisions during the guarantee period, microsynetics shall carry out a repair or a replacement delivery free of charge in accordance with these guarantee conditions.

**If a buyer wishes to file a guarantee claim, they should start by contacting microsynetics Customer Services: Tel: (+49) (0) 2921 94838 88 / e-mail: [support@gardia.net](mailto:support@gardia.net).**

The buyer will then receive further information about the processing of the guarantee claim, e.g. where they can send the product and what documentation is required.

A guarantee claim shall only be considered if the buyer can provide microsynetics, or an authorised microsynetics partner, with a copy of the invoice/purchase receipt, and the original product.

The following are explicitly excluded from this guarantee:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by microsynetics;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Furthermore, the guarantee expressly does not cover functional impairments or functional failures arising from circumstances beyond the control and responsibility of microsynetics. This applies in particular to functional impairments or failures that are fully or partially attributable to:

- services, omissions or other circumstances within the responsibility of third parties – in particular network, platform or other service providers – which supply services that are required for the operation or use of the product and the conduct of which microsytetics has no influence on;
- the eventuality that the product can no longer be used as intended wholly or partly due to external technical changes or changes to infrastructure;
- the availability, continuity, quality or operability of telecommunications networks, mobile networks or mobile communications standards (e.g. 2G, GSM, LTE, NB IoT);
- the switch-off, restriction, modification or discontinuation of such networks, technologies or transmission standards;

Repairs or an exchange in full do not extend the guarantee period under any circumstances.

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Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.
Google Play and the Google Play logo are trademarks of Google LLC.
Android is a trademark of Google LLC.
Distributed by: Beurer GmbH • Söflinger Str. 218 • 89077 Ulm, Germany