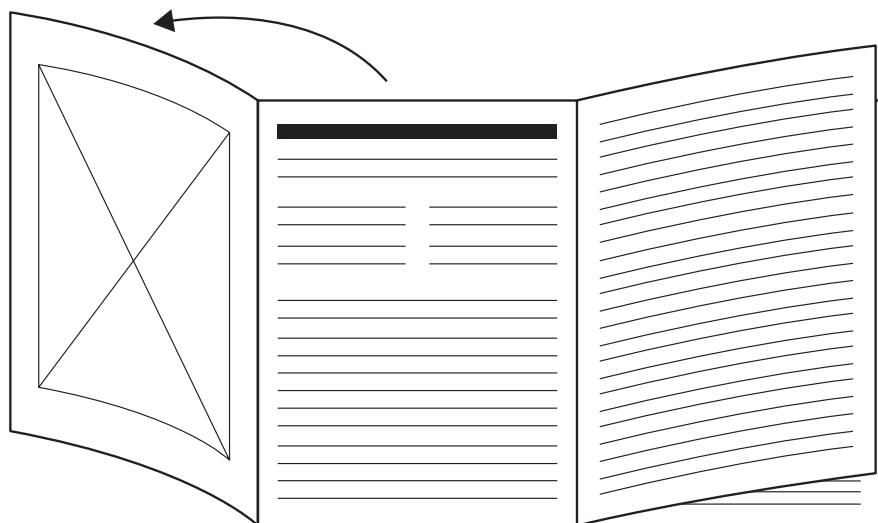
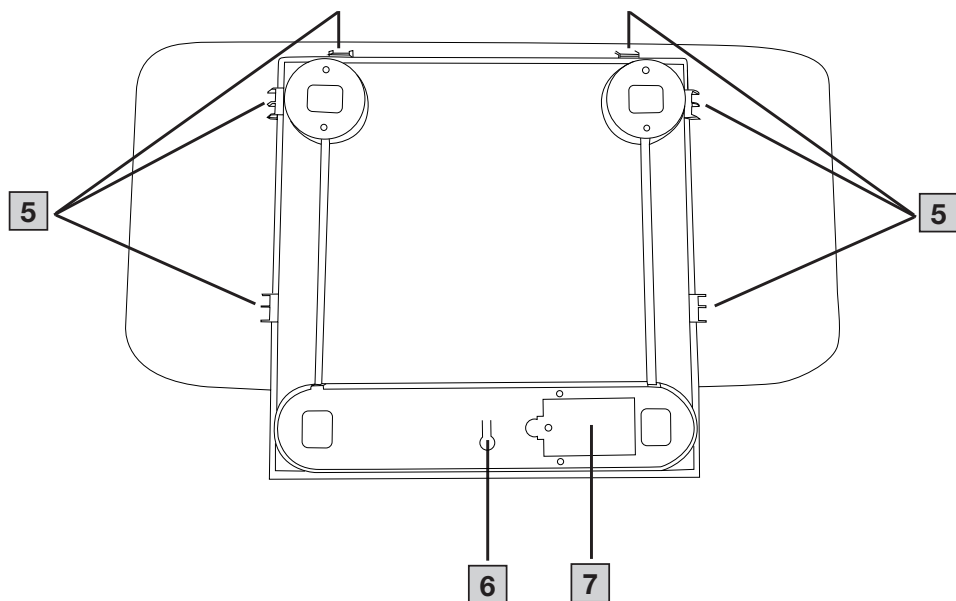
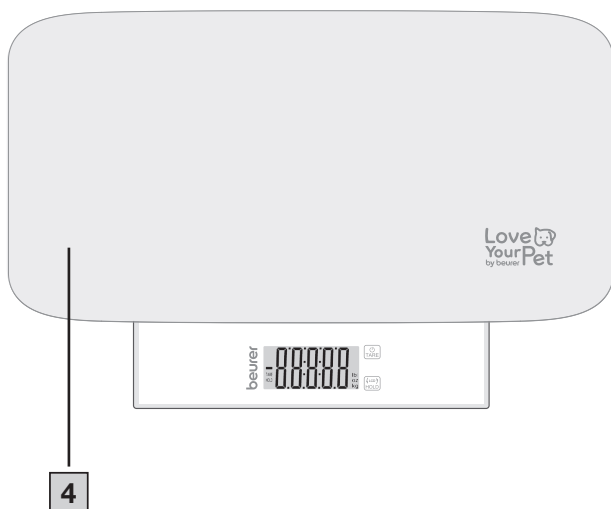
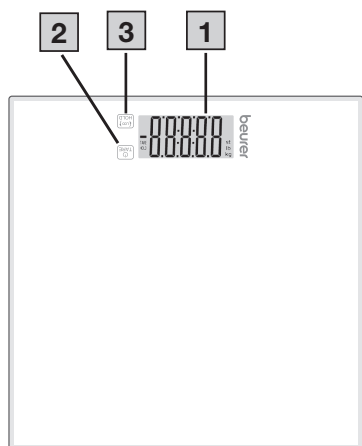
DE Multifunktions-Haustierwaage Gebrauchsanweisung	4	PL Wielofunkcyjna waga dla zwierząt domowych Instrukcja obsługi	34
EN Multifunctional pet scale Instructions for use.....	9	NL Multifunctionele huisdierweegschaal Gebruiksaanwijzing	38
FR Balance multifonction pour animaux domestiques Mode d'emploi.....	13	DA Multifunktionsvægt til kæledyr Brugsanvisning	42
ES Báscula multifunción para mascotas Instrucciones de uso	17	SV Multifunktionsvåg för husdjur Bruksanvisning.....	46
IT Bilancia multifunzione per animali domestici Istruzioni per l'uso	21	NO Multifunksjonell vekt til kjæledyr Bruksanvisning.....	50
TR Çok fonksiyonlu evcil hayvan terazisi Kullanım kılavuzu	25	FI Monitoiminen lemmikkivaaka Käyttöohje.....	54
RU Многофункциональные весы для домашних питомцев Инструкция по применению	29		

- DE** Klappen Sie vor dem Lesen der Gebrauchsanweisung die Seite 3 aus.
- EN** Unfold page 3 before reading the instructions for use.
- FR** Dépliez la page 3 avant de lire le mode d'emploi.
- ES** Despliegue la página 3 antes de leer las instrucciones de uso.
- IT** Prima di leggere le istruzioni per l'uso aprire la pagina 3.
- TR** Kullanım kılavuzunu okumadan önce 3. sayfayı açın.
- RU** Перед чтением инструкции по применению разложите страницу 3.
- PL** Przed przeczytaniem instrukcji obsługi otworzyć stronę 3.
- NL** Vouw pagina 3 uit om de gebruiksaanwijzing te kunnen lezen.
- DA** Fold side 3 ud, før du læser betjeningsvejledningen.
- SV** Vik ut sid. 3 innan du läser bruksanvisningen.
- NO** Slå ut side 3 før du leser bruksanvisningen.
- FI** Käännä sivu 3 auki ennen käyttöohjeen lukemista.

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Read these instructions for use carefully. Observe the warnings and safety notes. Keep these instructions for use for future reference. Make the instructions for use accessible to other users. If the device is passed on, provide the next user with these instructions for use.

CONTENTS

1. Included in delivery	9	6.3 Switching between pet mode and person mode.....	11
2. Signs and symbols	9	6.4 Person mode (without the pet attachment).....	11
3. Intended Purpose	9	7. Cleaning and maintenance.....	11
4. Warnings and safety notes	10	8. What if there are problems?	12
5. Device description	10	9. Disposal	12
6. Usage	10	10. Technical specifications	12
6.1 Initial use	10	11. Guarantee	12
6.2 Pet mode (with pet attachment).....	11		

1. INCLUDED IN DELIVERY

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device and accessories and/or replacement parts, and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 Multifunctional Pet Scale PP 170
- 1 pet attachment
- 3 AAA batteries (1.5V)
- These instructions for use

2. SIGNS AND SYMBOLS

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

DANGER

Indicates an imminent danger. If it is not avoided, death or serious injury will occur.

WARNING

Indicates a potentially imminent hazard. If it is not avoided, there is a risk of death or serious injury.

CAUTION

Indicates a potentially imminent hazard. If it is not avoided, slight or minor injuries may result.

NOTICE

Indicates a potentially harmful situation. If it is not avoided, the device or something in its vicinity may be damaged.

Product information

Note on important information

Read the instructions

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

Do not dispose of batteries containing hazardous substances with household waste

Manufacturer

CE labelling

This product satisfies the requirements of the applicable European and national directives.

Separate the product and packaging elements and dispose of them in accordance with local regulations.

Marking to identify the packaging material.

A = Material code, B = Material number: 1-7 = Plastics, 20-22 = Paper and cardboard

The products demonstrably meet the requirements of the Technical Regulations of the EAEU.

Conformity assessed marking for the United Kingdom

Importer symbol

3. INTENDED PURPOSE

The multifunctional pet scale is only intended for measuring the weight of people and pets. The scale is not a medical device. The scale is only intended for private use. It is not suitable for commercial or clinical use. The scale must only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use.

4. WARNINGS AND SAFETY NOTES

⚠ WARNING

Safety notes

- Protect the scale from knocks, moisture, dust, chemicals, drastic changes in temperature and nearby sources of heat (ovens, heaters).
- Keep packaging material away from children (risk of suffocation).
- Make sure that your pet is lying as still as possible on the pet attachment. If they move around too much they could fall off and suffer serious injury.
- This product is not a toy. Keep it out of the reach of children and pets.
- People with disabilities – such as impairments or any restrictions on the scope of their activity or their participation in everyday life – or who are physically frail should be assisted by another person when using this scale.
- Repairs must only be carried out by Customer Services or authorised suppliers.
- Do not press the buttons violently or with pointed objects.

⚠ CAUTION

- Make sure that you do not weigh any animals on this scale that protrude over the pet attachment **4**. The scale could tip over.
- Do not weigh more than one person or pet at a time on the scale.
- Do not step on the edge of the scale as it may tip over.
- Neither you nor your pet may jump on the scale.
- Danger of slipping! Do not step or stand on the scale if your body and/or feet are wet, e.g. after bathing or showering.
- Do not use the device on slippery surfaces, e.g. wet floors.
- Risk of injury! If the glass platform is damaged, there is a risk of injury from broken glass.
- Make sure that the device is on a flat, level and stable surface. Make sure that the device cannot tip over.

NOTICE

- Do not place any objects on the scale when it is not in use.
- The precision of the scale may be impaired by strong electromagnetic fields (e.g. induction hobs, mobile phones).
- Avoid impacts with hard objects.
- Do not drop the scale or drop any objects onto it.
- The scale's maximum capacity is 180 kg (396 lb, 28 st). Do not exceed this maximum weight on the scale.

Notes on handling batteries

⚠ WARNING

- **Risk of explosion! Risk of fire!** Failure to comply with the following points can result in personal injury or cause overheating, leakage, venting, breakage, explosion, or fire on the battery.
- This device contains non-rechargeable batteries which must not be charged.
- Do not throw batteries into a fire.
- Never charge, forcibly discharge, heat, disassemble, open, crush, deform, encapsulate, or modify batteries.
- Never short-circuit batteries or battery compartment contacts.
- Protect the batteries from direct sunlight, rain, heat, and water.
- Exposure of batteries to an environment with extremely high temperatures or an extremely low air pressure may result in explosion or leakage of flammable liquids and gases.
- Dispose of defective and discharged batteries immediately and properly (see chapter on disposal).
- Do not use modified or damaged batteries.
- Always select the correct battery type.

- Always insert the batteries correctly, taking into account the polarity (+ / -).
- Never mix batteries of different manufacturers, capacities (new and used), size, or type within a device.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- If fluid from a battery comes into contact with your skin or eyes, wash the affected areas with water and seek medical assistance.
- **Choking hazard!** Keep batteries out of the reach of children. Seek medical attention immediately if swallowed. Swallowing them may cause burns, severe internal injuries, and death.
- Never allow children to replace batteries without adult supervision.

⚠ CAUTION

- Store batteries in a well-ventilated, dry, and cool place in a non-conductive container in which the batteries cannot be short-circuited to each other or by other metal objects.
- Keep batteries clean and dry.
- Keep batteries away from water.
- If the device is not going to be used for a long period of time, remove the batteries from the battery compartment.

NOTICE

- When Do not use rechargeable batteries.

5. DEVICE DESCRIPTION

The associated drawings are shown on page 3.

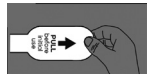
- 1** Display
- 2** ON/OFF/TARE button
- 3** LCD/HOLD button
- 4** Pet attachment
- 5** Attachment fasteners
- 6** UNIT button
- 7** Battery compartment

6. USAGE

6.1 Initial use

Initial use

Remove the battery insulating strip from the battery compartment.


Inserting/replacing batteries

1. Using a screwdriver, unscrew the battery compartment **7** on the bottom of the scale.
2. Remove the battery compartment lid.
3. Remove the flat batteries from the battery compartment as required.
4. Correctly insert three new AAA batteries into the battery compartment **7** according to the polarity markings.
5. Using a screwdriver, securely re-tighten the screw on the battery compartment lid.
6. When "Lo" appears on the display **1**, you must replace the batteries again.

Fitting the pet attachment

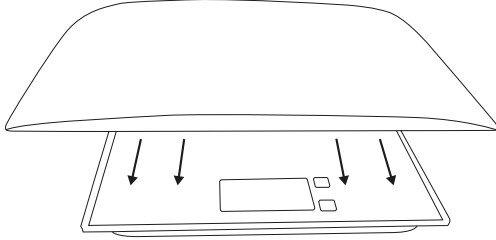1. Align the attachment fastener [5] at the glass edges of the personal bathroom scale.
2. Slide the pet attachment [4] onto the scale as far as it will go. Push the pet attachment [4] onto the glass surface from the back to the front, leaving the display [1] free.

Positioning the scale

To ensure an accurate measurement, place the scale on a flat, level and stable surface. Do not place the scale on carpet.

You can only change the display orientation using the LCD/HOLD button [3] when the display shows zeros.

6.2 Pet mode (with pet attachment)

1. Make sure that the pet attachment [4] has been correctly fitted (see chapter "Initial use"). The pet attachment [4] cannot be moved to the left or right.
2. Press the ON/OFF/TARE button [2] to switch on the scale. When zeros appear on the display [1], the scale is ready for use. If your pet weighs more than 2.5 kg, you can also place it directly on the scale. The scale switches on automatically.
3. If necessary, you can adjust the orientation of the display [1]. Make sure that the display is showing zeros for this purpose. Briefly press the LCD/HOLD button [3].
4. Use the UNIT button [6] on the bottom of the scale to select your preferred unit of measurement (kg, lb, oz or lb:oz). The scale remembers your last setting, provided that the batteries are not replaced in the meantime.
5. Place your pet in the centre of the scale. If you do not want to place your pet directly on the weighing surface, use a mat. First place the mat on the scale and press the ON/OFF/TARE button [2]. The mat should not protrude over the pet attachment [4].
6. This scale has an automatic and a manual HOLD function. The automatic HOLD function displays the weight when it has stabilised sufficiently for an exact value to be measured. The display [1] shows the measured weight for around 30 seconds. If your pet moves around too much, the automatic HOLD function will not be activated. In this case, press the LCD/HOLD button [3] when your pet is as still as possible. The numbers stop flashing within three seconds and the measured weight is displayed.
7. To switch off the scale, press and hold the ON/OFF/TARE button [2] for approx. 3 seconds. Alternatively, the scale switches off automatically after 30 seconds if no button is pressed.

6.3 Switching between pet mode and person mode

1. Slide the pet attachment [4] off the scale (if fitted).
2. Press the ON/OFF/TARE button [2] to switch on the scale. When the display shows zeros, the scale is ready for use.
3. Briefly press the LCD/HOLD button [3] to align the display [1] so that it can be read from the front when you are standing on the scale.

4. To set your preferred unit of measurement (kg, lb, st or kg, lb, oz, lb:oz), press the UNIT button [6] on the back of the scale.

6.4 Person mode (without the pet attachment)

If you want to use the scale as a normal personal bathroom scale rather than for your pet, proceed as follows:

1. Step directly onto the scale to take a measurement. Alternatively, you can switch on the scale using the ON/OFF/TARE button [2]. When the display [1] shows zeros, the scale is ready for use.

While the scale is taking a measurement, the [2] and [3] buttons are disabled to prevent you accidentally pressing them with your foot.

2. The scale switches off automatically after approx. 30 seconds. When in person mode, the scale cannot be switched off using the ON/OFF/TARE button [2].

7. CLEANING AND MAINTENANCE

Clean the surface of the scale and the pet attachment [4] from time to time using a slightly damp cloth. You can use a mild cleaning product if required. Do not use the scale or the pet attachment [4] again until they are completely dry.

⚠ CAUTION

- Do not use any abrasive cleaning products.
- Never submerge the scale in water. Never rinse the scale under running water.
- Do not place any objects on the scale when it is not in use.
- Protect the scale from knocks, moisture, dust, chemicals, drastic changes in temperature and nearby sources of heat (ovens, heaters).

Storage

If you do not plan to use the scale for a long period of time, we recommend that you store it in the original packaging in a dry environment. During storage, do not place any objects on the scale. Store the scale away from children and pets. Remove the batteries from the scale.

8. WHAT IF THERE ARE PROBLEMS?

Error message in display	Possible cause	Solution
C	In person mode, the scale has an automatic TARE function for weights from 0 to 2.5 kg grammes. Please note that the weight remains visible for 2 seconds before it is reset to zero. The display briefly shows "C" during this time.	Please use pet mode for weights from 0 to 2.5 g.
EEEE	Maximum measurement range value of 180 kg (396 lb, 28 st) exceeded	Do not exceed the maximum measurement range value
Lo	Batteries almost flat	Insert new batteries
Empty display	Batteries are flat or incorrectly inserted	Replace the batteries and ensure they are inserted the correct way round
Display shows unrealistic result	Scale was placed on a soft surface or moved during the measurement	Wait until the scale switches off automatically and repeat the measurement.
		Place the scale on a hard surface (no carpets or rugs)
		Stand still during the measurement

9. DISPOSAL

For environmental reasons, do not dispose of the device in household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Observe the local regulations for material disposal. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal. You can obtain the location of collection points for old devices from the local authorities.

Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries appropriately.

The codes below are printed on batteries containing harmful substances:

- Pb = battery contains lead
- Cd = battery contains cadmium
- Hg = battery contains mercury

10. TECHNICAL SPECIFICATIONS

Model:	PP 170	
Weight of scale with pet attachment and batteries:	Approx. 2.0 kg	
Dimensions with pet attachment:	515 x 327 x 71 mm	
Dimensions without pet attachment:	302 x 302 x 27 mm	
Measurement range:	0.05 kg to 180 kg (0.11 lb to 396 lb) (0.8 st to 28 st) min. person mode: 2.5 kg / 5.5 lb / 0.39 st	
Scale interval:	• 50 g - 5 kg: 10 g increments • >5 kg - 50 kg: 50 g increments • >50 kg - 180 kg: 100 g increments in lb: • 0.11 lb - 11 lb: 0.1 lb increments • >11 lb - 110 lb: 0.1 lb increments • >110 lb - 396.8 lb: 0.2 lb increments in st: • 0.8 - 7.9 st: 0.1 lb increments • >7.9 - 28.4 st: 0.2 lb increments	
Repetition accuracy:	Weight	Tolerance
	0 - 5 kg	± 0.05 kg
	5 - 50 kg	± 0.2 kg
	50 - 100 kg	± 0.3 kg
Absolute precision:	Weight	Tolerance
	0 - 5 kg	± 0.05 kg
	5 - 50 kg	± 0.2 kg
	50 - 100 kg	± 0.3 kg
	100 - 180 kg	± 0.4 kg
Battery type:	3 x AAA batteries	
Units of measurement:	Pet mode: kg, lb, oz, lb:oz Person mode: kg, lb, st	
Temperature limit	• Operation: 5°C to 40°C (41°F to 95°F), relative humidity 20 - 85% • Storage: 20°C to 60°C (68°F to 140°F), relative humidity 20 - 90%	

11. GUARANTEE

Further information on the guarantee and guarantee conditions can be found in the guarantee leaflet supplied.

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BEURER UK Ltd.
Beurer GmbH • Söflinger Str. 218 • 89077 Ulm, Germany