ONECONCEPT

Arielle

Duschsitz
Shower Seat
Silla para la ducha
Siège de douche
Sedile per doccia

10034209

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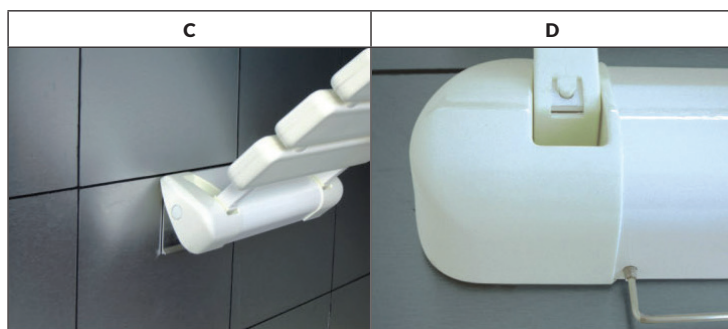
Dear Customer,

Congratulations on purchasing this device. Please read the following instructions carefully and follow them to prevent possible damages. We assume no liability for damage caused by disregard of the instructions and improper use. Scan the QR code to get access to the latest user manual and more product information.


**PRODUCER**

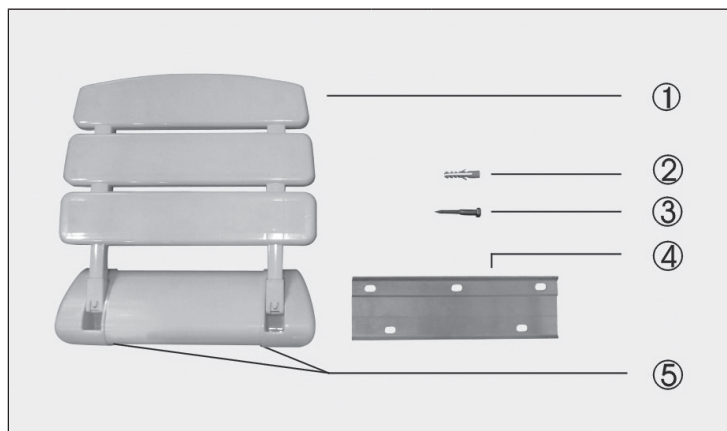Chal-Tec GmbH, Wallstraße 16, 10179 Berlin, Germany.

SAFETY INSTRUCTIONS

- Read the instructions carefully before installation.
- If the supplied screws and plugs are not suitable for installation, use suitable screws and plugs.


Note: The maximum user weight is 130 kg.

ATTACHMENT ARIELLE


1	Seat	4	Wall bracket
2	8*60 mm Dowel	5	5*8 mm Allen screw
3	5*60 mm Expansion screw		

ATTACHMENT ARIELLE

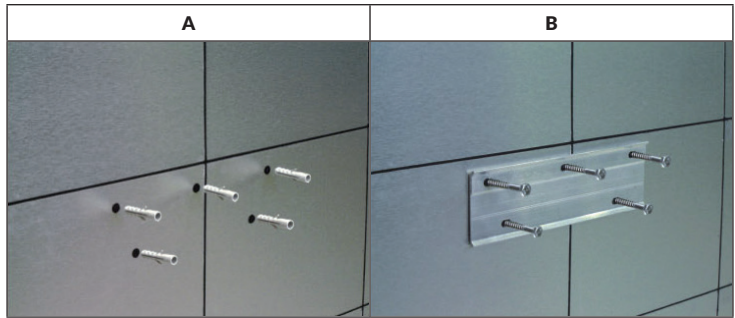Installation

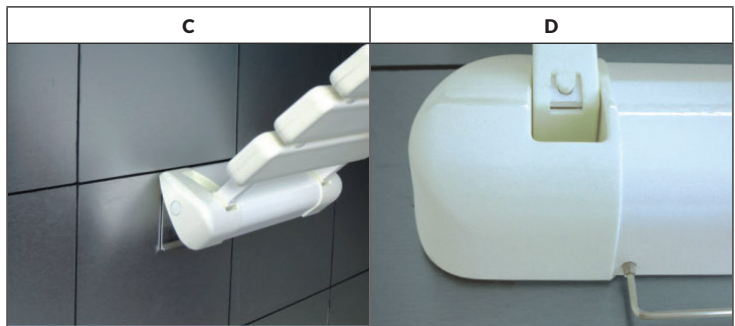- (A) Hold the wall bracket (04) against the wall at the desired height. The edge with the three holes should point upwards. The bottom of the wall bracket should be 40-50 cm above the floor. Mark the screw holes and remove the wall bracket. Drill five holes with a depth of 60 mm and a diameter of 8 mm and insert the dowels (2) into the holes.
- (B) Return the wall bracket to its original position and tighten the expansion screws (03).
- (C) Place the shower seat bracket (1) on the wall bracket from top to bottom until it is firmly seated on the wall bracket and completely covers it.
- (D) Secure the shower seat bracket with the two 5*8 mm Allen screws (05) by screwing the screws into the underside of the bracket using an Allen wrench.

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Chal-Tec GmbH, Wallstraße 16, 10179 Berlin, Allemagne.
Chal-Tec GmbH, Wallstraße 16, 10179 Berlin, Germania.