ONECONCEPT

## **Arielle Deluxe**

Duschsitz  
Shower Seat  
Silla para la ducha  
Siège de douche  
Sedile per doccia

10034210

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**Dear Customer,**

Congratulations on purchasing this device. Please read the following instructions carefully and follow them to prevent possible damages. We assume no liability for damage caused by disregard of the instructions and improper use. Scan the QR code to get access to the latest user manual and more product information.

---

**CONTENTS**

---

Safety Instructions 10  
Attachment Arielle Deluxe 10

---

**MANUFACTURER & IMPORTER (UK)**

---

**Manufacturer:**

Chal-Tec GmbH, Wallstrasse 16, 10179 Berlin, Germany.

**Importer for Great Britain:**

Chal-Tec UK limited  
Unit 6 Riverside Business Centre  
Brighton Road  
Shoreham-by-Sea  
BN43 6RE  
United Kingdom

---

## SAFETY INSTRUCTIONS

---

- Read the instructions carefully before installation.
- If the supplied screws and plugs are not suitable for installation, use suitable screws and plugs.

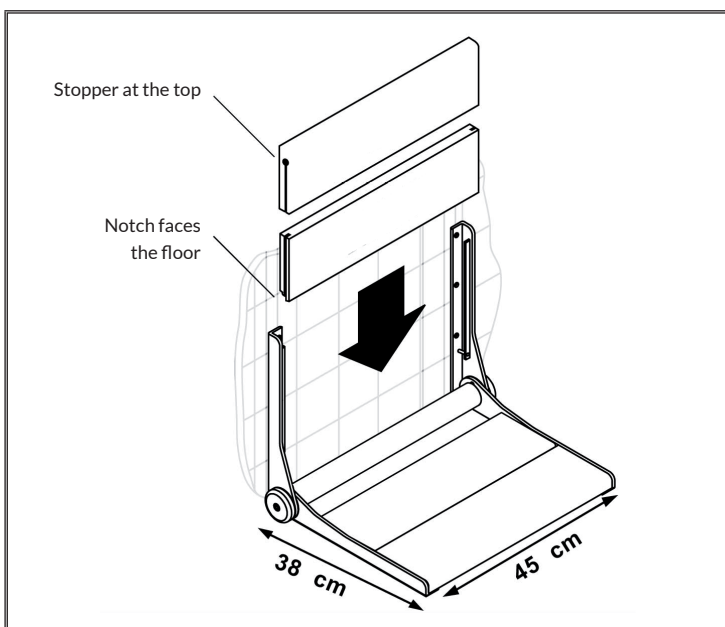
**Note:** The maximum user weight is 130 kg.

---

## ATTACHMENT ARIELLE DELUXE


---



### Dimensions


## Installation

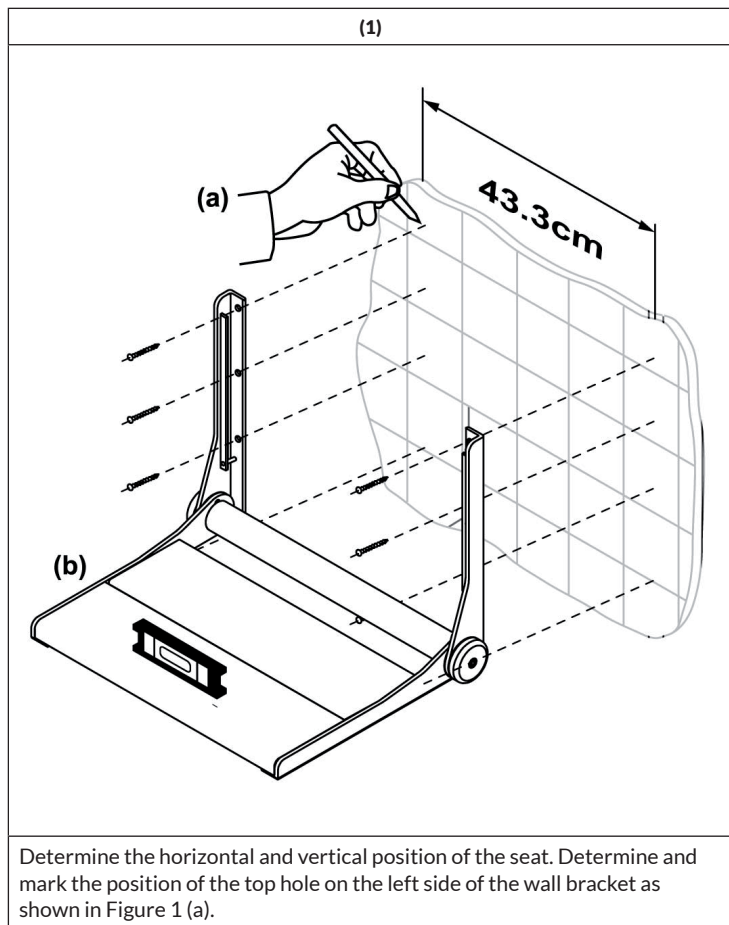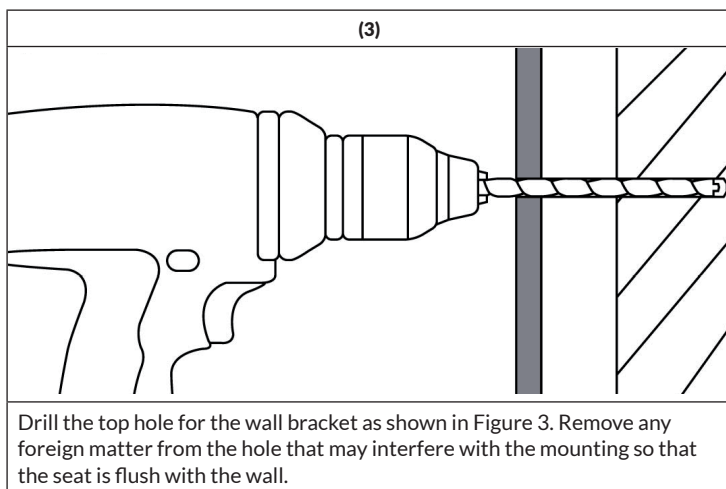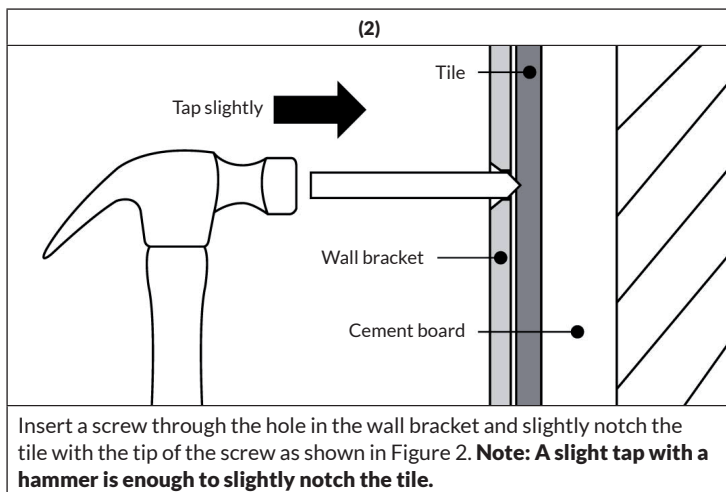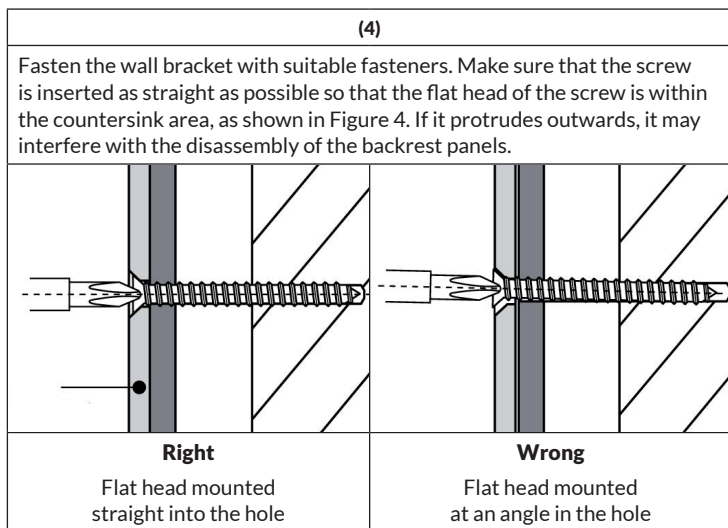**Note:** Place a blanket in the area where you will install the seat to protect the surfaces of the shower or bath from parts that may fall during assembly.

- (5) After mounting, use the spirit level shown in Figure 1 (b) to ensure that the seat is straight. Then determine the position of the top hole on the right side of the wall bracket.

---

**Important:** Ensure that the distance between the wall brackets allows easy movement of the backrest panels.

---

- (6) First repeat steps 2 to 4 for the upper hole on the right side of the wall bracket and make sure it is level.
- (7) Repeat steps 2 to 4 and the remaining holes.

**Note:** Perform a final suit and inspection of all fasteners. Test the installation by pressing the seat down or sitting on it.

- (8) Once all screws are in place, slide the backrest plates onto the wall bracket. Make sure they are seated correctly.

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Chal-Tec GmbH, Wallstraße 16, 10179 Berlino, Germania.