CAPITAL SPORTS

Kinesio Tape

10037715 10037716 10037717 10037718 10037719 10037720
10037721 10037722 10037723 10037724 10037725 10037744
10037745 10037746 10037791 10037792 10037793 10037794

Allgemeine Information

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Chal-Tec UK limited
Unit 6 Riverside Business Centre
Brighton Road
Shoreham-by-Sea
BN43 6RE
United Kingdom

General Information

Kinesiology Taping was developed by Dr. Kenzo Kase in Japan in the 1970s. In traditional Chinese medicine, therapy methods have been successfully used for thousands of years, which, like Kinesiology Taping, are based on the self-regulating abilities of the human body (e.g. acupuncture or acupressure). Initially, Kinesiology Taping was mainly used in competitive sports and thus became known in Europe as well.

Material

Capital Sports Essential Kinesiology Tapes are made of high quality cotton and an acrylic adhesive applied in waves. The Kinesiology Tapes are absolutely latex-free. Lengthwise, the tape can be stretched to 130-140% of its original length. The material has excellent adhesive strength, is breathable, skin-friendly and waterproof.

Application

The skin must be dry and free of grease, strong hair should be shaved off if necessary. Do not use the tape on wounds, burns and mucous membranes. If itching or other symptoms occur, the tape should be removed immediately. Before applying the tape, it must be rubbed to activate the full adhesive power of the acrylic adhesive. Shortly after applying the tape is no longer perceived as a nuisance. You can go showering, bathing or swimming with the taping without any problems. Afterwards you should dab the tape dry with a towel. Do not wear the tape for more than one week. Do not tear off the tape in a jerky way. Always pull the tape off in the direction of the hair growth. Clean the affected skin area immediately after removing the tape.

| |
|---|
| Warnings |
| Allergic reactions are registered only in very rare cases and are very often caused by medication intake as well as by increased alcohol or nicotine consumption. In case of existing hypersensitivity to plasters or tape dressings, the tape should first be tested on a smaller skin area. People with cancer should not use the tape because of possible interactions with treatment. Do not use on the abdomen during pregnancy. |

| |
|---|
| Warnings |
| Allergic reactions are registered only in very rare cases and are very often caused by medication intake as well as by increased alcohol or nicotine consumption. In case of existing hypersensitivity to plasters or tape dressings, the tape should first be tested on a smaller skin area. People with cancer should not use the tape because of possible interactions with treatment. Do not use on the abdomen during pregnancy. |

Storage instructions

Store the tape out of reach of children, out of direct sunlight and at normal room temperature.

| |
|---|
| Warnings |
| Allergic reactions are registered only in very rare cases and are very often caused by medication intake as well as by increased alcohol or nicotine consumption. In case of existing hypersensitivity to plasters or tape dressings, the tape should first be tested on a smaller skin area. People with cancer should not use the tape because of possible interactions with treatment. Do not use on the abdomen during pregnancy. |

Manufacturer
Chal-Tec GmbH, Wallstraße 16, 10179 Berlin, Germany.

| |
|-----------------------------------|
| Importer for Great Britain |
| Chal-Tec UK limited |
| Unit 6 Riverside Business Centre |
| Brighton Road |
| Shoreham-by-Sea |
| BN43 6RE |
| United Kingdom |

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Chal-Tec GmbH, Wallstraße 16, 10179 Berlin, Alemania.
Chal-Tec GmbH, Wallstraße 16, 10179 Berlino, Germania.
Chal-Tec GmbH, Wallstraße 16, 10179 Berlin, Allemagne.