

# A strong team for sensitive surfaces

## FD 366 sensitive Rapid surface disinfection



## FD 360 Vinyl upholstery cleaning and care

With FD 366 sensitive, you can perform fast, effective and gentle sensitive disinfecting between treatments for sensitive surfaces like plastics, dental units or acrylic glass. This product has been specifically developed for continuous protection of surfaces.

1. After every treatment, disinfect contaminated surfaces and objects.
2. Use FD 366 sensitive undiluted on a cloth and wash.
3. Wipe surfaces and objects to be disinfected completely.
4. Disinfect cleaned surfaces again.
5. Allow preparation to react.
6. Allow surfaces and objects to dry.

Use FD 360 to gently cleanse and maintain the vinyl upholstery surfaces of dental chairs and stools. The cleanser ensures cleanliness and – with a few millilitres weekly – a long-lasting glossy and shining surface. With FD 360 you protect the long-term value for your dental equipment.

1. Use FD 360 as required.
2. Spray the cleansing fluid undiluted on the surface to be cleansed or the special sponge.
3. Rub FD 360 in with a cloth or with a special sponge for intensive cleaning. This helps with an even more efficient deep cleaning.
4. Wipe off the residue of FD 360 with a dry cloth when finished.