

Dentapure™ Independent Water Bottle Cartridge DP365B Installation Instructions


Getting Started

Refer to instructions for use enclosed with the cartridge and also available at www.crosstex.com for precautionary statements, directions for use, storage, disposal and troubleshooting prior to first use of the new unit.

- ! Before you begin, please have the following available:**
- » The included Dentapure™ Cartridge DP365B installation kit containing the DP365B cartridge and Luer fitting.
 - » Scissors or knife

A short installation video can also be found at:
www.crosstex.com

For installation, or other Dentapure Cartridge questions, contact:
DentaPure Cartridge Technical Support
(631) 257-1033 / (888) 276-7783
dentapuretechnical@crosstex.com

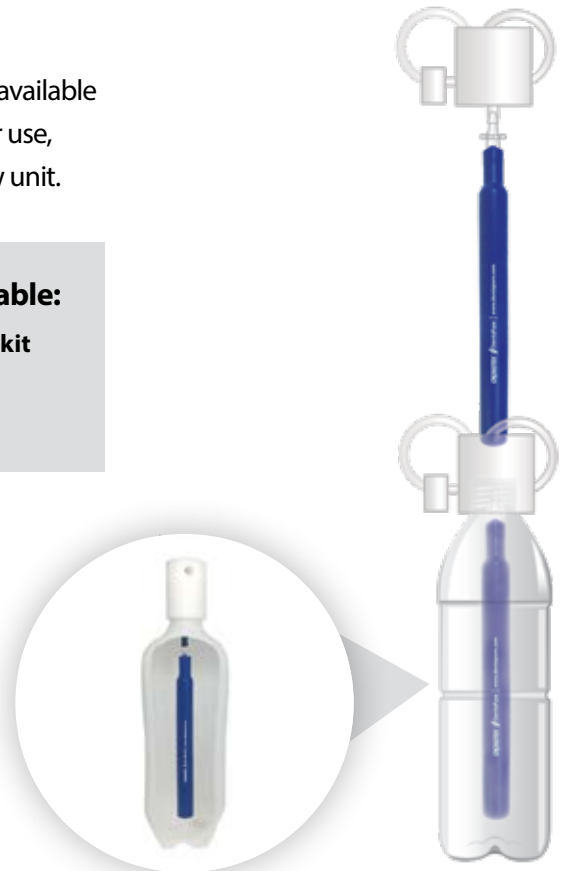

Install your DentaPure™ Cartridge in just a few easy steps:

1 Remove

Remove the dental unit's water bottle to reveal the pick-up tube.



2 Align

Align the bottle alongside the pick-up tube to ensure your cartridge can attach to the pick-up tube with enough space to fit inside the bottle (Leave ¼" to ½" of empty space from the bottom of the bottle).




3 Cut

Once measured, cut the pick-up tube and securely insert the included Luer fitting.



4 Twist

Take off the white or black protective end caps and attach the cartridge to the Luer fitting with a simple twist.

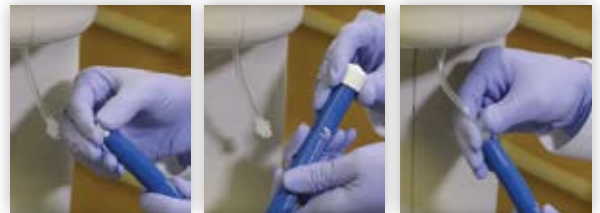

5 Fill

Fill the bottle with tap or distilled water and return to manifold. Check that the PSI is set to 45. Operate dental unit instruments until all air is purged from the water system.

Note: Initial dark discoloration of water is normal and is caused by the release of activated carbon. Operate water until flow appears clear.



All product names are trademarks of Crosstex International, Inc., a Cantel Medical company, its affiliates or related companies, unless otherwise noted.

© 2018 Crosstex International, Inc. 0918 DLIT00447 Rev A