


# HyFlex™ EDM

## Reduced Number of Files

The use of HyFlex™ EDM files allows the reduction of the number of files. In small, straight canals the usage of the HyFlex™ OneFile can be enough. Depending on the clinical situation the number of files can be reduced or expanded. All HyFlex EDM NiTi Files should be used at 400 rpm and at a torque of up to 2.5 Ncm (25 mNm) excepted the Glidepath files which should be used with 300 rpm and at a torque of up to 1.8 Ncm (18 mNm).

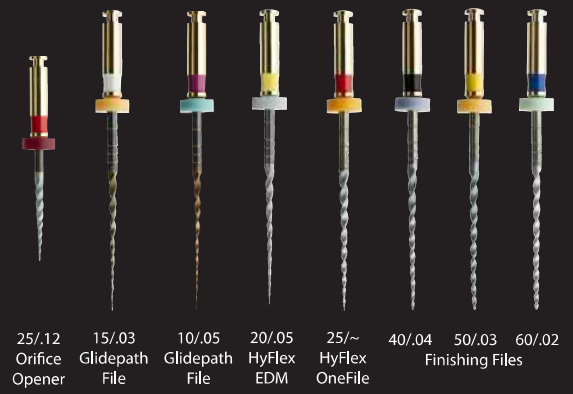

	Optional Access Opening	Glidepath	Shaping	Optional Finishing
	 25/12 Orifice Opener	 Hand File 10/05 Glidepath File	 25~ HyFlex OneFile	 40/04 50/03 60/02 Finishing Files
	 25/12 Orifice Opener	 Hand File 10/05 Glidepath File	 20/05 HyFlex EDM 25~ HyFlex OneFile	 40/04 50/03 60/02 Finishing Files
	 25/12 Orifice Opener	 Hand File 15/03 Glidepath File 10/05 Glidepath File	 20/05 HyFlex EDM	

# Order Informationen

HyFlex EDM Sets	21 mm	25 mm	
HyFlex EDM Shaping Set Simple 25/12; 10/05; 25/~	60022267	60019363	
HyFlex EDM Shaping Set Medium 10/05; 20/05; 25/~	60022277	60022278	
HyFlex EDM Shaping Set Max Curve 15/03; 10/05; 20/05	60023494	60023495	
HyFlex EDM Finishing Set 40/04; 50/03; 60/02	60022268	60019364	
HyFlex EDM Refills, 3 pcs	15 mm	21 mm	25 mm
Hyflex EDM 25/12 Orifice Opener	60019596		
HyFlex EDM 15/03 Glidepath Feile		60023493	60023492
HyFlex EDM 10/05 Glidepath Feile		60022269	60019594
HyFlex EDM 20/05 Preparation Feile		60022275	60022276
HyFlex EDM 25/~ OneFile		60022270	60019595
HyFlex EDM 40/04 Finishing Feile		60022271	60019597
HyFlex EDM 50/03 Finishing Feile		60022272	60019598
HyFlex EDM 60/02 Finishing Feile		60022274	60019599
HyFlex Accessories			
3 x Endo Practice Blocks, Acryl		60011085	

Size / taper	HyFlex EDM Paper points 100 pcs	HyFlex EDM GuttaperchaPoints 60 pcs
20/05	60022742	60022706
OneFile 25/~	60022743	60022707
40/04	60022676	60022708
50/03	60022741	60022709
60/02	60022740	60022710
Assortment 20/05 - 60/02	60022744	60022705



New user video for safe and efficient root canal preparation with HyFlex EDM NiTi files. ▼



<https://bit.ly/3h6BOYo>

**Coltene/Whaledent Ltd.**  
The President Suite  
Kendal House, Victoria Way  
Burgess Hill, West Sussex  
RH15 9NF / UK  
T +44 1444 235486  
F +44 1444 870640  
[info.uk@coltene.com](mailto:info.uk@coltene.com)

**Coltene/Whaledent AG**  
Feldwiesenstrasse 20  
9450 Altstätten / Switzerland  
T +41 71 757 5300  
F +41 71 757 5301  
[info.ch@coltene.com](mailto:info.ch@coltene.com)

**Coltene/Whaledent GmbH + Co. KG**  
Raiffeisenstraße 30  
89129 Langenau / Germany  
T +49 7345 805 0  
F +49 7345 805 201  
[info.de@coltene.com](mailto:info.de@coltene.com)

 **COLTENE**