

The Soundbright app provides you with complete control over your Soundbright Discovery hearing aids! This guide will walk you through how to download the app on your smartphone, pair your hearing aids, and control them with the app.

### Download the Soundbright app:






Android users can scan the QR code to the left or search for "Soundbright" on Google Play app store




iOS users can scan the QR code to the left or download or search for "Soundbright" in the App Store

### Connect your Discovery hearing aids to the Soundbright app:

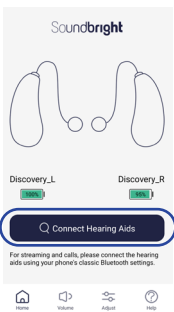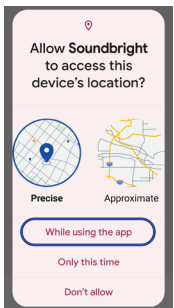
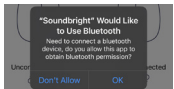


1. Turn your hearing aids on by **removing them** from the charging case
2. Enable the **Bluetooth function** on your smartphone
3. Open the **Soundbright app**
4. Allow **Bluetooth and Location permissions** in the app if asked
5. Click the **"Connect Hearing Aids"** button
6. After the **"Searching for Devices"** is completed, the app will automatically connect to the hearing aids
7. You will see the "Discovery\_L" and "Discovery\_R" in your Soundbright app as well as the battery percentage of each hearing aid.


## Control your hearing aids in the Soundbright app:

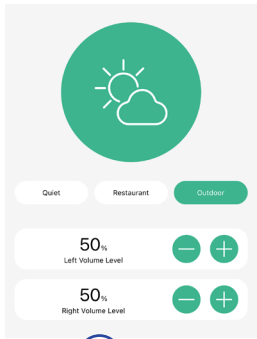

1. Click the "Volume" button to control settings:
  - Click the "Volume" button to control settings:
    - Volume: increase or decrease overall volume across all frequencies
    - The "Left Volume Level" and "Right Volume Level" can be adjusted separately
    - Mode: select one of the three modes for specific listening situations
      - Quiet
      - Restaurant
      - Outdoor

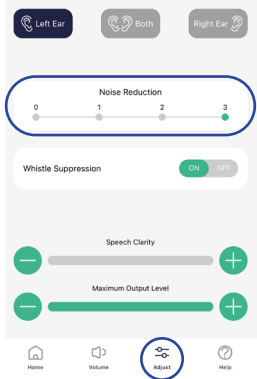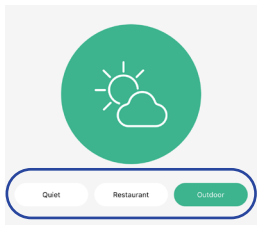

2. Click the "Adjust" button to fine tune each hearing aid:
  - Noise Reduction
    - Filter out background noise to improve speech understanding
  - Whistle Suppression toggle switch
    - Only enable this if you frequently have issues with feedback or whistling
  - Speech Clarity
    - Boost the volume of frequencies that are key to understanding speech
  - Maximum Output Level: set the maximum amplification level of the hearing aid to ensure loud sounds are not uncomfortably amplified
    - Best for use in noisy environments or situations with sudden loud noises

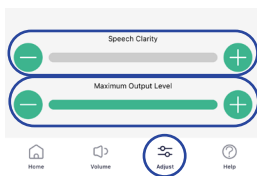

3. Each hearing aid can be adjusted separately or together by clicking the "Left", "Both", or "Right" buttons



4. All settings will be saved each time you turn off your hearing aids or reconnect them to the app