

enovid™ — USER GUIDE

ENG

Indication for use:

enovid™ is a Nasal Spray medical device intended for use as a protective mechanical and chemical barrier against viruses within the nasal cavity. enovid™ may be applied 3 to 4 times over the course of the day (two sprays per nostril each time), or up to 6 times a day following a suspected exposure to viruses.

enovid™ is a nasal spray which forms a physical barrier, destroys viruses and prevents them from entering the body.

enovid™ was scientifically tested and proven to kill 99.9% of viruses within 2 minutes.

enovid™ has been clinically tested and shown to be safe for human use.

enovid™ can be used in addition to other methods to protect against viruses, as part of an overall strategy to prevent viral infection. For example, enovid™ may be used in addition to using hand sanitizer and wearing a mask.

enovid™ is considered a medical device, not a drug. It does not cause drowsiness.

Who can use enovid™?

enovid™ is suitable for adults and children older than 12 years of age.

Contraindications:

Children under 12, consult a physician before use.

If pregnant or breast-feeding, consult a physician before use.

Do not use this product if you are sensitive to any of the ingredients.

⚠ Cautions:

- **To avoid contamination do not share the nasal spray applicator with anyone.**
- **Only use up to 6 times a day.**
- Keep out of reach of young children.
- Avoid contact with eyes. If the spray gets into your eyes, rinse with water.
- Do not swallow (not for ingestion).
- After use, always return the protective cap to the sprayer.
- If you experience reactions such as headache, nosebleed, or persistent burning sensation, discontinue immediately. If symptoms persist, consult your physician.
- Keep away from fire.

How to store enovid™?

Store at room temperature. Do not freeze.

Discard enovid™ sprayer on the expiration date printed on the sprayer or 60 days after first opening, whichever comes first.

How to use enovid™?

To defend against viruses, enovid™ may be applied 3 to 4 times daily (2 sprays per nostril each time) or up to 6 times per day after potential exposure, to maintain an effective barrier against viruses.

Remember - enovid™ sprayer cannot be shared with other people – this will prevent potential transmission of viruses or germs from person to person.

enovid™ sprayer contains 25 mL of solution.

Once applied it becomes effective immediately.

Before you start using enovid™ for the first time, prime the sprayer by squirting it away from your body into the air until a consistent spray is generated (only with first dose).

Please participate in a customer survey, at the following link - <https://sanotize.com/customer-survey/>

Directions for Use



Step 1 – Blow your nose gently before using the spray.



Step 2 – Wash your hands thoroughly with soap and water.



Step 3 – Align the triangles on cap and bottle and pull up the protective cap, then **remove the safety clip**. Prime the sprayer until a consistent spray is generated (only with first dose).



Step 4 - Gently insert the bottle tip just into one nostril. Press on the other side of your nose with one finger to close off the other nostril. Aim slightly away from the center of your nose and pump one spray. Repeat in the other nostril. Repeat in each nostril for a total of 4 sprays (2 in each nostril). Sniff gently through the nose while applying but do not inhale deeply. Exhale through your mouth.



ENSURE SPRAYER IS HELD VERTICALLY.



Step 5 - Wipe the sprayer with a clean tissue to maintain hygiene.

You may also feel some form of mild irritation, especially if your nasal membranes are already inflamed or irritated. This will subside quickly. **enovid™ sprayer should be used by one person only. Sharing a sprayer with other people could expose them to infection.**

Components:

Sodium chloride, Citric acid, HPMC, Sodium nitrite, Benzalkonium Chloride. Do not use the spray if you are sensitive to any of its components.

Manufacturer:
SaNotize R&D Israel Ltd.
58 Harakevet st., Tel Aviv 6777016, Israel
Tel.: +972-3-7787333
Email: info@sanotize.com
Website: www.sanotize.com

Marketing and Distribution:
Tradis Gat Ltd.
32 Shacham St. Petach Tikva, Israel
Tel.: +972-3-9260400
Email: info@tradisgat.com
Website: www.tradisgat.com



Registration No. In Israel MOH (AMAR): 37860001

Symbols Glossary:

	Legal Manufacturer		Do not use if package is damaged
	Caution, consult accompanying documents		Minimum and maximum storage temperatures
	Consult instructions for use		Keep away from sunlight
	Batch Number		Expiry date (In format: DD-MM-YYYY)
	Keep away from fire		Keep dry
	Single patient multiple use		

