


FACE SHIELDS & GASKET SEALING GOGGLES

Face shields or gasket sealing goggles are to be worn during all direct patient care situations. They come in a variety of forms. Please use this flyer as a reference for how your CMH-issued PPE is to be worn.



Reusable - Single User

These goggles are assigned to a single user. They should be wiped thoroughly with hospital-grade disinfectant after each removal. Lenses may need to be rinsed with water, see manufacturer instructions.

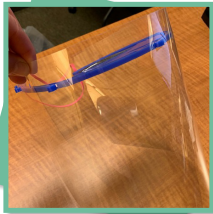

Reusable - Single User

These face shields are assigned to a single user. When removed, they must be wiped with a hospital-approved disinfectant or placed face down on a paper towel to prevent cross-contamination.



The single user reusable shield with a black foam pad at the top must have the foam pad sprayed at the end of each shift and hung to dry. Spray with Clorox hydrogen peroxide spray.



Reusable - Between Staff/Multiple Users

This type of face shield can be used by multiple users. It must be thoroughly disinfected with a hospital-approved disinfectant between users. When removed by a user, it must be wiped with a hospital-approved disinfectant or placed face down on a paper towel to prevent cross-contamination.



Disposable - Single User

Face shields of the Waymaker or Key Surgical brand should be used by a single user for an entire shift and thrown away at the end of the shift. When removed by a user, it must be wiped with a hospital-approved disinfectant or placed face down on a paper towel to prevent cross-contamination.