

Affiliate Provider Imaging Access Portal

Request Access: email KP-SCAL-OnlineAffiliate@kp.org

1 <https://mitipublic.kp.org/viewimages.php>

KAISER PERMANENTE

Affiliate Provider Imaging Access Portal

Request Access Password Resets: 1-866-529-0933

Sign On

Kaiser Permanente providers, affiliates, and medical office staff: Please sign on to gain access to secure features.

User ID

Password

2

3

1. Type <https://mitipublic.kp.org/viewimages.php> in the address bar.
2. Enter **User ID** and **Password**.
3. Click **Sign On**.



Affiliate Provider Imaging Access Portal

Patient Search

Welcome, **[User Name]**

This session will expire at **11:01:01 AM on Mon Dec 09 2019**.

All patient identifiers are required.

Patient Last Name

Patient First Name

Patient Medical Record Number

4

5

Expire in 5 minutes after the initial logged in.

NOTE - the "View" button above is time-limited in nature and will expire **5 minutes** after it was generated. The link will expire at **10:47:02 AM on Mon Dec 09 2019**.

4. Enter **Exact Last Name**, **Exact First Name** and **Exact Medical Record Number** that exist in Kaiser Permanente's Systems.
5. Click **Search**. Carestream Vue Motion website launches with the Patient's Studies.

For MITI Support: 866-529-0933

Carestream Vue Motion

The screenshot shows the Carestream Vue Motion interface for patient Valens, Misty Stroke N. The interface includes a patient history sidebar, a main image viewer, and a bottom toolbar. Callouts provide the following instructions:

- Filter by Modality, Body part or Description.** (Points to the filter dropdowns in the Patient History sidebar)
- Highlighted study opened. Scroll to see other studies and double click to open the study.** (Points to a study entry in the list)
- Click left icon to see Series Chooser. Click the left or right double arrows to move backward or forward.** (Points to navigation icons in the toolbar)
- Tools to manipulate the images but the images will **NOT** be saved.** (Points to manipulation icons in the toolbar)
- Scroll bar to views stacked images or use the mouse to scroll.** (Points to the vertical scroll bar on the right side of the image viewer)

 **Note:** CareStream Vue Motion is supposed to automatically open up in a separate tab (that way the “View” button doesn’t need to be clicked); HOWEVER, if the user’s browser has automatic pop-up/redirection blocked, then CareStream Vue Motion will NOT automatically open as intended. The user has to either disable the blocking altogether in the browser settings, or they need to acknowledge the browser message that a pop-up has been blocked and allow for CareStream Vue Motion to load (many browsers will allow you to “save” the override for a given page when you do this, and users are free to choose that option).

In any case, the user can always click on the “View” button to manually launch the CareStream Vue Motion tab/window.

For MITI Support: 866-529-0933