



PROVIDER *bulletin*

This is an official **CDC HEALTH ADVISORY**

Distributed via the CDC Health Alert Network
May 17, 2017, 0900 ET (9:00 AM ET)
CDCHAN-00403

Potential for Falsely Low Blood Lead Test Results from LeadCare® Analyzers

Summary

The U.S. Food and Drug Administration (FDA) has issued a safety communication warning about the use of Magellan Diagnostics' LeadCare® analyzers (LeadCare, LeadCare II, LeadCare Ultra and LeadCare Plus) with venous blood samples because they might result in falsely low test results. FDA is now advising that Magellan Diagnostics' LeadCare® analyzers should no longer be used with venous blood samples. The safety alert does not apply to capillary blood lead test results collected by fingerstick or heelstick. The purpose of this Health Advisory is to notify state and local health departments, healthcare providers, and laboratories about CDC's re-testing guidance in light of the safety alert.

Recommendations

CDC recommends that healthcare providers re-test patients who:

- 1) are younger than 6 years (72 months) of age at the time of the alert (May 17, 2017) and
- 2) had a venous blood lead test result of less than 10 micrograms per deciliter ($\mu\text{g}/\text{dL}$) analyzed using a Magellan Diagnostics' LeadCare® analyzer at an onsite (e.g., healthcare facility) or at an offsite laboratory.

CDC also recommends that healthcare providers re-test currently pregnant or lactating women who had a venous blood lead test performed using a Magellan Diagnostics' LeadCare® analyzer.

CDC recommends parents discuss re-testing with their healthcare provider or health department to determine if their child's blood should be re-tested.

If re-testing indicates blood lead levels in excess of the CDC reference level

(www.cdc.gov/nceh/lead/acclpp/blood_lead_levels.htm), or the state or local action level, the healthcare provider or public health official should refer to CDC and/or local guidelines for appropriate follow-up action (www.cdc.gov/nceh/lead/acclpp/actions_blls.html).

Re-tests are not recommended if the provider is certain that analyzers other than those described by this Health Advisory were used to analyze the venous blood samples.

For future blood lead testing, healthcare providers and public health officials should:

- Send venous samples to Clinical Laboratory Improvement Amendments (CLIA)-compliant

9700 Stockdale Highway, Bakersfield, CA 93311-3617

(661) 664-5000 • Fax (661) 664-5151

www.kernhealthsystems.com

The Heart of Kern Family Health Care





PROVIDER *bulletin*

laboratories using inductively coupled plasma mass spectrometry (ICP-MS) or graphite furnace atomic absorption spectrometry (GFAAS) (also known as electrothermal atomic absorption spectrometry [ETAAS]) instruments.

- Send capillary samples to CLIA-compliant laboratories using any CLIA compliant analyzer including ICP-MS, GFAAS, or LeadCare® analyzers.

For More Information

CDC's Lead Poisoning Prevention Program: <https://www.cdc.gov/nceh/lead/>

CDC's Lead and Multi-element Proficiency Program: <https://www.cdc.gov/labstandards/lamp.html>

Reference

FDA safety communication warning, May 17, 2017. Available at:
<https://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm558733.htm>

The Centers for Disease Control and Prevention (CDC) protects people's health and safety by preventing and controlling diseases and injuries; enhances health decisions by providing credible information on critical health issues; and promotes healthy living through strong partnerships with local, national, and international organizations.

