

Enkola ya KOVA Plastics e Egobererwa Okukebera Omusulo

ENKOLA YA KOVA PLASTICS EGOBERERWA OKUKEBERA OMUSOLO

Okukebera omusulo, nga bwe kukolebwa mu laabu nnyingi, kukolebwa mu ngeri nnyingi ezitagoberera mitendera egirina okugobererwa. Emitendera gino gyawukana okuva ku laabu emu okudda ku ndala era emirundi mingi akakodyo kennyini akakozesebwa mu laabu kakyukakyuka okusinziira ku ani akebera.

Ebiviirako enjawukana mu kukebera omusulo okwa bulijjo:

- obungi bw'omusulo obwawukana
- enkamula ez'enjawulo ezireetera omuwendo gw'ebisejja eby'okwekebejja mu microscope okukyukakyuka
- obungi bw'ebisejja ebikunanyiziddwa n'okuyiibwa wansi w'ekisaanikira ky'ekyuma okwawukana
- obukodyo obw'enjawulo abantu ssekinnoomu bwe bakozeza.

Okusobola okuteekawo enkola y'okukebera omusulo egobererwa ey'awamu, obungi bw'omusulo ogukeberegwa, amaanyi agakozesebwa okukamula n'obungi bw'ebisejja birina kuba nga tebikyuka, era wateekwa okubaawo enkola emu ekozesebwa okukebeza microscope n'okulaga ebivuddemu mu bikeberwa. Enkola ya KOVA Plastics etuukiriza engoberera ey'awamu eno ng'ekendeeza ebyawukana, n'obukodyo obw'enjawulo abakebezi bwe bakozeza.

EKIGENDERERWA MU KUGIKOZEZA

Enkola ya KOVA Plastics ewa omutendera n'ebintu ebisobola okukozesebwa okufuna ebivaamu ebitalyukakyuka mu kukebera omusulo okwa bulijjo. Okufuga obungi, obutakyukakyuka n'obuyonjo bitandikira ku kukung'anya n'okutambuza ebisejja by'omusulo okutuuka ku kugwekebejja mu microscope. Enkola ezigobererwa zisobola okukozesebwa okusobola okulambika omutindo gw'ebikebererwa mu buliwo, eddagala erikozesebwa n'emitendera egigobererwa okukebera mu microscope.

EBIRUNGI EBIRIMU

Singa omutendera ogulagiddwa gugobererwa nga tegukyusibwakyusibwa, omuntu asobola okukozesa emiwendo egifuniddwa mu kukebera omusulo awatali kubusabuusa kwonna. Abasawo basobola okulondoola enkyukakyuka mu mbeera y'abalwadde n'obujanjabi obubaweebwa nga bakakafu; enkyukakyuka zonna ezibeerawo enkola eno z'ekkiriza nga zibadde tezisubirwa zisobola okutwalibwa nga nkulu. Laabu ziyinza okugeraageranyizibwa era abalwadde abali mu kwetegerezebwa basobola okukebererwa omusulo gwabwe mu laabu ez'enjawulo ebivaamu ne bigeraageranyizibwa.

ENKOLA YA KOVA PLASTICS N'EBIGIRIMU

Ennamba y'Ekyamaguzi	Ebikwata ku Kyamaguzi	Ebiri mu Buli Kitereke
87153E	KOVA Plastics System Super Pac ey'o 1000 n'Obusaanikira KOVA Plastics Glasstic 100 Slide 10 (10 mu kasenge), Obuseke 1000 bwa KOVA Plastics, Obukopo Obunene obwa KOVA Plastics 1000, Obusaanikira bwa KOVA Plastics 1000	1000
87154E	KOVA Plastics System Super ey'o 1000 KOVA Plastics Glasstic 100 Slide 10 (10 mu kasenge), Obuseke 1000 bwa KOVA Plastics, Obukopo Obunene obwa KOVA Plastics 1000	1000
87162E	KOVA Plastics Super Pac ey'o 1000 n'Ebiba KOVA Plastics Glasstic 100 Slide 10 (10 mu kasenge) n'Ebiba, Obuseke 1000 bwa KOVA Plastics, Obukopo Obunene obwa KOVA Plastics 1000	1000
87155E	EKOVA Plastics System Pac II 100 KOVA Plastics Slide II (ya busenge 4), Obuseke 400 bwa KOVA Plastics, Obukopo Obunene obwa KOVA Plastics 400	400
87156E	KOVA Plastic System Value Pac ey'e 500 KOVA Plastics Glasstic 50 Slide 10 nga zirina ebiba Obukopo 500 obwa KOVA Plastics Economy, Obusaanikira 100 obwa KOVA Plastics	500
87158E	KOVA Plastics System Value Pac ey'e 500 n'Ebiba KOVA Plastics Glasstic 50 Slide 10 (10 mu kasenge), Obuseke 500 bwa KOVA Plastics, Obukopo Economy obwa KOVA Plastics 500	500
87141E	KOVA Plastics KO-LEC-PAC Obukopo 500 obwa KOVA Plastics Super, Obusaanikira 500 obwa KOVA Plastics, Ebikopo 500 ebya KOVA Plastics, Obulamba 500 n'Obutandaalo kwe Babitambuliza 5	500
87100E	KOVA Plastics Slide II nga erina Ekiba eky'okupimisa; Obunnya bw'amazzi obwa 100 x 4; nga buli kamu ka kiba eky'enkanankana ekya 1mm x 1mm	400
87118E	KOVA Plastics Slide II(omutali kiba)	400

87146E	Obunnya bw'amazzi 100 x 4 KOVA Plastics Glasstic Slide IO Obunnya bw'amazzi obwa 100 x 10 obwa pulasitiika atangaala obulungi ddala	1000
87157E	KOVA Plastics Glasstic Slide IO Obunnya bw'amazzi obwa 50 x 10 obwa pulasitiika atangaala obulungi ddala	500
87144E	KOVA Plastic Glasstic Slide IO n'Ekiba Obunnya bw'amazzi obwa 100 x 10 obwa pulasitiika omugumu era atangaala obulungi ddala* ng'aliko n'ebipimo ebiba; buli kasenge kalimu µl 6.6 era ka mm 3 x mm 3 ebiba ebyawuddwa obulungi ebya mm 0.33 x mm 0.33. Omutendera gw'okukebera mulimu enkola y'okupima obutoffaali bwa buli µl ya sampolo z'abalwadde.	1000

KOVA PLASTICS SYSTEM N'EBIGIRIMU - KYONGEDDWAKO

Ennamba y'Ekyamaguzi	Ebikwata ku Kyamaguzi	Ebiri mu Buli Kitereke
87137E	KOVA Plastics Super Tube Ke bakunanyizaamu n'okukamuliramun nga karambiddwako naye nga tebasoose kutta buwuka akakola omulundi ogumu obukopo, obukoleddwa mu pulasitiika omugumu atamenyeka okufuna olunyaafa oba okwatika nga bakamula.	500
87138E	Akakopo ka KOVA Plastics Economy Nga waggulu naye mu pulasitiika atali wa bbeeyi, omugumu akoleddwa mu styrene.	500
87135E	Akaseke ka KOVA Plastics Akaseke ka pulasitiika ak'okukyusizaamu akakola omulundi gumu nga kakolebwa okuterekamu ml 1.0 ey'omusulo ogumaze okukamulwa. Akasiba akasongovu ak'enjawulo kawa enkola y'okukamula ey'omulundi ogumu etaliimu kucaafuwazibwa.	500
87139E	Akasaanikira ka KOVA Plastics Kasembewwa olw'okwewala okuyiwa ebitambuzibwa, nga kw'otadde n'okugendamu obucaafu mu kukamula.	500
87136E	Akatandaalo ka KOVA Plastics Akayibwako Ebintu Akatandaalo akayibwako ebikebererwa ebiwera e 10.	1 Akatandaalo

OKUKUNJANYA EBIKEBERERWA N'OKUBITAMBUZA

- Enkola ya KOVA Plastics eya KO-LEC-PAC ewagirwa okukozesebwa mu ngeri bw'eti:
1. Lamba Obukopo bwa KO KOVA Plastics era omulwadde omuwe oz 3 1/2. Ekikopo kya KOVA Plastics.
 2. Lagira omulwadde afuuyise omusulo mu Kikopo kya KOVA Plastics.
 3. Kyuusa omusulo okuva mu Kikopo kya KOVA Plastics oguzze mu Kakopo ka KOVA Plastics, okajjuze okutuuka ku walambiddwa ml 12.
 4. Saanikira akasaanikira ka KOVA Plastics ku Kakopo ka KOVA Plastics bw'omala okateke ku Katandaalo Akatambuza KOVA Plastics okukatambuza n'okukatwala okukateraka.
 5. Bituuse mu laabu amangu ddala nga bwe kisoboka, naddala mu ssaawa bbiri, naye nga tezisukka ssaawa nnya ng'ebikeberwa bimaze okuggyibwako.


OMUTENDERA GW'OKUKEBERA KOVA PLASTICS SYSTEM

1. Kebera obuzito bwennyini ng'oteeka ettondo ly'omusulo limu oba abiri mu kipima ebbugumu oba kozesa akawuzi akakebera eddagala akaliko ekipimo ky'obuzito era owandiike ebivuddemu.
2. Ng'okozesa obuwuzi obukebera ebirungo, kebera eddagala okusinziira ku biragiro by'omukozi. Wandika ebivuddemu by'olabye. Wateekwa okutekebawo ebifuga buli kibinja okulondoola oba nga ebikozesebwa, eddagala n'emitendera gy'okukebera mu microscope bya mutindo.
3. Kamula Obukopo bwa KOVA Plastics (buli kamu nga kalimu ml12 ez'omusulo ogukebererwa oba Ekijulizibwako) ng'okozesa amaanyi matonotono agali ku 400 okumala eddakiika ttaano; nga keetooloola emirundi 1500 buli ddakiika n'ekiwujjo ekiweza obugazi bwa yinchi 6. Fomula ekozesebwa:

$$rcf = 28.38 (R) \left(\frac{N}{1000} \right)^2 \frac{R}{N} = \text{obuwanvu bw'akalimi mu yinchi}$$

$$= \text{emirundi egyetooloolwa buli ddakiika}$$

Ebbanga eryetooloolwa lye bbanga eripimibwa okuva akalimi we katandikira okutuuka amazzi we gakoma munda mu bukopo okuviira ddala awawanvu ng'opimye okuva akalimi we katandikira.

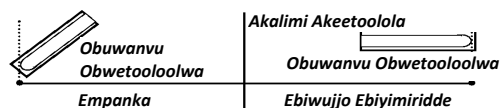

- Ggyamu Obukopo bwa KOVA Plastics mu kikamulirwamu, nga weegendereza obutataataaganya oba okuseetula ebisejja.
- Sonseka Akaseke ka KOVA Plastics mu Kakopo ka KOVA Plastics. Sindika Akaseke ka KOVA Plastics okutuukira ddala wansi mu Kakopo ka KOVA Plastics okutuusa nga katuddemu bulungi (awalambiddwa ne ml1).
- Ttulula oyiwe ml11 okuva mu Kakopo ka KOVA Plastics ng'Akaseke ka KOVA Plastics okasibidde mu kifo kimu mu Kakopo ka KOVA Plastics. Kino kijja kulekamu ml1 ey'ebisejja by'omusulo ku ntobo y'Akakopo ka KOVA Plastics.
- Ggyamu Akaseke ka KOVA mu Kakopo ka KOVA Plastics.
- Teeka ettondo ly'eddagala limu mu ml1 ey'ebisejja by'omusulo. Weetegereze: Eddagala liyambako mu kwawula obutoffaali bw'ebintu ebingi ate lya kulondako.
- Ng'okozesa Akaseke ka KOVA Plastics, ddamu owanike ebisejja n'eddagala n'obwegendereza okutuusa nga byetabudde bulungi wamu.

EMITENDERA GY'OKUKEBERA KOVA PLASTICS SYSTEM - Eyongeddwako

- Ggyamu sampolo ntono okuva mu bisejja by'omusulo by'otabudde n'eddagala ng'oniyaga akapeesa k'Akaseke ka KOVA Plastics.
- Kyusa ebisejja ebitabuddwa bidde mu KOVA Plastics Slide ng'oteeka ettondo limu mu muwaatwa ogusaliddwa ku buli kasenge. Obusenge 1-5 bwe buba ku lunyiriri olusooka waggulu, omuwaatwa gubeera mu kasonda k'obusenge akali ku kkono, obusenge 6-10 bwe buba ku lunyiriri olusooka waggulu, omuwaatwa gubeera waggulu mu kasonda k'obusenge akali ku ddyo. Akasenge kaja kujjula okuyita mu kukulukuta. Weewale okukwata ku kasengezo akaakula nga V akali wakati w'obusenge ng'oggyamu amazzi. Singa obikwata obubi ng'obiggyamu biyinda okukulukuta nga biva mu kasenge akamu ne bidda mu kalala.
- Siimuula ebikeberegwa byonna ebisigadde w'okoledde ng'oniyagawo n'ekintu ekisobola okubinuunawo.
- Teeka KOVA Plastics Slide mu kifo microscope w'ekeberera ng'ogitunuzza mu kitangaala.
- Tangaaza akasenge akasonsekebawamu ng'okozesa amaanyi agazimbulukusa matono (10X eriiso eritangaaza/10X ky'otangaaza) okubala obucu. Bala ebintu ebirala byonna ebitondeddwa olw'okozesa amaanyi amangi agazimbulusa (10X eriiso eritangaaza/40X ky'otangaaza). Tokozesa nate bintu bya KOVA Plastics ebimaze okukozesebwa.

Okusobola okwekebejja ebintu by'ozimbulukusizza ebiri mu biba, laba EMITENDERA GY'OKUKEBERA EGYA KOVA PLASTICS SYSTEM - GRIDDED

EMIWENDO EGISUBIDDWA - MICROSCOPY†

1 + = Ffoomu ekozesebwa obw'olumu
2 + = Ekozesebwa mu buli kisaawe
3 + = Omuwendo omunene mu buli kifo
4 + = Ebifo byonna

HPF = Ekifo ekirabibwa n'Amaanyi Amangi
Agazimbulukusa 400X
LPF = Ekifo ekirabibwa n'Amaanyi Amatono
Agazimbulukusa 100X


Ekyekebejebwa	Ekya bulijjo	Ektali kya bulijjo	Okuwa Alipoota y'Ebiyuddemu
WBC	0-5/HPF	> 5/HPF	Ennamba/HPF
RBC	0-3/HPF	> 3/HPF	Ennamba/HPF
Obutoffaali bw'Olususu	0	Ekirala kyonna (ng'oggyeeko obutoffaali obukola olususu)	Ennamba/HPF
Ebitole Ebisengeke	0-3/HPF (obutaleeta ndwadde)	> 3 ku kyonna ektali kya bulijjo	Ennamba/HPF
Ebizimbulukusa	0	Kyonna	1 + ku 4 +/-HPF
Trichomonads	0	Kyonna	1 + ku 4 +/-HPF
Obucu	0	Kyonna naddala > 1 ebicu bya hyaline/LPF	Ennamba/LPF
Obuwuuka obusirikitu	0-5/HPF	> 5/HPF	1 + ku 4 +/-HPF
Amasavu	0	Obunyama obw'amasavu oba obutaliiko masavu	1 + ku 4 +/-HPF

† Bernard Statland, MLO. p 13-14; Jan. 1985

EBY'OKUJULIZAAKO OKUMANYA OBUBAKA OBWAWAMU

- Bradley, G.M., Benson, E.S., Todd-Sanford Clinical Diagnosis by Laboratory Methods, 15th Edition, Phila. Saunders, 1974.
- Kurtzman, N. A. and Rogers, P. W. (1974). A Handbook of Urinalysis and Urinary Sediment. Chas. C Thomas, Springfield, IL.
- Little, P. J. (1962). Urinary white-cell excretion. Lancet. pp. 1149-1151.
- Little, P. F. (1964). A comparison of the urinary white cell concentrations with the white cell excretion rate. Brit. J. Urol. 36, 360-363.
- Thomas, M. (1971) nga bano. A rapid slide method of urine cell counts. Med. Lab Technol. 28, 38-39.
- Moore, T., Hira, N. R., and Stirland, R. M. (1965). Differential urethrovesical urinary cell count. Lancet. pp. 626-627.
- Siegle, MD, Lab Med., 12:781, 1981.
- Sternheimer, R. ne Malbin, B. (1951). The clinical recognition of pyelonephritis with a new stain for urinary sediments. Am. J. of Med., 11:312-323.
- Muschetta, P. A. and Waters, Jr. F. O. (1962). Manual of Medical Laboratory Techniques. Herbert-Spence, Inc. New York, N.Y., Second Edition, pp 44-45.
- Lippman, R. W. (1957). Urine and the Urinary Sediment. Chas. C Thomas, Springfield, IL.
- Dudas, H. C., Lab Med., nga bano. 12:765. 1981.
- Weller, J. M. and Greene, J. A. (1966). Examination of the Urine. Meredith Publishing Co., New York.
- Albert Rabinovitch MD, PhD, Clinical And Laboratory Standards Institute, GP16-A3, Urinalysis; approved guideline – third edition Feb 2009, Volume 29 number 4

OMWESO

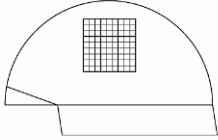

Sampolo Ezirimu Obutoffaali Obutono:

Bala omugatte gw'obutoffaali obw'ekika ekimu obuli mu biba ebitono **10** ebiri mu bitundu eby'enjawulo eby'ebiba by'obaze.

Omugatte gw'Obutoffaali	Obutoffaali / μL
1	1
2	2
3	2
4	3
5	4
6	5
7	5
8	6
9	7
10	8
11	8
12	9
13	10
14	11
15	11
16	12
17	13
18	14
19	15
20	15
21	16
22	17
23	18
24	18
25	19
26	20
27	21
28	21

Sampolo Ezirimu Obutoffaali Obungiko:

Bala omugatte gw'obutoffaali obw'ekika ekimu obuli mu biba ebitono **5** ebiri mu bitundu eby'enjawulo eby'ebiba by'obaze.

Omugatte gw'Obutoffaali	Obutoffaali / μL
5	8
6	9
7	11
8	12
9	14
10	15
11	17
12	18
13	20
14	21
15	23
16	24
17	26
18	28
19	29
20	31
21	32
22	34
23	35
24	37
25	38
30	46
35	54
40	61
45	69
50	77
60	92
70	107

WEETEGEREZE: Mu sampolo ezitaweza mL12, kendeza ku bungi bwa sampolo z'okukamula zibeere mL6 era okubiseemu ebivuddemu nga tonnaba kukozeza mweso (waggulu).

Ekika ky'Akatoffaali	Eky bulijjo	Akakyuseemu	Kya kukebera*
Obutoffaali bw'Omusanyi Obulwany	0-4/ μL	4-6/ μL	> 6/ μL
Obutoffaali bw'Omusanyi Obumyufu	0-2/ μL	2-3/ μL	> 3/ μL

Engeri y'Okubala Endala: Laga omuwendo gw'omukkati gw'obutoffaali mu buli kiba ekitono n'oluvannyuma okozese ennamba eno gy'okubisaamu okubala obutoffaali obuli mu buli μL .

Okubala obutoffaali / μL nga tukozeza KOVA Plastics Glasstic Slide 10 n'Ekiba:

- Mu sampolo ezitakamuddwa oba ezitajabuluddwa, omukkati gw'obutoffaali bw'ofunye mu buli kiba ekitono bukubiseemu x **90**.
- Mu sampolo za mL10 ezikendezeddwa okutuuka ku mL1, omukkati gw'obutoffaali bw'ofunye mu buli kiba ekitono bukubiseemu x **9**.
- Mu sampolo za mL10 ezikendezeddwa okutuuka ku mL0.5, omukkati gw'obutoffaali bw'ofunye mu buli kiba ekitono bukubiseemu x **4.5**.
- Mu sampolo za mL12 ezikendezeddwa okutuuka ku mL1 (Nkola ya KOVA), omukkati gw'obutoffaali bw'ofunye mu buli kiba ekitono bukubiseemu x **7.5**.

Eky'okulabirako ky'okubala (ng'Okozesa Enkola ya KOVA mL12 ku mL1):

Obutoffaali	Ebiba Ebialiddwa	Omugatte gw'Obutoffaali	Omukkati gw'Obutoffaali / Ebiba	Nnamba x Gy'okubisaamu (7.5)	Obutoffaali buli μL ya Sampolo
Obutoffaali bw'Omusanyi Obulwany	10	5	0.5	0.5 x 7.5	3.8
Obutoffaali bw'Omusanyi Obumyufu	10	14	1.4	1.4 x 7.5	10.5

* Ekijuliziddwa: Aiken, C. D. and Sokeland, J. (1983). Urologie. Thiems, Stuttgart, Ninth Edition, lup.79

OMWESO

SAMPOLO Z'OMUSULO OBA EBIKEBERWA EBIGGYIBWA KU MUNTU EZITAJEEBUDDWA, EZITAKAMUDDWA

SAMPOLO EZIRIMU OBUTOFFAALI OBUTONO

Bala omugatte gw'obutoffaali obw'ekika ekimu obuli mu biba ebitono **36** oba ebitundu 4 ebijuvu eby'ebiba by'obaze.

Omugatte gw'Obutoffaali	Obutoffaali/μL	Obutoffaali/mL
1	3	2,500
2	5	5,000
3	8	7,500
4	10	10,000
5	13	12,500
6	15	15,000
7	18	17,500
8	20	20,000
9	23	22,500
10	25	25,000
11	28	27,500
12	30	30,000
13	33	32,500
14	35	35,000
15	38	37,500
16	40	40,000
17	43	42,500
18	45	45,000
19	48	47,500
20	50	50,000
25	63	62,500
30	75	75,000
40	100	100,000
50	126	125,500

SAMPOLO EZIRIMU OBUTOFFAALI OBUNGI

Bala omugatte gw'obutoffaali obw'ekika ekimu obuli mu biba ebitono **10** mu bitundu eby'enjawulo eby'ebiba by'obaze.

Omugatte gw'Obutoffaali	Obutoffaali/μL	Obutoffaali/mL
1	9	9,000
2	18	18,000
3	27	27,000
4	36	36,000
5	45	45,000
6	54	54,000
7	63	63,000
8	72	72,000
9	81	81,000
10	90	90,000
20	180	180,000
25	225	225,000
30	270	270,000
35	315	315,000
40	360	360,000
50	450	450,000
60	540	540,000
70	630	630,000
80	720	720,000
90	810	810,000
100	900	900,000
150	1350	1,350,000
200	1800	1,800,000
250	2250	2,250,000

Engeri y'Okubala Endala:

Kubisaamu 90 mu mukatti gw'obutoffaali ogwa buli kiba ekitono okufuna obutoffaali bwa buli μL; kubisaamu 90,000 okufuna obutoffaali bwa buli mL.

Engeri y'Okubala Endala:

Kubisaamu 90 mu mukatti gw'obutoffaali ogwa buli kiba ekitono okufuna obutoffaali bwa buli μL; kubisaamu 90,000 okufuna obutoffaali bwa buli mL.

ENGERI EY'OKUBALA AMAZZI G'OMUBIRI AGAJEEBUDDWA:

Obutoffaali / μL = Omukatti gw'omuwendo gw'obutoffaali mu buli kiba ekitono x 90 (nnamba gy'okubisaamu) x okujaabulula okugeza, Amazzi g'oku mugongo agajeebuddwa 1:10; omugatte gwa RBC 50 ezibaliddwa mu biba ebitono 10

$$\text{RBC}/\mu\text{L} = \frac{\text{Obutoffaali } 50}{\text{ebiba } 10} \times 90 (\text{gy'okubisaamu}) \times 10 (\text{enjaabulula})$$

$$= 5 \times 900 = 4,500 \text{ RBC's}/\mu\text{L}$$

okugeza, Amazzi g'ekisajja agajeebuddwa 1:20; omugatte gwa nkwaso 150 nga zibaliddwa mu biba ebitono 5

$$\text{Enkwaso}/\mu\text{L} = \frac{150}{5} \times 90 (\text{gy'okubisaamu}) \times 20 (\text{enjaabulula})$$

$$= 30 \times 1800 = \text{enkwaso } 54,000/\mu\text{L}$$

OMUGATTE GW'OMUWENDO GW'OBUTAFFAALI OGWA BULIJO GUBEERA ⁽¹⁾

AMAZZI	EKIKA KY'AKATOFFAALI	EKYA BULIJO	EKITALI KYA BULIJO	AMAZZI	EKIKA KY'AKATOFFAALI	EKYA BULIJO	EKITALI KYA BULIJO
Omusulo (2)	Obutoffaali bw'Omusaaayi Obulwanyi. Obutoffaali bw'Omusaaayi Obumyufu	0-6/μL 0-3/μL	> 6/μL > 3/μL	Ennyingo	Obutoffaali bw'Omusaaayi Obulwanyi. Obutoffaali bw'Omusaaayi Obumyufu	< 200/μL < 2,000/μL	> 200/μL > 2,000/μL
CSF (Mu Muntu Akuze)	Obutoffaali bw'Omusaaayi Obulwanyi	0-5/μL	> 5/μL	Akasusu K'Ekifuba	Obutoffaali bw'Omusaaayi Obulwanyi	< 1,000/μL	> 1,000/μL
				Akasusu k'oku Mutima	Obutoffaali bw'Omusaaayi Obulwanyi	< 1,000/μL	> 1,000/μL
Amazzi g'ekisajja	Enkwaso	40,000/μL - 160,000/μL	< 40,000/μL	Akasusu k'oku Lubuto	Obutoffaali bw'Omusaaayi Obulwanyi. Obutoffaali bw'Omusaaayi Obumyufu	< 300/μL < 100,000/μL	> 300/μL > 100,000/μL

Ebiwandiiiko ebijuliziddwa: (1) Strasinger, S.K. (1985) **Urinalysis and Body Fluids**, F.A. Davis, Philadelphia • (2) Alken, C.D., and Sokeland, J. (1983) **Urologie**, Thiems, Stuttgart, Nineth Edition, pg. 79

Akabonero	Lungereza
	Koodi y'Ekitereke/Ebintu
	Okuggwaako/Kozesa Obutasukka
	Omukozi
	Ennamba y'Olukalala lw'Ebintu
	Mulimu Omuwendo
	Toddamu Kukikozesa
	Akalamba Ak'enjawulo
	Bya Kukozesa Kukebera Biggyiddwa ku Biramu
 www.kovaplastics.com	Ebiragiro by'okukozesa/ Ebiragiro by'Okukozesa eby'oku Kompyuta
	Kikoleddwa mu Ggwanga (Amerika)
	Ekkomo ku kutereka

	Alltrista Plastics LLC 20 Setar Way Reedsville, Pa 17084 United States Customer Service: +1 864-879-8100		Advena Ltd. Tower Business Centre, 2 nd Flr. Tower Street, Swatar, BKR 4013 Malta
	EU Economic Operator MDR/IVDR Article 13 Advena Services Ltd. Tower Business Centre, Tower Street Swatar, BKR 4013 Malta		Axon Lab Ag Täfernstrasse 15 CH-5405 Baden-Dättwil Switzerland

CE