


Barcode Plus™ | Label

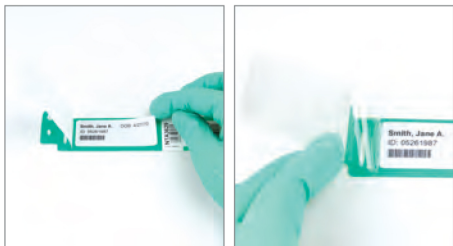


1. **OBTAIN** the pre-printed label per your protocol.
VERIFY the patient's identity.
PLACE a patient label onto the "PLACE PATIENT INFORMATION HERE" area of the band.
PEEL the liner from the shield.
COVER the patient label with the shield.
SEAL the edges down on a flat surface using your fingers.



2. **WRAP** the band tail around the patient's extremity to size.
SNAP the button closure to secure.
CUT the excess band tail material with scissors.



3. **DRAW** the pre-transfusion specimen per your protocol.
PEEL the specimen tube sticker from the band.
PLACE sticker on the specimen tube lengthwise.

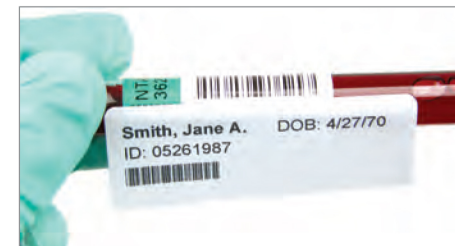

4. **APPLY** a pre-printed patient label onto this sticker beneath the dashed line.
SIGN/DATE as required by your protocol.




5. **LABEL** any forms at the patient's bedside as required by your protocol.
REMOVE the tail of stickers from the band at the perforation.



6. **ATTACH** the tail of stickers to the specimen tube, if desired.
TRANSPORT the specimen to the blood bank.



7. **SCAN** the Blood Bank ID (BBID) into the Laboratory Information System (LIS), or enter it manually.

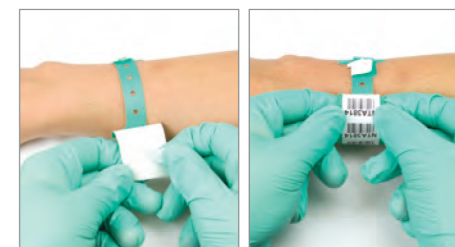

8. **PEEL** the adhesive tab on underside of the extra stickers.
SECURE the extra stickers to the band.
REMOVE the extra stickers with scissors if not required by your protocol.

Visit www.typenex.com or call us at 866.897.3639 to learn more.