

R3™ Band | Reattachment System

Easily relocate or resize any Typenex® Medical Blood Band using the R3™ Band | Reattachment System.



1. **CUT** the band to the left of the far left slot, cutting off the snap closure of the barcode band.

CUT the band to the right of the far right slot, behind the barcode stickers. Be sure to leave any large barcode stickers intact, if applicable.



2. **INSERT** the R3 Band through the slot on the left, making sure the closure of the R3 Band faces you.

PULL the R3 Band through the slot on the right, making sure to not cover the patient information.



3. **SIZE** the R3 Band for patient comfort.



4. **SNAP** the R3 Band closed and trim the excess band material.



The reattachment or resized barcode band now has the patient information and the barcoded Blood Bank ID (BBID) accessible.

Visit www.typenexmedical.com or call us at 866.897.3639 to learn more.