


Absorbent Floor Mat | Double-Sided

The Double-Sided version of Typenex Medical's Absorbent Floor Mats is designed to soak up fluid on the operating room floor during and after cases. The modular design allows the healthcare practitioner to customize the mat into various shapes and sizes to meet specific usage needs.



1. **REMOVE** bags containing mats from carton.


PLACE the bag of mats into a storage location after removing from carton.

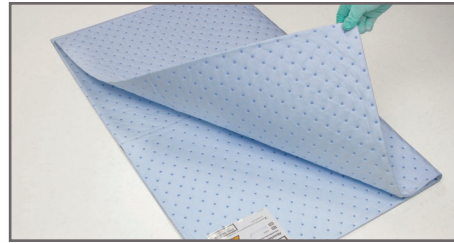

2. **REMOVE** mat from bag.
UNFOLD mat to prepare for use.




3. **CUSTOMIZE** mat size and shape, if desired, by holding mat firmly and tearing along perforations.

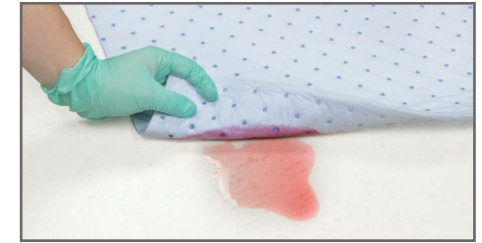

4. **PLACE** mat in desired area with label face up prior to procedure.

OPTION: Use mats to absorb existing fluids once they occur.

⚠ USE CAUTION when standing or walking on mats.

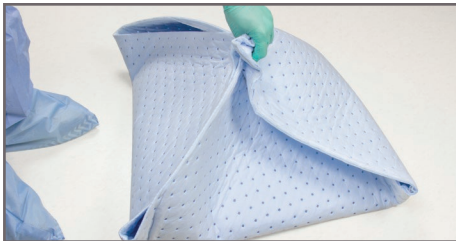

5. **HANDLE** soiled mat carefully by following facility personal protective equipment (PPE) protocol.



6. **DISPOSE** of mat by refolding and placing in an appropriate waste receptacle per your facility's protocol.

NOTE: It may be best to move appropriate waste receptacle near soiled mat during disposal.