
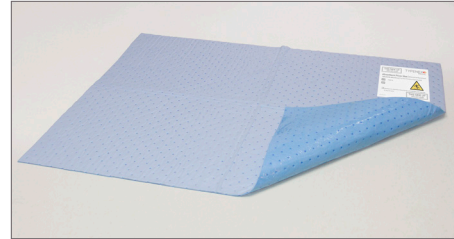


Absorbent Floor Mat | Waterproof Backsheet

The Waterproof Backsheet version of Typenex® Medical's Absorbent Floor Mats is designed to be placed on a dry floor before the case in order to keep the floor beneath the mat dry and aid in the clean up process. This mat features a non-slip waterproof backsheet that grips to a **clean, dry** operating room floor.

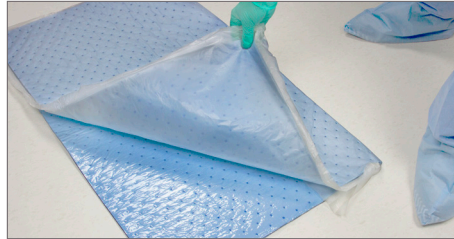

- 1. REMOVE** bags containing mats from carton.

PLACE the bags of mats into a storage location after removing from carton.

- 2. REMOVE** mat from bag.
UNFOLD mat to prepare for use.

- 3. PLACE** mat in desired area with label face up prior to procedure.
⚠ CAUTION: Use caution when standing or walking on mats. Mats must lay completely flat and be placed on a dry, clean floor.

- 4. HANDLE** soiled mat carefully by following facility personal protective equipment (PPE) protocol.

DISPOSE of mat by refolding and placing in an appropriate waste receptacle per facility protocol.

NOTE: It may be best to move appropriate waste receptacle near soiled mat during disposal.