

BASS[®] Suction Mat

The BASS[®] Suction Mat is designed to help keep contaminated fluid from hitting the operating room floor. It covers an extensive surface area and features continuous suction capability that effectively captures and removes fluid. These features make the mat an ideal solution for **high volume cases**.

PRE-SURGERY



1. **UNFOLD** the BASS mat and place it on the OR floor with the fiber side up.



2. **POSITION** the BASS mat with the suction port facing closest to the suction source.



3. **CONNECT** the BASS mat pre-assembled tubing to the suction source.

ACTIVATE suction.

POST-SURGERY



4. **FOLD** the BASS mat in half with tubing connected and suction power ON.

FOLD in half a second time.



5. **ALLOW** for residual fluid in the BASS mat to be vacuumed into the suction source.



6. **DISPOSE** of the BASS mat per facility protocol once fluid is removed.

Visit www.typenexmedical.com or call us at 866.897.3639 to learn more.