

Big Foot® Quiet | Suction Disc

The Big Foot Quiet Suction Disc by Typenex® Medical is designed to quietly vacuum fluid off the floor during low, moderate or high volume surgical cases. The design allows for easy repositioning to effectively manage and remove fluid on the operating room floor.



1. **PLACE** Big Foot Quiet flat on the floor with the “This Side Up” label facing upward.



2. **CONNECT** suction tubing from Big Foot Quiet to a fluid management system and turn on vacuum source.



3. **POSITION** Big Foot Quiet either by foot or hand as needed throughout the procedure.

**For best use, allow the white pad on the bottom of the disc to be saturated in fluid before suctioning.*



4. **DISPOSE** of the soiled Big Foot Quiet carefully by following facility personal protective equipment (PPE) protocol and placing in an appropriate waste management receptacle.