

Big Foot® | Suction Disc

The Big Foot® Suction Disc by Typenex Medical is designed to continuously vacuum fluid off the floor during low, moderate, or high volume surgical cases. The design allows for easy repositioning to effectively manage and remove fluid on the operating room floor.



1. **PLACE** Big Foot flat on the floor with the “This Side Up” label facing upward.



2. **CONNECT** suction tubing from Big Foot to a fluid management system and turn on vacuum source.



3. **POSITION** Big Foot either by foot or hand as needed throughout the procedure.



4. **DISPOSE** of the soiled Big Foot carefully by following facility personal protective equipment (PPE) protocol and placing in an appropriate waste receptacle.