

Slim 3™


The Slim 3™ Blood Band keeps patient safety at the forefront of every blood transfusion, while increasing efficiency with easy-to-use features.



- 1. VERIFY** the patient's identity.
WRITE the patient's information over the prompts on the band using a ballpoint pen.
NOTE: Press firmly while writing, as a copy of the handwritten information will remain on the band and darken over time.



- 2. PEEL** the handwritten patient information label from the band.
PLACE the handwritten patient information label on the specimen tube lengthwise.

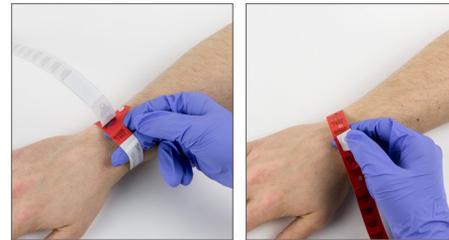

- 3. WRAP** the band around the patient's extremity to size.
SNAP the button closure to secure.
DRAW the pre-transfusion specimen per your protocol.



- 4. LABEL** any forms at the patient's bedside as required by your protocol.
REMOVE the tail of the detachable BBID stickers at the perforation.



- 5. PEEL** the adhesive tab on the underside of the extra stickers.
SECURE the extra stickers to the specimen tube, if desired.



- 6. TRANSPORT** the specimen to the blood bank.
ENTER the BBID into the Laboratory Information System (LIS).
NOTE: The BBID will always start with 4 alpha characters, and be followed by 4 numbers.

Visit www.typenexmedical.com or call us at 866.897.3639 to learn more.