

# Fluid Fighter™ | Suction Disc

1. Place the Fluid Fighter™ Suction Disc on the floor with the flat side down.
2. Connect suction tubing from the Fluid Fighter Suction Disc to a suction source. Power on vacuum source.  
  
Optional: If needed, attach the plastic elbow to the tubing first and then attach the elbow to the Fluid Fighter Suction Disc.
3. As needed, position the Fluid Fighter Suction Disc either by foot or hand throughout the procedure.
4. Dispose of the soiled Fluid Fighter Suction Disc per facility protocol and place it in an appropriate waste receptacle.