

# Absorbent Floor Mat Roll | Waterproof Backsheet

The Waterproof Backsheet Mat Roll by Typenex® Medical features a non-slip sheet that is designed to hold a tight grip on your operating room floor. By placing the mat on the surface before a case, the floor beneath remains dry and clean.



1. **REMOVE** the floor mat roll from the bag.





2. **HOLD** the mat firmly and tear along the perforations to fit the operating room floor.



3. **STORE** the rest of the roll in your facility's designated area.

**NOTE:** Utilize our mobile roll cart for easy transportation and storage.

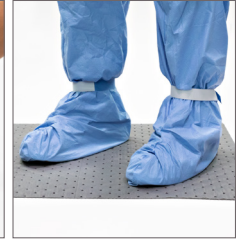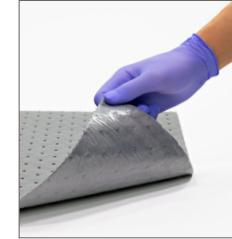

4. **PLACE** the mat in the desired area prior to the procedure. The mat must lay completely flat on a dry, clean surface.

**NOTE:** Use caution when standing or walking on mats.



5. **DISPOSE** of the soiled mat by refolding it and placing it in an appropriate waste receptacle per your facility's Personal Protective Equipment (PPE) protocol.

**NOTE:** It may be best to move the waste receptacle closer to the mat that requires disposal.