

# Typesafe™ | Segment Piercing Device



1. Insert the narrow end of the Typesafe device into the test tube. Be sure the device sits directly on top of the test tube during use, so the narrow end is facing straight down.

- Wear gloves when using Typesafe



2. While holding the test tube and Typesafe device steady with one hand, insert the blood segment into Typesafe with the other hand, targeting the center of the device. Gently press until the needle has pierced the segment. Apply pressure to the segment, forcing blood into the test tube.

- Do not squeeze until the needle has pierced the segment



3. When the desired sample has been dispensed into the test tube, remove the blood segment and the Typesafe device. Properly dispose of both items in a designated hazardous medical waste container.

- Single-use only
- For in-vitro diagnostic purposes