

Absorbent Floor Mat | Waterproof Backsheet

The Waterproof Backsheet version of Typenex Medical's Absorbent Floor Mats is designed to be placed on a dry floor before the case in order to keep the floor beneath the mat dry and aid in the clean up process. This mat features a non-slip waterproof backsheet that grips to a dry operating room floor.




- 1. REMOVE** bags containing mats from carton.

PLACE the bags of mats into a storage location after removing from carton.



- 2. REMOVE** mat from bag.
UNFOLD mat to prepare for use.

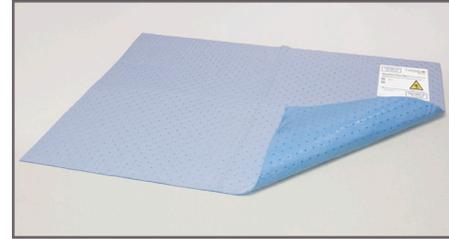

- 3. PLACE** mat in desired area with label face up prior to procedure.

⚠ CAUTION: Use caution when standing or walking on mats.



- 4. HANDLE** soiled mat carefully by following facility personal protective equipment (PPE) protocol.

DISPOSE of mat by refolding and placing in an appropriate waste receptacle per facility protocol.

NOTE: It may be best to move appropriate waste receptacle near soiled mat during disposal.