


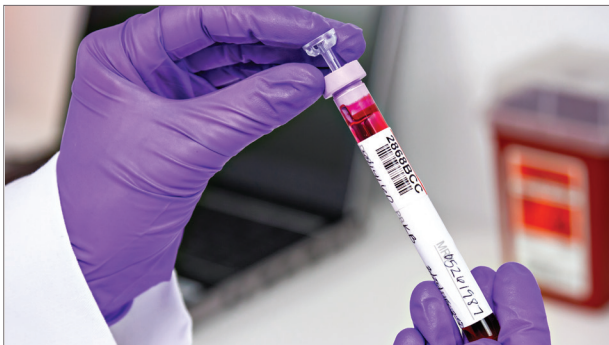

# The seamless way to sample.

Smearsafe™ offers a simple, seamless way to dispense a drop of blood from a tube. In a matter of seconds, a lab tech can insert this single-use device, collect the required blood, and dispose of any unwanted components — without ever removing the lid of the tube.

# Smearsafe™ | Blood Dispenser



## Features and Benefits

**Easy to Use** — By varying the pressure and speed at which the tube is pressed into the slide, the tech can easily control the size of the blood drop. Because the part of Smearsafe that comes in contact with the slide is clear, it is easy to monitor how much blood is being transferred every time.

**Efficient Design** — With fewer required steps and a four-prong design, Smearsafe offers a more efficient and safe way to obtain a drop of blood than capillary tubes or applicator sticks.

**Safe Transfer** — Smearsafe allows the tech to obtain a drop of blood without removing the lid of the tube. This eliminates the risk of cross contamination and prevents any mess in the lab.

**Convenient Compatibility** — Works with Vacutainer® and Vacuette® tubes.

## Ordering Information

Part Number	Color	Units/Case
C1000	Clear	1000

Visit [www.typenexmedical.com](http://www.typenexmedical.com) or call 866.897.3639 to learn more.