

# Fluid Fighter® | Suction Mat

The Fluid Fighter® Suction Mat by Typenex® Medical is a unique floor fluid management solution that absorbs fluid from the surface while suction is drawn through an internal system of channels. Made with quality materials, this low-profile mat can help to ensure an efficient turnover rate between procedures.




1. **UNROLL** the mat and place it on the operating room floor with the labeled side facing up.



2. **POSITION** the mat so that the suction port is close to the suction source.



3. **PEEL** off the adhesive liners on two corners and press down on the mat to secure it to the floor. Pull the mat taut, then adhere the other two corners to the floor.

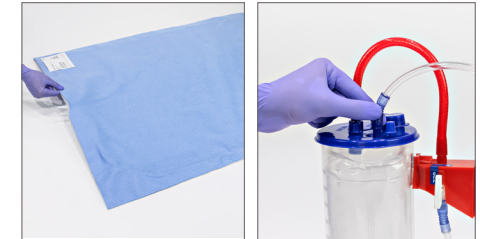

4. **CONNECT** the pre-assembled tubing to the suction source and power on the vacuum.



1. **FOLD** the mat in half (inward) with the tubing still connected and the suction power on. Fold it in half a second time.



2. **ALLOW** any residual fluid in the mat to be vacuumed into the suction source.



3. **DISPOSE** of the mat per your facility's protocol once the fluid has been removed.