

Absorbent Floor Mat | Waterproof Backsheet

The Waterproof Backsheet Absorbent Floor Mat by Typenex® Medical features a non-slip sheet that is designed to hold a tight grip on your operating room floor. By placing the mat on the surface before a case, the floor beneath remains clean and dry to aid in the clean-up process.



1. **REMOVE** the packaging containing the mats from the carton.

STORE any remaining mats in your facility's designated area after removing them from the carton.



2. **REMOVE** the mat from its packaging.

UNFOLD the mat to prepare for use.



3. **PLACE** the mat in the desired area with the label facing up, prior to the procedure. The mat must lay completely flat on a dry, clean surface.

USE CAUTION when standing or walking on mats.



4. **REMOVE** the soiled mat from the floor by pulling from a corner and grabbing the underside of the mat to release the waterproof backsheet. Handle carefully per your facility's personal protective equipment (PPE) protocol.

DISPOSE of the soiled mat by refolding it and placing it in an appropriate waste receptacle per your facility's protocol.

NOTE: It may be best to move the appropriate waste receptacle near the soiled mat during disposal.