

Barcode Plus™ | Handwritten/Label Combination

The Barcode Plus™ | Handwritten Blood Band can be used with a pre-printed patient label. This solution allows you to save time and maximize efficiency, but also gives your facility the ability to handwritten information on the blood band as a backup method.



1. **OBTAIN** pre-printed patient labels per your protocol.

VERIFY the patient's identity.



2. **PLACE** a patient label onto the area where there are prompts for handwriting. **DO NOT REMOVE** this handwriting label before applying the pre-printed label.

PEEL the liner from the shield and cover the patient label with the shield.

SEAL the edges down on a flat surface using your fingers.



3. **WRAP** the band tail around patient extremity to size.

SNAP the button closure to secure.

CUT the excess band tail material with scissors.



4. **DRAW** the pre-transfusion specimen per your protocol.

PEEL a specimen tube sticker from the band's tail and place it on the specimen tube lengthwise.


APPLY a pre-printed patient label onto the specimen tube.

SIGN/DATE as required by your protocol.



5. **LABEL** any forms at the patient's bedside as required by your protocol.

REMOVE the tail of stickers from the band at the perforation.

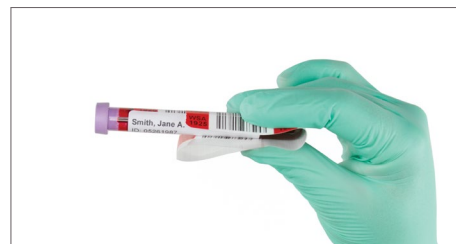

6. **ATTACH** the tail of stickers to the specimen tube, if desired.



7. **PEEL** the adhesive tab on underside of extra stickers and secure to the band.

REMOVE the extra stickers with scissors if not required per your protocol.



8. **TRANSPORT** to the blood bank.

SCAN the Blood Bank ID (BBID) into the Laboratory Information System (LIS) or enter it manually.

Visit www.typenexmedical.com or call us at 866.897.3639 to learn more.