

# Absorbent Floor Mat | Double-Sided

The Double-Sided Absorbent Floor Mat by Typenex® Medical offers a tailored solution to soak up fluid on the operating room floor during and after cases. The modular design allows for easy configuration into various shapes and sizes for ideal coverage and specific needs.



1. **REMOVE** the packaging containing the mats from the carton.


**STORE** any remaining mats in your facility's designated area after removing them from the carton.



2. **REMOVE** mat from its packaging.  
**UNFOLD** the mat to prepare for use.



3. **CUSTOMIZE** the mat's size and shape, if desired, by holding the mat firmly and tearing along the perforations.

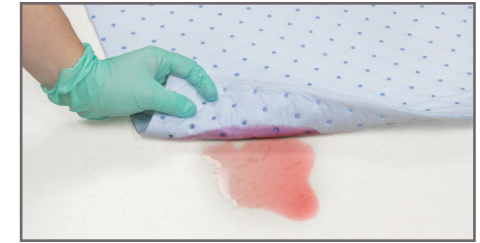

4. **PLACE** the mat in the desired area with the label facing up, prior to the procedure.

**OPTIONAL:** Use mats to absorb fluids as they occur.

**⚠ USE CAUTION** when standing or walking on mats.



5. **REMOVE** the soiled mat from the floor by pulling from a corner and grabbing the underside of the mat. Handle carefully per your facility's personal protective equipment (PPE) protocol.



6. **DISPOSE** of the soiled mat by refolding it and placing it in an appropriate waste receptacle per your facility's protocol.

**NOTE:** It may be best to move the appropriate waste receptacle near the soiled mat during disposal.