
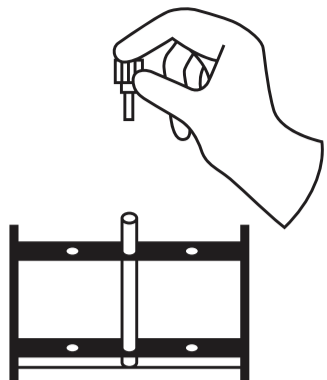
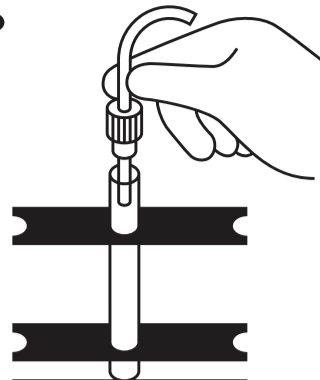
1.

2.

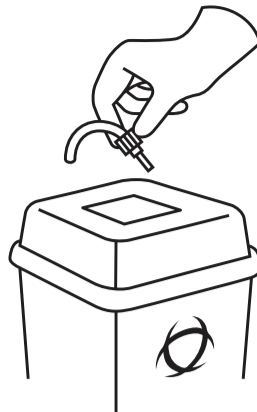3.

Instructions

English

Intended Use:

Typesafe® is indicated for the piercing of a blood segment and transfer of that blood into a test tube.

- Intended for use with 10mm and 12mm tubes.
- Typesafe is intended for use by laboratory professionals.

Upon removal from package, inspect device to ensure no damage has occurred during shipping. Insert the narrow end of the Typesafe® segment sampling device into the test tube.

- Make sure the tube is properly aligned in an appropriately sized test tube rack
- Always wear proper safety attire when working with blood

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- Do not squeeze until needle has pierced segment
- Be sure to place the Typesafe® directly on top of test tube, so the narrow end is facing straight down, reducing angles and gaps

When desired sample has been dispensed into the test tube, remove the blood segment and the Typesafe® device. Properly dispose of both in designated hazardous medical waste receptacles.

- For single-use only
- For in-vitro diagnostic purposes only
- Reuse may cause contamination

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English GB

Intended use:

Typesafe® is intended for the piercing of a blood segment and transfer of that blood into a test tube.

- Intended for use with 10-mm and 12-mm tubes.
- Typesafe is intended for use by laboratory professionals.

Upon removal from the package, inspect the device to ensure that no damage has occurred during shipping. Insert the narrow end of the Typesafe® segment sampling device into the test tube.

- Make sure that the tube is properly aligned in an appropriately sized test tube rack
- Always wear proper safety attire when working with blood

Insert blood segment into Typesafe® and gently press until the needle has pierced the segment. Apply pressure to the segment, forcing blood into the test tube.

- Do not squeeze until needle has pierced segment
- Make sure that you place the Typesafe® directly on top of the test tube, so the narrow end is facing straight down, reducing angles and gaps.

When the desired sample has been dispensed into the test tube, remove the blood segment and the Typesafe® device. Properly dispose of both in designated hazardous medical waste receptacles.

- For single-use only
- For in vitro diagnostic purposes only
- Reuse may cause contamination

CAUTION:

Any serious incident that has occurred in relation to the device shall be reported to Typenex Medical.

Instructions

English IE

Intended use:

Typesafe® is intended for the piercing of a blood segment and transfer of that blood into a test tube.

- Intended for use with 10-mm and 12-mm tubes.
- Typesafe is intended for use by laboratory professionals.

Upon removal from the package, inspect the device to ensure that no damage has occurred during shipping. Insert the narrow end of the Typesafe® segment sampling device into the test tube.

- Do not squeeze until needle has pierced segment
- Make sure that you place the Typesafe® directly on top of the test tube, so the narrow end is facing straight down, reducing angles and gaps.

When the desired sample has been dispensed into the test tube, remove the blood segment and the Typesafe® device. Properly dispose of both in designated hazardous medical waste receptacles.

- For single-use only
- For in vitro diagnostic purposes only
- Reuse may cause contamination

CAUTION:

Any serious incident that has occurred in relation to the device shall be reported to Typenex Medical.

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Typenex Medical, LLC