


Extubation Bib

Instructions for Use




1. Unfold the bib to prepare for patient extubation.

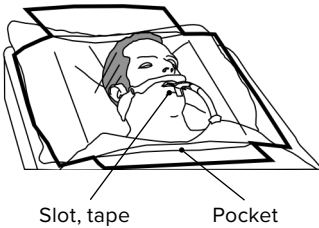

2. Disconnect the airway circuit and place the bib over the patient's face, with the airway tube going through the slot.



3. Remove the adhesive liner and attach the bib's tab to the airway tube. Reattach the circuit and extubate the patient per your facility's protocol.

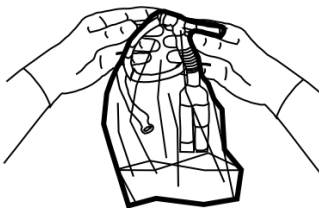

4. Wrap the airway and circuit components in the bib to contain for disposal.

WARNING: The Extubation Bib should only be used immediately prior to performing tracheal extubation. Never leave the patient unattended with the Extubation Bib in place. Never cover or block the patient's airway. In order to prevent occlusion of the airway, insert the airway tube through the Extubation Bib's center slot as soon as the Extubation Bib has been placed on the patient. Place the Extubation Bib loosely over the patient's face; do not secure or tuck the edges.



Single use
only



Not made
with natural
rubber latex