

# Absorbent Floor Mat Roll | Waterproof Backsheet

The Waterproof Backsheet Absorbent Floor Mat Roll by Typenex® Medical features a non-slip sheet that is designed to hold a tight grip on your operating room floor. By placing the mat on the surface before a case, the floor beneath remains clean and dry to aid in the clean-up process.



1. **REMOVE** the floor mat roll from its packaging.




2. **HOLD** the mat firmly and tear along the perforations to customize the shape and size to fit your needs.




3. **STORE** the rest of the roll in your facility's designated area.

**NOTE:** Utilize our mobile roll cart for easy transportation and storage.

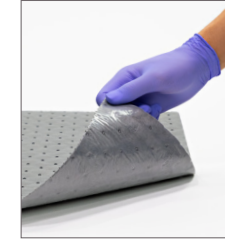

4. **PLACE** the mat in the desired area prior to the procedure. The mat must lay completely flat on a dry, clean surface.

**⚠ USE CAUTION** when standing or walking on mats.

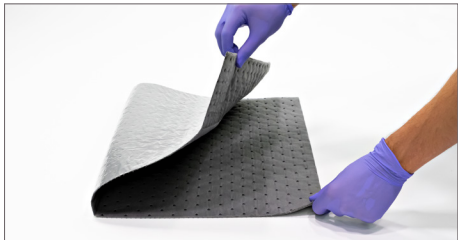

5. **REMOVE** the soiled mat from the floor by pulling from a corner and grabbing the underside of the mat to release the waterproof backsheet. Handle carefully per your facility's personal protective equipment (PPE) protocol.



6. **DISPOSE** of the soiled mat by refolding it and placing it in an appropriate waste receptacle per your facility's protocol.

**NOTE:** It may be best to move the appropriate waste receptacle near the soiled mat during disposal.