


Absorbent Floor Mat Roll | Double-Sided

The Double-Sided Absorbent Floor Mat Roll by Typenex® Medical offers a tailored solution to soak up fluid on your operating room floor during and/or after a case. The perforated, modular design allows you to configure the mats, so you can cover as much surface as needed.



1. **REMOVE** the floor mat roll from its packaging.

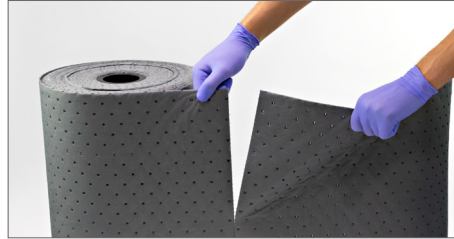

2. **HOLD** the mat firmly and tear along the perforations to customize the shape and size to fit your needs.



3. **STORE** the rest of the roll in your facility's designated area.

NOTE: Utilize our mobile roll cart for easy transportation and storage.



4. **PLACE** the mat in the desired area prior to the procedure. You may also place the mat on existing fluid during and/or after the procedure.

⚠ USE CAUTION when standing or walking on mats.



5. **REMOVE** the soiled mat from the floor by pulling from a corner and grabbing the underside of the mat. Handle carefully per your facility's personal protective equipment (PPE) protocol.



6. **DISPOSE** of the soiled mat by refolding it and placing it in an appropriate waste receptacle per your facility's protocol.

NOTE: It may be best to move the appropriate waste receptacle near the soiled mat during disposal.