

# Fluid Fighter™ | Suction Mat

## **Pre-Surgery**

1. Unfold the Fluid Fighter™ Suction Mat and place it on the operating room floor with the label side facing up.
2. Position the Fluid Fighter Suction Mat with the suction port facing closest to the suction source.
3. Remove adhesive strip covers and secure to the floor.
4. Connect the Fluid Fighter Suction Mat tubing to the suction source and the mat's external connection.
5. Power on vacuum source.

## **Post-Surgery**

1. Fold the Fluid Fighter Suction Mat in half with tubing connected and suction power on. Fold in half a second time.
2. Allow for residual fluid in the Fluid Fighter Suction Mat to be vacuumed into the suction source.
3. Dispose of the soiled Fluid Fighter Suction Mat per facility protocol and place it in an appropriate waste receptacle.