



The clear choice in bulb syringes.

Visibulb™ | 2 oz allows the user to see the contents suctioned from ear, oral, and nasal passageways. The clear design shows the cleanliness of the device before and after use.

Visibulb™ | 2 oz



Features and Benefits

Clear Design - Visibulb™ | 2 oz allows the user to see the contents they are suctioning from the orifice in which the bulb syringe is placed.

Effective - Similar to traditional bulb syringes, Visibulb™ | 2 oz allows the removal of fluids with a simple squeeze, but with the added benefit of clarity.

Clean - The clarity of Visibulb™ | 2 oz provides a visual cue of cleanliness before and after use.

Instructions

1. Squeeze the bulb of the syringe and then place it gently inside the patient's orifice.
2. When the pressure has been released completely from the bulb, remove it from the patient's orifice.
You should see fluid being pulled into the bulb of the syringe.
3. Repeat the prior steps until the patient's orifice is clear of fluids.

NOTE: After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

- Sterile unless the package is opened or damaged.
- Do not reuse.

Ordering Information

Order Number	Description	Unit/Case	Color
CEB02	Ear/Ulcer Syringe	50	Clear

For more information, visit www.typenexmedical.com.