

# Visibulb™ | 2 oz

## Ear/Ulcer Syringe

### Instructions

1. Squeeze the bulb of the syringe, and then place it gently inside the patient's orifice.
2. When the pressure has been released completely from the bulb, remove it from the patient's orifice.
3. Repeat the prior steps until the patient's orifice is clear of fluids.

You should see fluids being pulled into the bulb of the syringe.

### Note:

- After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

**STERILE**

Sterile unless package is opened or damaged.



Do not use if package is opened or damaged



Single use