


Original™ | Blood Band

The first blood band designed specifically for transfusion patients, the Original™ Blood Band provides a unique Blood Bank ID (BBID) identifier in an easy to read alphanumeric format.



- 1. VERIFY** the patient's identity.
WRITE the patient's information over the prompts on the band using a ballpoint pen.
NOTE: Press firmly while writing, as a copy of the handwritten information will remain on the band and darken over time.

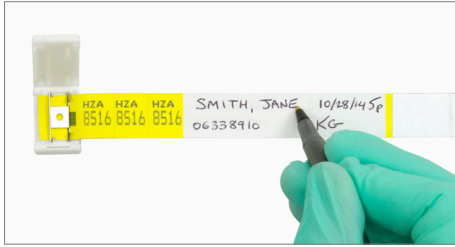

- 2. WRAP** the tail of detachable BBID stickers around the patient's extremity to size and align between the two plastic posts in the clip.
SECURE the band by closing the self-cutting clip and pressing firmly.




- 3. DRAW** the pre-transfusion specimen per your protocol.
PEEL the handwritten patient information label from band.



- 4. PLACE** the handwritten patient information label on the specimen tube lengthwise.



- 5. LABEL** any forms at the patient's bedside as required by your protocol.
REMOVE the tail of detachable BBID stickers by holding the self-cutting clip and tearing away.

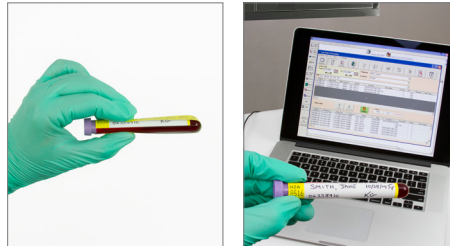

- 6. ATTACH** the tail of stickers to the specimen tube, if desired.
TRANSPORT the specimen to the blood bank.
ENTER the BBID into the Laboratory Information System (LIS).

Colors Available

- Red
- Green
- Yellow
- Blue
- Orange

Visit www.typenex.com or call us at 866.897.3639 to learn more.