


Fluid Fighter® Suction Liner Serial Connection

INSTRUCTIONS FOR USE

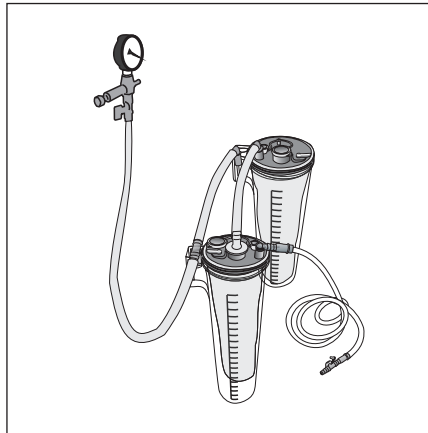

Multiple Fluid Fighter® Suction Canister and Liner installation using the Fluid Fighter® Suction Liner Serial Connection Tube.

INTENDED USE

If high volumes of fluid are expected during a surgical case, the Fluid Fighter® Suction Liner Serial Connection Tubes can be used to connect a series of Fluid Fighter® Suction Canisters and Liners together. Serial connection requires a minimum of 2 and a maximum of 4 Fluid Fighter Suction Canisters and Liners. To make a serial connection of Fluid Fighter Suction Canisters and Liners, please refer to the list below. To connect more Fluid Fighter Suction Liners in a series, the number of units required as mentioned below increases accordingly, except for the vacuum connector and silicone tubing that is connected to the suction source.

ITEMS REQUIRED FOR MINIMUM SERIAL CONNECTION

- 2 Fluid Fighter Suction Canisters
- 2 Fluid Fighter Suction Liners
- 1 Vacuum Connector
- 1 Fluid Fighter Suction Canister Vacuum T-Connector
- 1 Fluid Fighter Suction Liner Serial Connection Tube
- 1 piece of silicone tubing for connection to the suction source

ASSEMBLY

1. Insert the Fluid Fighter Suction Canister Vacuum T-Connector into the Fluid Fighter Suction Canister where fluid will first be suctioning (*as shown in the figure above*).
2. Attach a piece of silicone tubing from the suction source to the nearest inlet of the Fluid Fighter Vacuum T-Connector (*as shown in the figure above*).
3. The distance between the vacuum connectors on the Fluid Fighter Suction Canisters should be measured and additional silicone tubing must be cut 1.5 times the measured distance.
4. Attach the cut silicone tubing on the open side of the Fluid Fighter Suction Canister Vacuum T-Connector

on the first Fluid Fighter Suction Canister to the vacuum connector on the subsequent Fluid Fighter Suction Canister *(as shown in the figure above)*.

5. Attach the Fluid Fighter Suction Liner Serial Connection Tube's large orange connector to the large port on the first Fluid Fighter Suction Liner in the series by removing the attached cap. Then attach the Fluid Fighter Suction Liner Serial Connection Tube's green connector to the subsequent Fluid Fighter Suction Liner's patient port.
6. Connect the device's tubing that is being used to suction fluid to the patient port on the first Fluid Fighter Suction Liner in the series.

NOTE: Fluid Fighter Suction Liners can only be used with the same size Fluid Fighter Suction Canister.

Symbol Glossary available digitally at www.pergomed.com/symbolglossary.aspx.