

Convenience Bag®

Vomit Bag with Hand Protection


Instructions for Use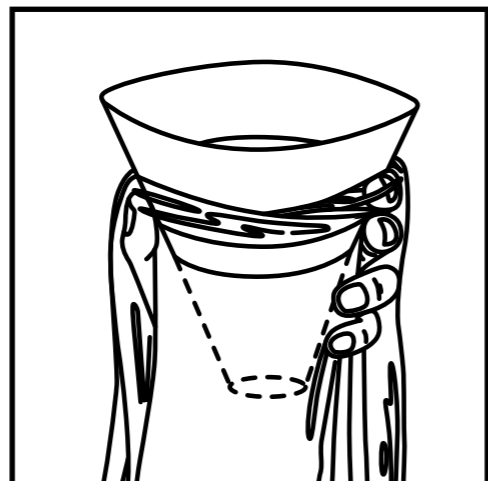

1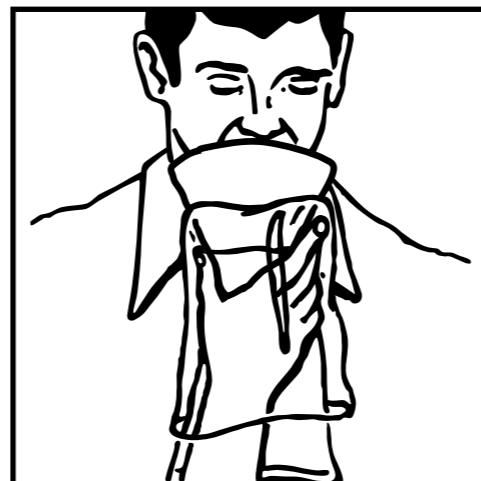

2

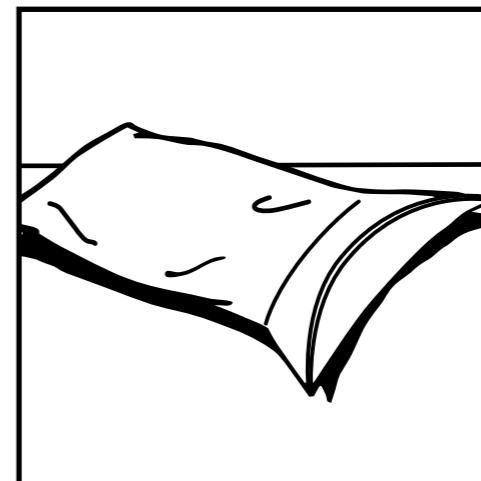3

Directions for Use: EN – English

Contents: 240 Convenience Bags with Hand Protection

Indications for Use: Intended to be used for safe vomit collection and disposal by children and adults.

1. Tear the pouch along the perforated line on the right-hand side and remove the Convenience Bag. Holding the Convenience Bag upright, place hand in hand protective sleeve.

2. Press the ends of the top to open the funnel and place over mouth to evacuate emesis into the open bag.

3. After use, close the Convenience Bag and fold in the top on the scored line for added leak protection. Dispose of bag according to facility protocol.

Warnings:

- Single-use product. Do not reuse. (Reuse may create risks of infection, cross-contamination, or other complications.)
- Supervise children using the Convenience Bag.

Note: Consult a healthcare professional if unsure about use or disposal.

Made in China.

Symbol	Meaning	Description of Symbol	ISO 15223-1
	Catalog number	Indicates the manufacturer's catalog number so that the medical device can be identified	5.1.6
	Batch code	Indicates the manufacturer's batch code so that the batch or lot can be identified	5.1.5
	Authorized representative in the European Community/European Union	Indicates the authorized representative in the European Community/European Union	5.1.2
	Indicates the authorized representative in Switzerland	Indicates the authorized representative in Switzerland	N/A
	Manufacturer	Indicates the medical device manufacturer	5.1.1
	Date of Manufacture	Indicates the date when the medical device was manufactured	5.1.3
	Do not reuse	Indicates a medical device that is intended for one single use only	5.4.2
	Consult instructions for use	Indicates the need for the user to consult the instructions for use	5.4.3
	CE Marking of Conformity	Indicates that a product has been assessed by the manufacturer and deemed to meet EU requirements	N/A
	Medical Device	Indicates the item is a medical device	5.7.7

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Qualrep Services B.V.