

# Floor Buddy | Absorbent Suction Mat

## **Pre-Surgery**

1. Remove the Floor Buddy from the polybag, unfold it, and place it on the operating room floor with the label side facing up.
2. Position the Floor Buddy with the suction tubing facing closest to the suction source.
3. Ensure the Floor Buddy is pulled taut and placed flat on the floor.
4. Connect the Floor Buddy's pre-attached tubing to the suction source.
5. Power on the vacuum source.

## **Post-Surgery**

1. Following the procedure, run the vacuum source for a few minutes to allow for residual fluid in the Floor Buddy to be vacuumed into the suction source.
2. Power off the suction source, disconnect the tubing from the suction source, and fold the Floor Buddy in half (pulling from the corners first).
3. Dispose of the Floor Buddy per facility protocol and place it in an appropriate waste receptacle.